


<b>Reorder #</b>	<b>Length</b>	<b>Unit</b>
☐101080	30 Inches	Dozen
☐101090	30 Inches	36/Cs.

### **Purpose**

Safety secures catheter tube to patient's leg. May also be used to secure and position nasogastric tubes. A comfortable alternative to skin-irritating tape.

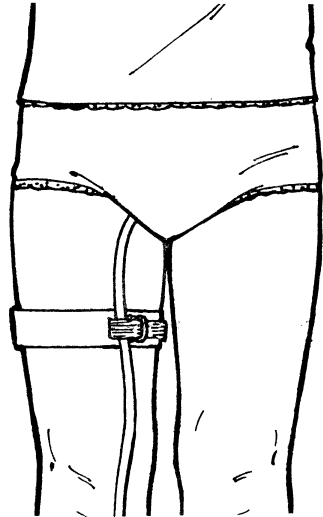

### **APPLICATION**

1. Wrap tube holder band around patient's leg or head as shown in illustrations. Cloth-backed side faces out. Band should be trimmed if there is more than a five-inch overlap.
2. Secure band by pressing "hook" tab against "loop" cloth backing.
3. Position tube at next to double ring. Bring "hook" material over tube and through open space in ring. Making certain that tube is snugly secured, press "hook" material against "loop" cloth backing.

### **Caution**

Do not wrap band so tightly as to impair circulation or cause patient discomfort.