

Anti-Reflux Wedge

Reorder #	Description	Size	Unit
☐555055	Height adjusts to 5" or 7" high	24" X 24"x7"	Ea
☐555056	Height adjusts to 8" or 10" high	24" X 24"x7"	Ea

Purpose

The anti-reflux pillow was designed to be placed on a bed for the purpose of decreasing night time reflux and heartburn. It is also recommended for asthma and emphysema sufferers. Its height could be adjusted to either 5" or 7" depending on specific patient needs. The cover is wipe clean and has a non-slip bottom to hold it in place.

Use Instructions

This Anti-Reflux pillow can be placed on the bed under the patient's head while either lying on the side or back. If desired for comfort, an additional pillow can be placed on top of the Anti Reflux Pillow. (see illustrations 1,2). As an alternative, the pillow can be placed under the mattress in the head area. If a queen or king size bed, multiple pillows may be required as necessary.

Based on comfort needs and specific patient requirements the height of the pillow can be adjusted to either 5" or 7" by removing the 2" wedge from the foam portion. (see illustration 3).

Care Instructions


The cover top is constructed from a waterproof wipe clean fabric that shouldn't be laundered. If necessary, the fabric could be hand washed and air dried. The non slip bottom also can be wiped clean using a spray and wipe cleaner.



Illust. 1



Illust. 2

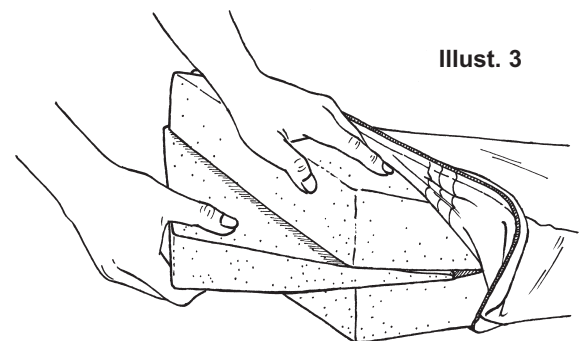

Illust. 3