



Innovative Healthcare Solutions



Extra Wide Seat: 25" W
Weight Capacity: 500 lbs.
Durable 5" Castors

Bariatric 3 Position Geri-Chair Recliner Model # GC500

Maximum weight capacity: 500lbs. | Castors should be in the locked position during patient transfer.

ASSEMBLY INSTRUCTIONS

1. Remove back assembly (C) from unassembled chair.
2. Slide the 1 piece back assembly (C) into the square tubes on the lower frame assembly (B) so that the holes in the back rest line up with the holes located on the horizontal brace of the lower frame assembly.
3. Raise the recliner lock bar (F) adjust position of back assembly (C) until the lock release button (G) positively engages middle slot lock bar link when chair is in the storage position.
4. Insert included screws through the holes of the horizontal brace and tighten all screws with a screwdriver

NOTE: Make sure that the top screw on both sides of the square tubing of the back assembly are secure before using the recliner.

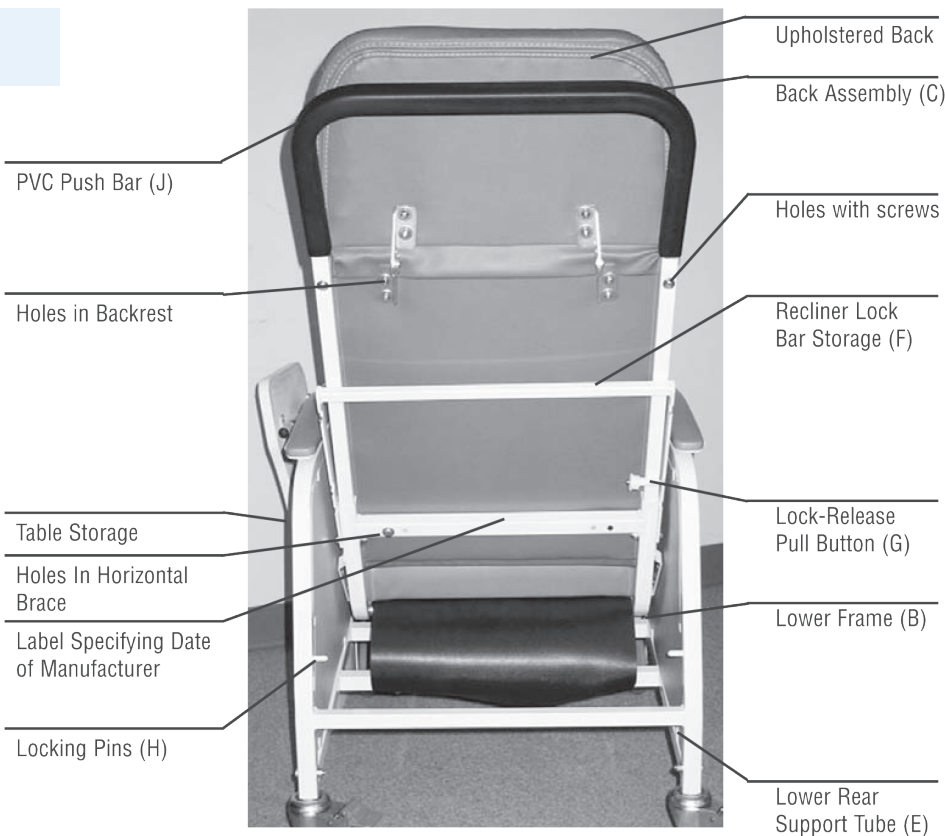


Figure 1

DIRECTIONS FOR USE

To place chair in reclining position:

BY USER:

Place hands on armrests and push against seat back. Legrest will automatically adjust to each position.

NOTE: The table can only be used in the upright sitting position. Patient should be transported in reclined position with recliner lock set.

BY NURSE OR AID:

1. Ensure recliner lock bar is in the stored position. See Fig. 1.
2. Position yourself at either side of the recliner.
3. Grasp armrest with one hand, and PVC push bar (J) with the other hand.
4. Push down on the push bar to recline the back to the first reclining position. For the second reclining position, push on push bar again.
5. To raise upright, pull forward on push bar.
6. Set recliner Lock Bar (F).

To place chair in the locked position:

TO LOCK RECLINER:

1. Place recliner into desired position.
2. Pull out Lock-Release Pull Button (G) to release Lock bar (F) from stored position.
3. Rotate recliner Lock Bar (F) down until notches are fully engaged over Locking Pins (H).

TO UNLOCK RECLINER:

1. Lift recliner Lock Bar (F) up to disengage from Locking Pins.
2. While holding Lock-Release Pull Button (G) out, rotate recline Lock Bar (F) all the way up until completely against seat back.
3. Release Lock-Release Pull Button (G) to hold recliner Lock Bar in stored position.

WARNING: Do not attempt to change the position of the chair with the recliner lock bar engaged. This could cause damage to the mechanism and will void the warranty.

DIRECTIONS FOR USE

HEADREST

Headrest can be adjusted to desired position by pulling forward or pushing back.

TABLE TOP

To Operate:

1. Underneath each arm, pull down and hold pull button (Fig. 2 A&B) and slide lockout keys forward on both sides (D).
2. Pick up the tabletop by circular table arm and slide into receiving tubes in front of each arm.
3. Tabletop can slide on and off by keeping lock/lockout key forward.
4. The tabletop can be locked into one of 5 positions. Pull down the pull button, hold the pull button and slide the key backwards. Proceed to the other arm and move that key backwards. Tabletop is now locked.

TABLE TOP STORAGE

The tabletop is stored in the small round tube underneath the left armrest, leaving the tabletop facing out.

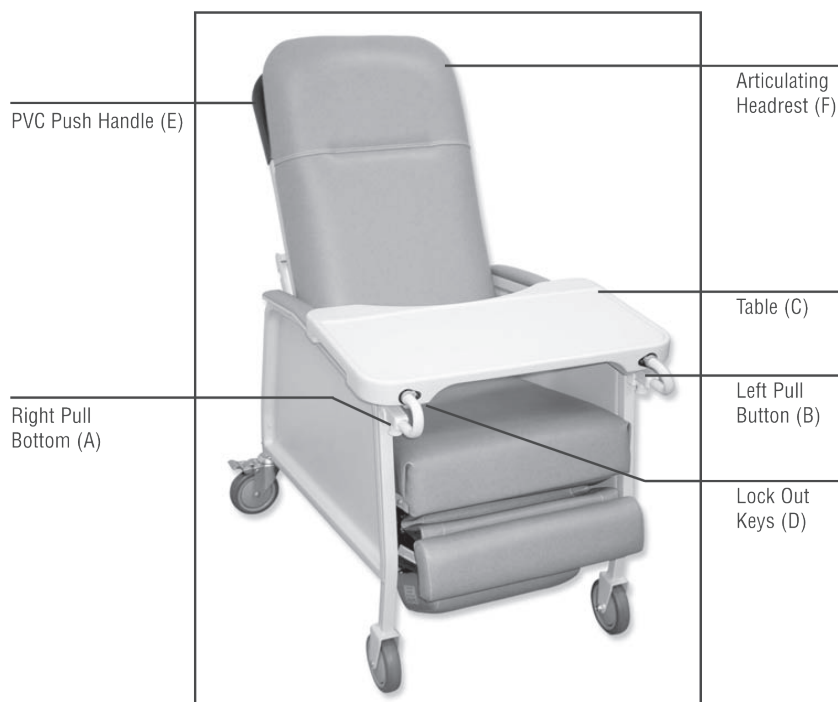


Figure 2

CAUTION:

DO NOT USE FORCE OR PRESSURE TO THE LEGREST TO OPERATE THE RECLINER. DO NOT PLACE HANDS OR LEGS IN CREASES NEAR RECLINER MECHANISM.

MAINTENANCE & CLEANING

THE FOLLOWING PARTS SHOULD BE CHECKED AND INSPECTED PERIODICALLY:

Check and Clean Casters | Tighten Nuts | Check Brakes on Casters | Check and Clean Lock Recline Mechanism

UPHOLSTERY CARE AND MAINTENANCE/STAIN REMOVAL:

Ordinary Dirt: Wash with warm water and a mild soap or detergent.

Stubborn Dirt: Using a soft bristle brush, follow directions above.

Ground-in Dirt: Scrub with powdered cleanser or similar detergent.

Chewing Gum: Scrape carefully, remove with kerosene or naphtha.

Nail Polish and Remover: Blot immediately to prevent damage.

Tars, Asphalt, Creosote: Remove to prevent staining. Clean area with kerosene or naphtha.

Warning: When using kerosene or naphtha, do so in a well ventilated area and keep away from fire and flame.

Paint, Shoe Polish, Heel Marks: Remove immediately. Use a white cloth dampened in kerosene naphtha or turpentine. Do not use paint remover or liquid brush cleaners.

Ball Point Pen: Blot immediately with a white cloth dampened in water or mineral oil.

Wax and Refinishing: Improver wearability. Use any quality paste wax.

A diluted solution of 10% (or less) of bleach or clorox may be used when cleaning the reclining chair.