


# BED LADDER

| Re-Order # | Unit |
|------------|------|
| 914154     | Each |

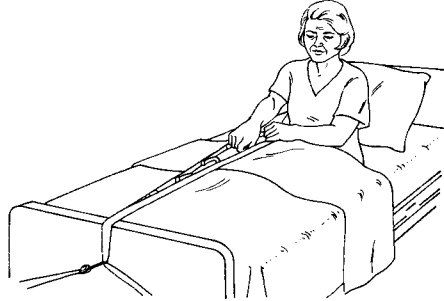

## PURPOSE

Skil-Care's Bed Ladder is designed to enable residents to pull themselves to a sitting position while in bed, or lower themselves to a supine position. It may also be used to provide residents with a mild form of exercise while in bed.

## INSTRUCTIONS

1. Wrap the webbing around the footboard supports or around the legs of the bed and secure the clip hook to the webbing as show in the illustration.
2. Using the slider, adjust the length of the webbing so that the Bed Ladder is within the resident's reach.
3. Show resident how to raise or lower himself/herself by successively grasping the ladder's web rungs using a hand-over-hand motion.

## EASY CARE

The Bed Ladder may be cleaned using a spray-type cleanser and damp cloth. A mild, non-chlorine disinfectant may be used. Do not launder.

