
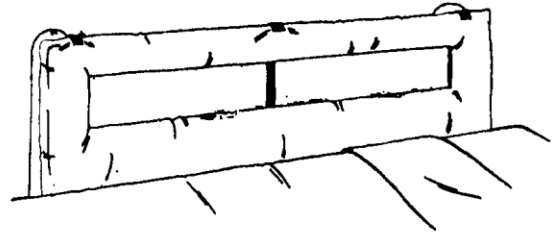


Vinyl Bed Rail Pads

Thru-View Vinyl Bed Rail Pad

Full-View Vinyl Bed Rail Pads



Full Rail (pair)

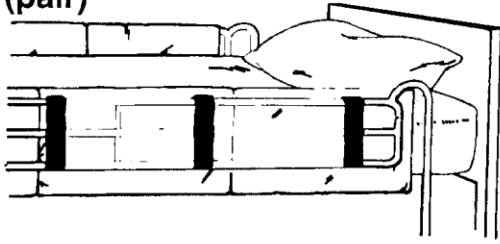

NEW Full-View Vinyl Bed Rail Pads

Reorder #	Product Description	Size
□401010	Vinyl Bed Rail Pads	15" x 1" x 60"
□401015	Thru-View Vinyl Bed Rail Pads	15" x 1" x 60"
□401040	Vinyl Bed Rail Pads	15" x 1" x 72"
□401045	Thru-View Bed Rail Pads	15" x 1" x 72"
□401050	Vinyl Bed Rail Pads	15" x 1" x 80"
□401100	Vinyl Bed Rail Pads	15" x 2" x 60"
□401140	Vinyl Bed Rail Pads	15" x 2" x 72"
□401150	Vinyl Bed Rail Pads	15" x 2" x 80"
□911525	Vinyl Bed Rail Pads (Special Order)	15" x 1" x 60"
□912200	Full-View Vinyl Bed Rail Pads	15" x 1" x 60"
□912205	Full-View Vinyl Bed Rail Pads	15" x 1" x 72"
□912210	Full-View Vinyl Bed Rail Pads	15" x 1" x 80"

Half Rail (4 Units/Package)

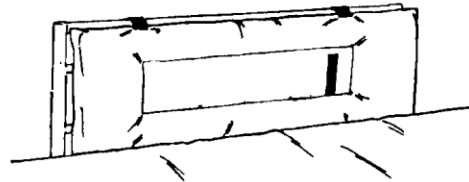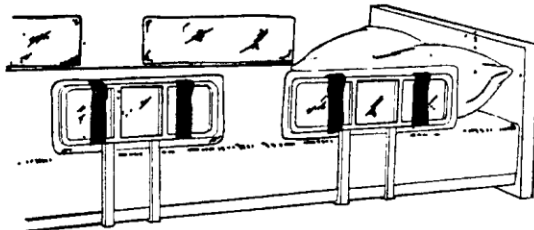

NEW Full-View Bed Rail Pads.

Reorder #	Product Description	Size
□401080	Vinyl Bed Rail Pads (Half Rail)	9" x 1" x 28"
□912220	Full-View Vinyl Bed Rail Pads (Half Rail)	15" x 1" x 29"
□912225	Full-View Vinyl Bed Rail Pads (Half Rail)	15" x 1" x 40"

Purpose

Skil-Care's bed rail pads are designed to protect patients against side rail entrapment and against skin tears and bruises resulting from contact with metal rails. These pads are not intended as substitutes for proper supervision of patients or residents for whom bed rails are being used.

Application

Place bed rail pads on patient side of rail and secure Velcro[®] over rails as shown in illustrations.

Easy Care

All Skil-Care vinyl bed rail pads may be cleaned using spray-and-wipe cleanser and a damp cloth. Mild disinfectants may be used.

Note

Bed rail pad covers are non-allergenic, mildew resistant, bacteriostatic, and conform to flammability standards set forth in California Bulletin 117B.