


Bed Positioning Wedge

| Reorder # | Product Description | Size | Unit |
|-----------|-----------------------------|--------------|------|
| □911184 | Foam only positioning wedge | 9"Dx8"Hx20"L | 6/cs |
| □911182 | Foam only positioning wedge | 9"Dx8"Hx20"L | PR |

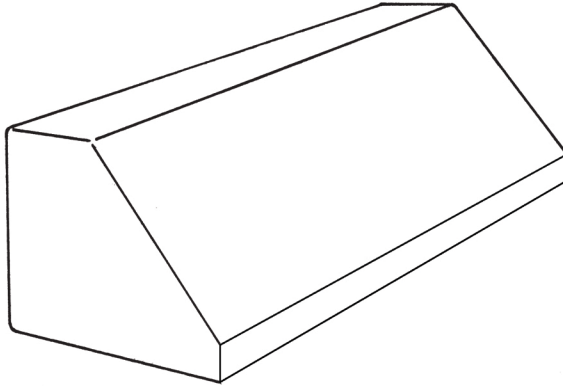

PURPOSE

Proper positioning in the bed is critical when looking to avoid pressure sores on the hipbone and coccyx. The geometry of this wedge is designed to help maintain the resident in the side lying position.

USE INSTRUCTIONS

Place this tapered wedge against the side lying resident with the vertical side facing the rails. Note that this side has the two shorter cut offs. It is recommended that the resident be turned every 2 hours or shorter if necessary. A healthcare professional should check for reddening of the skin at least every 8 hours and if reddening does occur a wound care professional should be consulted.

CARE INSTRUCTIONS

This foam wedge may be subjected to limited launderings as long as it is placed in a laundry bag maintaining a temperature less than 150°F (66°C).