

Door/Window Alarm

Reorder #	Description	Unit
□909223	Door/Window Alarm	5/pk

Purpose

Skil-Care's Door/Window Alarm is designed to alert caregivers that a door or window has been opened in a resident's room. It may also be used to monitor closets, supply rooms, stairways, cabinets and other "off-limits" areas.

Installation

1. Remove backing from adhesive strip on back of larger alarm section.
Press firmly onto door or window. Make certain to align alarm component with edge of door or window. (Illustrations A-1 & A-2).
2. Remove backing from adhesive strip on back of smaller alarm section. Align with larger section and press firmly against door frame or window frame. (Illustrations B-1 & B-2)

Test alarm before each use

1. Move selector switch to ALARM position. (Batteries are pre-installed.)
2. Open door or window. Alarm should sound. If alarm fails to sound or if signal is weak, remove battery compartment cover and replace all three batteries. (Illustration C) Use LR44 or AG-13 batteries. If alarm fails to work after batteries have been replaced, discontinue use of product.
3. Close door or window to silence and reset alarm.

Choose alarm or chime (Illustration D)


Skil-Care's Door/Window Alarm may be set for an alarm (continuous) signal or a chime (alternating) signal. This permits you to differentiate among alarm locations. If you move the selector switch from OFF to CHIME, you will hear three short beeps. It will then go silent. There will be no signal when you move the switch from OFF to ALARM.

Cautions


1. This product is intended only as an aid in patient/resident monitoring. It is not a substitute for caregiver supervision.
2. Check battery before each use. If alarm is weak or fails to sound, replace all three batteries. Use LR44 or AG-13 batteries.
3. If alarm fails to sound after batteries have been replaced, discontinue use of product.
4. Check alarm regularly to make certain that selector switch is in the CHIME or ALARM position.

Guarantee


Skil-Care's Door/Window is replacement guaranteed for six months following date of purchase. If alarm fails to work within the guarantee period, return it to the dealer from whom it was purchased. The dealer will arrange for Skil-Care to ship a replacement unit.

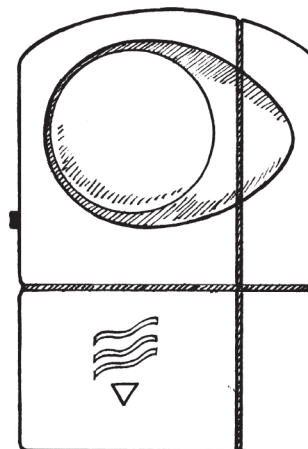


A-1

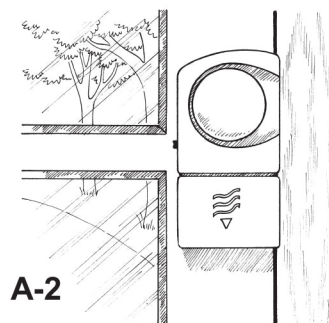


A-2

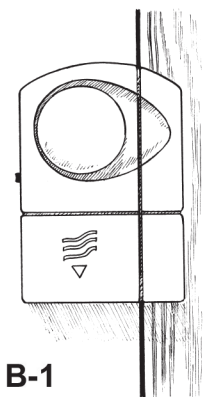


B-1

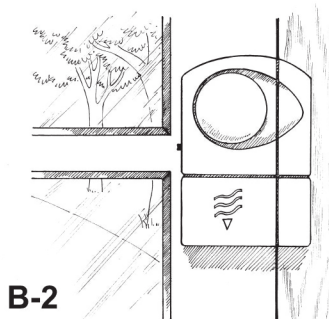

B-2

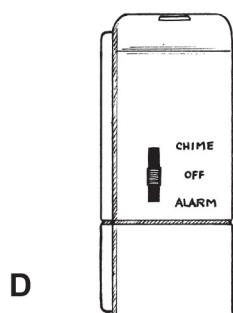


D

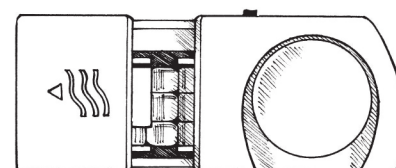

C

Window Alarm

Reorder # 909222

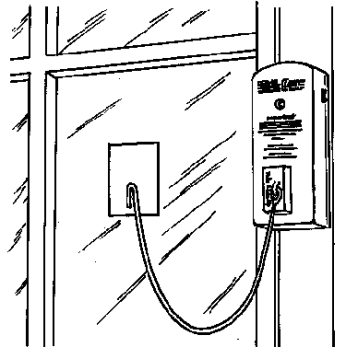

Purpose

Skil-care's Window Alarm is intended to alert caregivers when a resident/patient opens a window. It is designed to be used on a window that is fully closed or one that is left slightly open for ventilation purposes.

Warning

The Window Alarm is not intended for use on fire and emergency exit windows.

Instructions

1. Install 9-volt battery. Battery compartment is on the back of the alarm unit.
2. Separate the Velcro® loop material strip from the back of the alarm unit. Peel the backing off Velcro® strip and attach it to the window frame.
3. Attach the alarm to the window frame by pressing the hook material on the alarm against the loop material on the window frame.
4. Separate the Velcro® strips attached to the magnet pull-cord. Peel the backing off the loop material and attach it to the windowpane. Press the hook material on the pull-cord to the loop material on the window.
(See illustration above for proper positioning of alarm unit and window tab.)
5. Move the alarm switch to the HI or Low position. The alarm is now activated. The alarm is deactivated by moving the switch to the OFF (center) position.

Important

Test the Window Alarm before each use. Move the switch to the HI position and remove the magnet. If alarm is weak or fails to sound, remove the battery and replace it with a fresh one. Retest. If alarm still fails to sounds, discontinue use.