

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

hamper stand



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

assembly instructions for hamper stand

Step #1 Fasten locking casters to the front end of base. Fasten non-locking casters to the rear end of base.

Use (1) each of the $\frac{3}{8}$ " lockwasher (K) and the $\frac{3}{8}$ " hex nut (J).

Step #2 Insert the (2) tubular uprights (B) until their snap pins interlock with the pin holes in base (A).

Step #3 Attach the tubular top frame and lid subassembly (C) the opposite ends of the uprights (B) until the snap pins interlock with the hole in the lid and frame assembly.

Step #4 Assemble bag holder rod (E) through the uprights and secure ends using the (2) $\frac{1}{4}$ " lock nuts (G). This rod can be used to give additional hanging support for certain bag designs. Its use is optional.

Step #5 Assemble one end of each connecting rod (D) through the holes in lid (C). Assemble the opposite ends of the rod into the foot pedal. The rod ends must face inward. Secure the ends with the $\frac{5}{16}$ " lock nuts (L).

