Limited Lifetime Warranty

Your Inspired by Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Inspired by Drive device or this warranty, please contact an authorized Inspired by Drive dealer.

WARNING: This product can expose you to chemicals, including diisononyl phthalate (DINP), which is known to the State of California to cause cancer. For more information go to www.P65Warnings.ca.gov.



2021 Medical Depot, Inc. All rights reserved.
Drive DeVilbiss Healthcare
99 Seaview Boulevard
Port Washington N.Y. 11050 USA
Made in China



NIMBO POSTERIOR WALKER

Item#

KA1200-2GCR, KA1200-2GKB, KA1200-2GWP, KA2200-2GCR, KA2200-2GKB, KA2200-2GWP, KA3200-2GCR, KA3200-2GKB, KA3200-2GWP, KA4200-2GCR, KA4200-2GKB, KA4200-2GWP, KA5200-2GEB






EU Authorized Representative Y. Sung Handelsvertretung Duesselthaler Str. 24 40211 Duesseldorf Germany

www.inspiredbydrive.com


user instructions

- Remove Nimbo from carton.
- To unfold the Nimbo, pull handgrips and cross brace in opposite directions. Continue to unfold until the push pins "click" into place.
- To install the legs, match the leg numbers with the corresponding frame numbers. Depress the push pins on the legs and slide into the frame. When locked into position, the push pins will "CLICK" through the frame.
- Adjust height of walker by pressing push pins on each leg and slide leg to desired position.

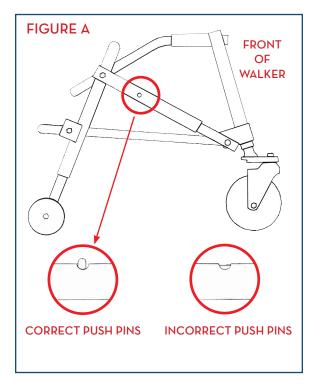


user information

 Lift and position knob (A) to set front wheels in either the swivel or locked position.

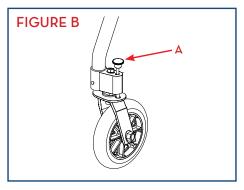

- To fold the Nimbo, face it from the open end, depress the push pins, tilt walker so that it rests on its rear legs only and push the front legs toward the rear legs.
- To override the one-directional feature of the rear wheels, lift up aluminum pin and insert into override bracket

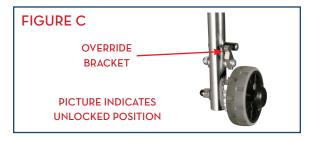

maintenance and care

- Inspect the Roller regularly. Tighten the nuts and bolts as necessary. If a product needs service or replacement parts, contact an authorized Drive dealer.
- To clean the Roller, use warm water with a mild detergent. Wipe clean with a soft, dry cloth.