Nitro™ 3-wheel rollator

Andador rollator Nitro™
de 3 ruedas

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item # RTL10266TWHL

SUNGO Europe B.V.
Olympisch Stadion 24, 1076DE
Amsterdam, Netherlands

www.drivemedical.com

drive™

parts list

WEIGHT CAPACITY: 300 LBS (136 KG)

- | | |
|-----------------------------------|-----------------------|
| 1. Handgrip | 6. Folding hinge |
| 2. Adjustable tubing handgrip | 7. Bag |
| 3. Rear reflector on handgrip | 8. Front wheel |
| 4. Brake lever | 9. Brake adjust screw |
| 5. Interlock button for handgrips | 10. Rear wheel |

introduction

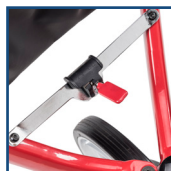
The Walking aid is suitable for a single user with limited mobility. The Nitro™ Rollator is ideal for indoor and outdoor use.

Nitro™ Rollator includes:

1. Bag
2. Operating Instructions

unfolding the Nitro™ Rollator

To unfold the Rollator, position the Nitro™ upright on all three wheels. Hold each hand grip and pull outward until the Nitro™ rollator unlocks and opens fully (figure 1-2). Gently push down on locking handle located near the front wheel. An audible “click” confirms that the folding hinge is fully engaged in lock position.

height adjustment

Press the red release button to adjust handgrips to desired height, as shown in figure 3. As the handle moves, release the button. This automatically locks in the next possible height. Repeat this process until the desired height is reached.

removing / attaching the carry bag

To remove bag, push down quick release tab and gently pull up on the carry bag handle (figure 4). Repeat with other side.

To replace bag, simply insert quick release tab back into the buckle. An audible “click” confirms the bag is secured in place.

braking / locking

Pull both brake levers upwards to apply the brakes while the Nitro™ Rollator is moving (see figure 5). Always brake slowly and steady.

Press both brake levers down to apply the locks while the Nitro™ Rollator is stationary (see figure 6). Pull up the brake levers and release to release the locks.

The effect of the brakes can be affected by abrasion of the wheels.

Regularly test the brakes.

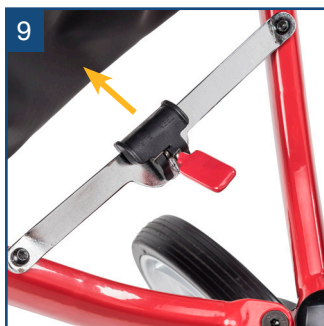
adjustment of brakes

To adjust the braking strength hold the nut firmly in place. To loosen, turn the brake adjuster clockwise (figure 7) towards the inside of the frame. To tighten, turn the brake adjuster counter-clockwise (figure 8) towards the outside of the frame.

folding / transport / storage

To fold, stand behind the rollator and pull the locking handle towards you until folding hinge is disengaged (figures 9 - 10). Push sides of frame together until rollator is flat.

curb assist

The Nitro™ Rollator features a Curb Assist for terrain transitions. The Curb Assist allows the user to move over curbs and small steps with ease. The Curb Assist is visibly located on the inside of the rear wheel (figure 11).

How to Use the Curb Assist:

1. When approaching a curb, step or transition firmly grasp both hand grips and place your right foot on the Curb Assist located on the right-hand side of the rollator. Be sure to use your right foot to activate the Curb Assist. DO NOT use your opposite foot.
2. While securely grasping both hand grips, gently apply pressure to the Curb Assist with your foot
3. Bring the front wheels of the rollator to a slight angled position just above the curb or step pushing forward if necessary
4. Once at a stable position, gently release your foot from the Curb Assist allowing the front wheels to rest of the curb, step or transition
5. Push the Rollator forward as the rear wheels approach the curb, step or transition and lift the rear wheels onto the curb, step or transition
6. Proceed with walking once observing that all four wheels are safely past the curb, step or transition

safety instructions

- Use only as a walking aid!
- Use only on even and solid surfaces.
- Maximum user weight: 300 lbs.
- Regularly check the screws and fasteners on the Nitro™ Rollator and retighten if necessary.
- Use bag only for transport of objects and do not overload. Maximum load: 11 lbs.
- Do not expose the product to temperatures above 104°F for an extended period of time, as this can cause the Nitro™ Rollator parts to overheat and burn the user.
- Please make sure the Rollator is fully opened, with locking handle and folding hinge in locked position.
- Be sure that height adjustment is secured in place and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
- Test the brake lever before operating the Rollator. When wheels are locked, lock bar on rear wheels should prevent wheels from turning. When locks are released, the wheels should turn freely.
- Replace tires when significantly worn or damaged.

cleaning

Clean regularly by hand with a damp cloth. Do not use aggressive detergents.

The Nitro™ features antimicrobial* treated handgrips to help minimize the growth of bacterial odors, mold, and fungi on treated surfaces.

*Antimicrobial properties are built-in to protect the products. These products do not protect users or others against bacteria, viruses, germs or other disease organisms.

maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more info.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2020 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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the 1990s, the number of people who have been employed in the public sector has increased in all countries. The increase has been particularly large in the United States, where the public sector has grown from 15% of the total labour force in 1970 to 25% in 1995. In the Netherlands, the public sector has grown from 12% of the total labour force in 1970 to 18% in 1995.

The increase in the public sector has been driven by a number of factors. One of the main factors is the increase in the number of people who are employed in the public sector. This is due to a number of reasons, including the fact that the public sector has become a more attractive place to work. This is due to a number of factors, including the fact that the public sector has become a more stable place to work, and that it offers a better work-life balance.

Another factor is the increase in the number of people who are employed in the public sector. This is due to a number of reasons, including the fact that the public sector has become a more attractive place to work. This is due to a number of factors, including the fact that the public sector has become a more stable place to work, and that it offers a better work-life balance.

A third factor is the increase in the number of people who are employed in the public sector. This is due to a number of reasons, including the fact that the public sector has become a more attractive place to work. This is due to a number of factors, including the fact that the public sector has become a more stable place to work, and that it offers a better work-life balance.

A fourth factor is the increase in the number of people who are employed in the public sector. This is due to a number of reasons, including the fact that the public sector has become a more attractive place to work. This is due to a number of factors, including the fact that the public sector has become a more stable place to work, and that it offers a better work-life balance.

A fifth factor is the increase in the number of people who are employed in the public sector. This is due to a number of reasons, including the fact that the public sector has become a more attractive place to work. This is due to a number of factors, including the fact that the public sector has become a more stable place to work, and that it offers a better work-life balance.

A sixth factor is the increase in the number of people who are employed in the public sector. This is due to a number of reasons, including the fact that the public sector has become a more attractive place to work. This is due to a number of factors, including the fact that the public sector has become a more stable place to work, and that it offers a better work-life balance.

A seventh factor is the increase in the number of people who are employed in the public sector. This is due to a number of reasons, including the fact that the public sector has become a more attractive place to work. This is due to a number of factors, including the fact that the public sector has become a more stable place to work, and that it offers a better work-life balance.

An eighth factor is the increase in the number of people who are employed in the public sector. This is due to a number of reasons, including the fact that the public sector has become a more attractive place to work. This is due to a number of factors, including the fact that the public sector has become a more stable place to work, and that it offers a better work-life balance.

drive[™]

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© 2020 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.