

---

## caution

---

Adult supervision is required at all times.  
The straps and supports must be carefully  
adjusted for the comfort and security of the child.  
Child must be strapped in at all times.

---

## assembly

---

Your Otter Bath Chair comes in one carton completely  
assembled. This carton contains: Bath chair, head  
cushion strap, positioning and leg straps.  
If your shipment is incomplete, please call a Inspired by  
Drive authorized dealer.

---

## optional accessories

---

### **Tub Stand**

Item # OT 8010, OT 8010XL

### **Shower Stand**

Item # OT 8020 (NOT AVAILABLE FOR XL)

### **Soft Fabric Kit for OT 1000**

Item # OT 1002, OT-1002-LL, OT-1002-BB

### **Soft Fabric Kit for OT 2000**

Item # OT 2002, OT-2002-LL, OT-2002-BB

### **Soft Fabric Kit for OT 3000**

Item # OT 3002, OT-3002-LL, OT-3002-BB

### **Soft Fabric Kit for OT 4000**

Item # OT 4002, OT-4002-LL, OT-4002-BB

---

## limited 1 year warranty

---

Your Inspired by Drive branded product is warranted  
to be free of defects in materials and workmanship for  
the lifetime of the product for the original consumer  
purchaser.

This device was built to exacting standards and  
carefully inspected prior to shipment. This Lifetime  
Limited Warranty is an expression of our confidence  
in the materials and workmanship of our products and  
our assurance to the consumer of years of dependable  
service.

This warranty does not cover device failure due to  
owner misuse or negligence, or normal wear and  
tear. The warranty does not extend to non-durable  
components, such as rubber accessories, casters, and  
grips, which are subject to normal wear and need  
periodic replacement.

If you have a question about your Inspired by Drive  
device or this warranty, please contact an authorized  
provider.

**⚠ WARNING:** This product can expose you to chemicals,  
including Styrene, which is known to the State of  
California to cause cancer. For more information go to  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).



2022 Medical Depot, Inc. All rights reserved.  
Drive DeVilbiss Healthcare  
99 Seaview Boulevard  
Port Washington N.Y. 11050 USA  
Made in China



## otter bath chair

OT 1000

OT-1000SF-BB

OT-1000SF-LL

OT 2000

OT-2000SF-BB

OT-2000SF-LL

OT 3000

OT-3000SF-BB

OT-3000SF-LL



EU Authorized Representative  
Y. Sung Handelsvertretung  
Duesselthaler Str. 24  
40211 Duesseldorf Germany

[www.inspiredbydrive.com](http://www.inspiredbydrive.com)

## maintenance and care

1. Inspect the chair periodically for cracks, breaks, loose or missing parts or malfunctions. Check the fabric for tears or worn areas. If a product needs service or replacement parts, contact an authorized Inspired by Drive dealer.
2. Wash chair and fabric with soap and water or 10% bleach. Do not use pine oil cleaners. Fabric and straps may be removed and machine-washed. Engage hook and loop before washing. Fabric cover may be washed on cool setting and drip-dried. Straps may be washed in warm water and dried on cool setting. Do not iron.

## adjustments

See Figure 1

1. Head cushion strap height can be adjusted by opening strap in the back and positioning it at desired height. Securely engage hook and loop of strap once positioned. Slide head cushions (A) along the strap to adjust width.
2. Straps (D) can be adjusted as needed by inserting in any of the loops along the entire unit.
3. To adjust legs (E), pull crossbar (F) outward, rotate legs & push in to engage.
4. To adjust seat angle, pull on Uni-Bar under the seat (Fig 2) rotate seat to desired position and release Uni-Bar. To adjust back angle using Uni-Bar behind the Otter back (Fig 3).

## caution

Adult supervision is required at all times. The straps and supports must be carefully adjusted for the comfort and security of the child. Child must be strapped in at all times.

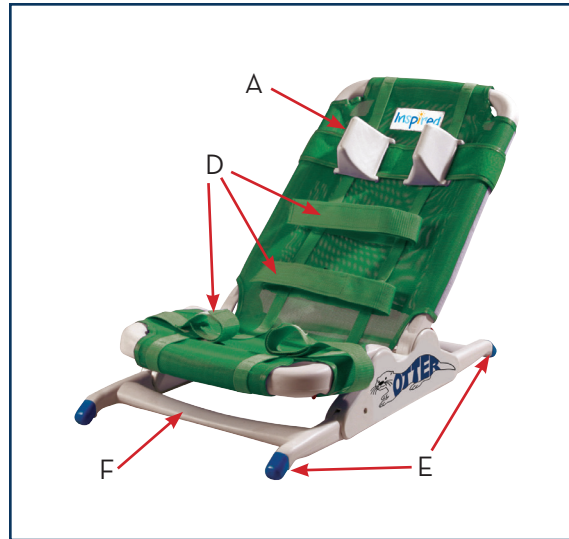


figure 1



figure 3



figure 2