

## limited 1 year warranty

Your Inspired by Drive branded product is warranted to be free of defects in materials and workmanship for 1 year from the date of purchase for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Limited 1 Year Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Inspired by Drive device or this warranty, please contact an authorized Inspired by Drive dealer.

**⚠ WARNING:** This product can expose you to chemicals, including bisphenol A (BPA), which is known to the State of California to cause birth defects or other reproduction harm. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).




2022 Medical Depot, Inc. All rights reserved.  
Drive DeVilbiss Healthcare  
99 Seaview Boulevard  
Port Washington N.Y. 11050 USA  
Made in China



## forearm crutch

item #FC120-GB  
#FC120-GP  
#FC300-2GB  
#FC300-2GP

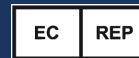

EU Authorized Representative  
Y. Sung Handelsvertretung  
Duesselthaler Str. 24  
40211 Duesseldorf Germany

[www.inspiredbydrive.com](http://www.inspiredbydrive.com)

## use and adjustments

The forearm crutch is intended to be used as a mobility aid for those that need additional support to prevent falls. Forearm crutches are to be used bilaterally with forearms placed through forearm collars, and hands gripping the hand grips. Height should be adjusted to provide proper support to prevent user from losing balance. The user can then use the forearm crutches by moving each crutch forward in an alternating pattern. All components/parts required to use the forearm crutches are included with the crutch. Forearm Crutch includes main crutch shaft, crutch tip, hand grip, anti rattle collar, and forearm collar.

## adjusting leg height

1. Before use, adjust height of leg to your specific needs by depressing the push button and loosening the anti rattle collar (Figure 1).
2. When desired height is reached, lock leg into place by aligning button into proper hole, (Figure 2).
3. To be sure leg is locked into place by pushing down on or by pulling leg.

NOTE: The small and medium sizes have been combined into one universal size (FC120-GB/GP).

## cleaning instructions

1. Use a non-abrasive mild detergent or a household cleaner with warm water.
2. Rinse with warm water and wipe dry.
3. **DO NOT USE AN ABRASIVE CLEANER OR CLOTH** with this product as damage may ensue.

FIGURE 1

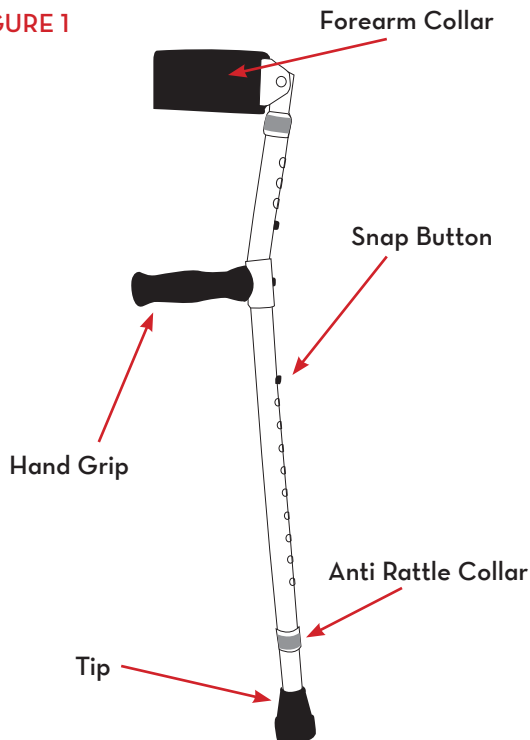


FIGURE 2

