www.drivemedical.com

drive
WWW.DRIVEMEDICAL.COM

premuim series bath bench
with back and arms

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EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com

drive
WWW.DRIVEMEDICAL.COM

Installation Instructions

NOTE: Before assembly, insure that all parts and quantities match parts list. Inspect all components for shipping damage. If any parts are determined to be missing or damaged, contact your dealer/retailer for replacements.

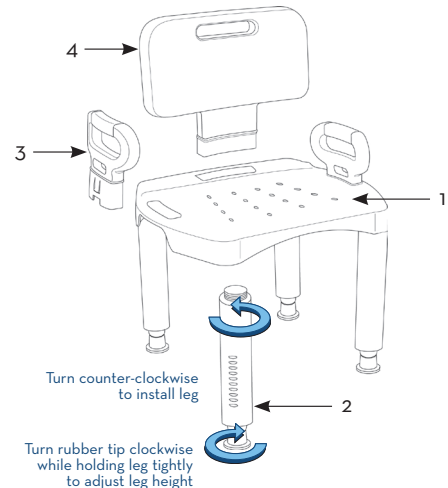
WARNING!

DO NOT use this equipment without first reading and understanding its instruction sheet. If you are unable to understand these instructions, contact a healthcare professional, dealer, or technical person before installing. Otherwise, injury or damage may occur. Use arms for assistance only.

PARTS LIST:

1. Seat - 1 each
2. Leg - 4 each
3. Arms - 2 each
4. Back - 1 each

To adjust legs for proper height or uneven surfaces, grab the bath bench leg with one hand while rotating the bottom of the leg (where the suction cup attaches to the leg) clockwise. The leg has an adjustment window that indicates the level with a red marker. Turn to desired height and repeat for all legs.

assembly instructions

1. Place seat (1) upside down on a flat surface. Insert leg (2) into the corner recess of the seat and rotate counter-clockwise until tight. Repeat step #1 for all 4 legs.
2. For model with arms (3) place chair upright and insert arms into holes in side of seat until arm is fully engaged.
3. For models with a back (4) place back in the slot, which is in the rear of the seat, and insert back until engaged.

disassembly instructions

1. The back has a lock tab that, when depressed, will allow the back to be pulled out of their slots freely.
2. To remove arms, pinch together the tabs on the arm which, when assembled, will be under the seat.
3. Turn the legs clockwise until removed from bench.

care and maintenance

1. Do not use an abrasive cleaner. Use a non-abrasive cleaner with warm water.
2. Periodically inspect all components for damage or looseness.

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lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in USA

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© 2005 Medical Depot, Inc. Todos los derechos reservados.
Drive es marca registrada de medical depot, Inc.
© 2005 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.