PreserveTech™ Secure  
Lock Raised Toilet Seat

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Item # RTL12C003-WH

 **PreserveTech™**  
Stays Cleaner • Lasts Longer

Y. Sung Handelsvertretung  
Duesselthaler Str. 24  
40211 Duesseldorf Germany

[www.drivemedical.com](http://www.drivemedical.com)

**drive™**

## parts/components

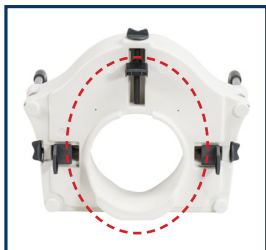
- A. Seat
- B. Armrests

**WEIGHT CAPACITY: 350 LB / 158 KG**

## assembly instructions

1. To install arms, depress push pins on armrests (B) and insert into mounting tubes. Make sure armrests are locked in place by pulling them.
2. Raise existing toilet seat and lid assembly; place rear wing of seat (A) around rim of toilet bowl.  
NOTE: Some toilet bowls may require the seat assembly to be removed to allow proper fit. In addition, the front adjustment brace must clear the rim of the toilet for the raised toilet seat to fit securely.
3. With seat (A) resting correctly on rim, straighten and center product as necessary. Turn adjustment knobs to tighten product securely in place.

NOTE: Rubber pads on front lock should touch the inside of the toilet bowl and rubber pads on the left and right locks should touch the outside of the toilet bowl.

NOTE: Do not use tools to tighten adjustment knob. Be careful not to over-tighten, as threads on adjustment brace may strip.

## adjusting armrests

Before use, adjust height of each armrest to your specific needs, making sure each arm is set at the same height. When desired height is reached, lock arms into place by aligning push pins into holes. Push pins will click into place when secure.

## additional precautions

- Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
- Check for secure fit routinely before using product. User's weight should be centered over toilet seat, not too far forward or too far to one side, as unit may tip.
- Use the arm supports only for assistance. DO NOT attempt to use the arms to support full body weight.

## maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit [www.drivemedical.com/help](http://www.drivemedical.com/help) for more info.

**NOTE:** To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Drive PreserveTech™ products are manufactured with an antimicrobial\* agent that helps to minimize the growth of stain and odor-causing bacteria and mold on treated plastic surfaces.

\*Antimicrobial properties are built in to protect the products. These products do not protect users or others against bacteria, viruses, germs or other disease organisms.

## lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

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© 2021 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.