

## maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit [www.drivemedical.com/help](http://www.drivemedical.com/help) for more info.


## lifetime limited warranty

Your Drive branded product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

 **WARNING:** This product can expose you to chemicals, including bisphenol A (BPA), which is known to the State of California to cause birth defects or other reproductive harm. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).



© 2020 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

wheeled drop arm  
commode w/  
4 locking casters  
and footrests

item # 11120SV-1F



SUNGO Europe B.V.  
Olympisch Stadion 24  
1076DE Amsterdam, Netherlands

[www.drivemedical.com](http://www.drivemedical.com)

**drive**<sup>TM</sup>

## commode assembly instructions and precautions



**WEIGHT CAPACITY: 350 LBS/158 KG**

## assembly instructions

1. Before assembling seat, allow toilet seat to reach room temperature if exposed to cold. This will help prevent the seat clamps from cracking while being assembled onto commode back cross brace.
2. Assemble the backrest by inserting the supporting poles into the back of seat frame. Secure in place using the lock knobs provided.
3. To drop arm, depress release button on arm and gently allow to drop.
4. Align the toilet seat clamps onto the back frame rail (both clamps should sit outside the rail brackets). Push down on seat until clamps snap firmly into place.
5. When removing the seat grasp the back of the seat and lift. Install commode bucket by placing in bucket frame. Reverse to remove.
6. Assemble the footrests by inserting the footrests into the top of the front legs from the outside of the frame. Once inserted, turn the footrests inward until they lock in position.

## additional precautions

- Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a healthcare, dealer or technical professional before attempting to install. Otherwise, injury or damage may occur.
- Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
- Ensure ALL screws, nuts and/or bolts are tightened.
- All four legs MUST touch the floor simultaneously at all times.
- Users with limited physical strength should be supervised or assisted while using commode.
- Use the arm supports only for assistance. DO NOT attempt to use the arms to support full body weight.
- Arms should be unlatched and dropped to assist in transfer onto/off the device.