


Skil-CareTM
CORPORATION

29 Wells Avenue
Yonkers, New York 10701 914-963-2040 800-431-2972

ECONOMY LIMB HOLDERS

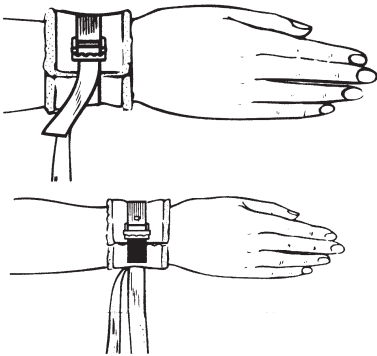

Reorder #	Fabric	Description	Size	Unit
<input type="checkbox"/> 306040	Polyfoam	Single Tie	Universal	36 Pr./Cs.
<input type="checkbox"/> 306041	Polyfoam	Double Tie	Universal	36 Pr./Cs.
<input type="checkbox"/> 306044	Polyfoam	Single Tie	Universal	Pair
<input type="checkbox"/> 306045	Polyfoam	Double Tie	Universal	Pair
<input type="checkbox"/> 306048	Synthetic Sheepskin	Single Tie	Universal	36 Pr./Cs.
<input type="checkbox"/> 306081	Polyfoam	Single Tie	Universal	36 Pr./Cs.

Caution: Federal law restricts this device to sale by or on the order of a physician or licensed health care professional.

Indications - Skil-Care Economy Limb Holders are designed for patients who require:


- Limitations placed on hand/arm movement
- Limitations placed on leg movement

Contraindications for Use - Do not use the Skil-Care Economy Limb Holders:


- For violent or psychiatric patients
- For patients in bed other than hospital-style bed fitted with side rails

Application

1. Wrap cuff around patient's wrist or ankle so that the end with the buckle overlaps the end without the buckle. Position the cuff by pressing the "hook" tab against the "loop" material on the cuff.
2. Wrap wrist strap around cuff. Secure the wrist strap through the buckle. The wrist strap should be snug but not so tight as to cut off circulation to the hand or foot. You should be able to slip one finger between the cuff and the patient's wrist or ankle.
3. Secure the tie straps to the bed frame.

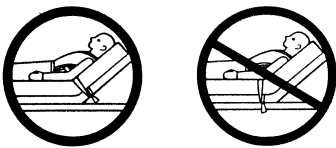

Always secure straps to the part of the bed that moves when patient is raised or lowered. Never attach straps to bed rails or any part of the bed that does not move when raised and lowered.

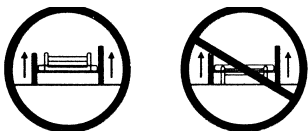

Always raise and lock bed rails when patient is wearing limb holders. Never leave bed rails in lowered position when patient is unattended.

See **important warnings and cautions** on reverse side.

Laundering

Do not launder these limb holders. They are single-patient use devices that are to be discarded after use.

See important warnings and cautions below.

Always...

Always inspect condition of device.



Always read package insert to determine if device is intended for wheelchair and/or bed use.



Always monitor patients in restrictive devices. Make certain to follow your facility's procedures for periodic release, toileting and exercise.



Never...

Never use a device that has worn or frayed fabric or webbing. Discard damaged units immediately. Do not alter products.



Never use a Skil-Care device as a restraint in a motor vehicle.



Never place a patient wearing a restrictive device in a place where he/she is not easily monitored.



t(Date of issue: 8/27/19)