


## GERI-CHAIR GEL OVERLAY

Reorder #	Unit
703001	Each

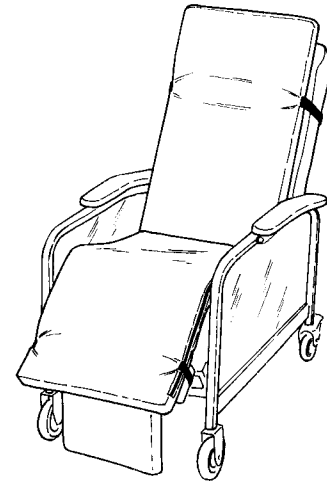

### Purpose

Skil-Care's Geri-Chair Gel Overlay uses a water-based gel pod to relieve interface pressure at the coccyx and ischials. It has foam padding to cushion and leg rests.

### Two-sided cover

The Geri-Chair Gel Overlay cover has two sides: One made of new "3Mesh" fabric that provides cooling airflow, and the other made of fluid-proof nylon. We recommend the use of the nylon side for incontinent residents.

### Application

1. Place the overlay on the geri-chair with the upper (longer) section against the backrest. Place the strap around the backrest and secure buckle. Pull strap through buckle to tighten. (Illustration A)
2. Secure the lower strap around the footrest. (Illustration B)

### Easy care

- 1, Both sides of the overlay may be wiped cleaned with a damp cloth. A mild non-bleach disinfectant may be used.
2. The overlay cover may also be laundered. **Be certain to remove both the upper and lower foam pads and the water-based pod before laundering.** The cover may be washed and dried at temperatures that do not exceed 180° F (82°C). Do not use bleach.

