
HEEL-FLOAT WALKER BOOT

Reorder #	Size	Unit
503142	Small	Each
503143	Medium	Each
503144	Large/Bariatric	Each
503095	Replacement Gel Pack(small)	Each
503097	Replacement Gel Pack(mediún	n)Each
503099	Replacement Gel Pack(large)	[′] Each

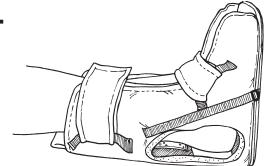

Purpose

Ski-Care's Heel-Float Walker Boot floats the heel over an air cavity to prevent pressure sores from forming or aid in the healing of existing sores. It includes a detachable non-slip tread pad that permits limited ambulation and facilitates bed/wheelchair transfer. The foot support combines a rigid plastic insert with flexion-limiting straps that maintain the foot at a 90° angle. A gel insert beneath the ankle relieves pressure and cools the skin.

Application

The Heel-Float Walker Boot may be used for either the right or left foot. Select he proper size for your patient.

Bed

- 1.Remove the detachable non-slip tread pad (Illustration B). For infection control purposes, the tread pad must be removed before patient's foot is placed on the bed.
- 2. Place the patient's lower extremity in the boot. Make certain that the heel is suspended over the air cavity (Illustration A).
- 3. Secure the Velcro^R ankle and toe straps as shown in illustration A.

Ambulation and Transfer

- 1. Seat patient on side of bed.
- 2. Attach non-slip tread pad (Illustration B).
- 3. Ambulate or transfer patient according to your facility's policies and procedures.
- 4. Remove tread pad before placing patient's foot on bed.

Important

Remove the Heel Float every 8 to 12 hours and check for ischemia or discoloration.

Easy Care

The Heel Float Walker Boot is launderable and may be washed and dried at temperatures that do not exceed 180°F (82°C). Do not use chlorine bleach. For infection control purposes, the outer nylon cover should be cleaned using a spray-and-wipe cleanser and disinfected with a non-chlorine disinfectant when patient is returned to bed.

Remove gel insert (Illustration C) and plastic foot support insert (Illustration D) before laundering. Guarantee

The Heel-Float Walker Boot is guaranteed against defects in materials and workmanship under conditions of normal use for six month following date of purchase.

