

Re-Order #	Description	Chair Width	Unit
704311	J-Hook Drop Seat Only	16" Wide WC	Ea.
704310	J-Hook Drop Seat Only	18" Wide WC	Ea.
704312	J-Hook Drop Seat Only	20" Wide WC	Ea.
704315	J-Hook Drop Seat w/Gel Cushion	16" Wide WC	Set
704320	J-Hook Drop Seat w/Gel Cushion	18" Wide WC	Set
704321	J-Hook Drop Seat w/Gel Cushion	20" Wide WC	Set


Re-Order #	Description	Chair Width	Unit
704325	Replacement Cushion	16" Wide WC	Ea.
704330	Replacement Cushion	18" Wide WC	Ea.
704331	Replacement Cushion	20" Wide WC	Ea.
704348	Replacement J-hooks	Universal	4 PK

Purpose

The J-Hook Drop-Seat lowers the wheelchair seat up to four inches permitting the wheelchair user to reach the footrests or floor. It also lowers the center of gravity, reducing the possibility of the chair tipping over. The drop-seat base can be set to a level or inclined position to meet the user's positioning needs. Designed for convenience, the J-Hook Drop-Seat easily lifts off the wheelchair frame when the chair must be folded for travel or storage. Weight tested to 400 pounds.


Easy Assembly

Remove all of the screws holding the canvas seat to the wheelchair and put a side. Adjust the dropseat to the proper height. Place the drop seat evenly located on the chair. If desired to keep the front higher to prevent sliding adjust accordingly. Using one screw each, screw the hold down brackets (#4) so that they overlap the dropseat bracket.

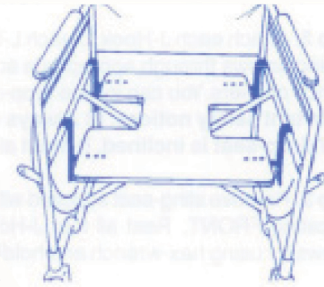

Four packets, each containing:

Note: A hex-wrench is included in one of the packets

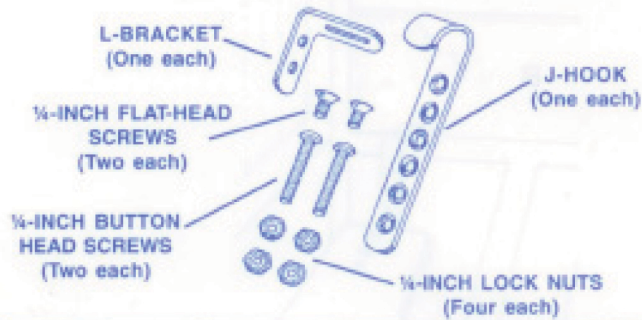

One packet containing:

