



Re-Order Number	Description	Size	Units
503150	Heels-Off (Medium-Large)	24" x 16" x 7"	Each
503151	Heels-Off (Small)	24" x 13" x 5.5"	Each
503165	Heels-Off w/Gel (Medium-Large)	24" x 16" x 7"	Each
503166	Heels-Off w/Gel (Small)	24" x 13" x 5.5"	Each
503167	Heels-Off (Bariatric)	26" x 16" x 7"	Each
503198	Replacement Cover for Heels-Off	Small/Medium	Each
503199	Replacement Cover for Heels-Off	Medium/Large	Each
503200	Replacement Cover for Heels-Off	Bariatric	Each

### Purpose

Heels-Off is designed to off-load both heels to prevent pressure sores from forming and can aid in the healing of existing ulcers. It elevates the legs to relax the spine and prevents leg and lower back pain.

In addition to the benefits of the standard Heels Off, the new Heels Off w/Gel has a surface layer of gel to provide enhanced pressure relief and therapeutic cooling for the lower leg.

### Easy to Use

Place the higher end of the Heels-Off device under the patient's knees while suspending the heels

### Easy Care

The Low-Shear II cover could be wiped clean with a spray and wipe cleanser. If disinfected, avoid using chlorine (bleach) chemicals. Do not launder.