

Health o meter[®]
P r o f e s s i o n a l

Model YG500R Mechanical Diaper/Small Platform Scale



ISO13485:2003
CERTIFIED

User Instructions



PELSTAR, LLC 9500 West 55th Street, McCook, IL 60525

P/N UMYG500R Rev20150630

Thank you for your purchase of this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

TABLE OF CONTENTS

Specifications for This Scale3
Initial Set Up4
Operating Instructions4
Maintenance.....5
Warranty.....6

Please register your scale for warranty coverage at:

www.homscales.com

For User Instructions updates and revisions please go to:

www.homscales.com

SPECIFICATIONS FOR THIS SCALE

YG500R Scale Specifications

Capacity:	500g	
Graduation	2g	
Physical Dimensions:	Scale (Overall Dimensions) Width: 6-2/3" (16.8cm) Depth: 7" (17.8 cm) Height: 8-1/2" (21.6 cm) Weight: 3 lb (1.4 kg)	Platform (Weighing Area) Width: 6" (15.2 cm) Depth: 4-2/3" (11.7 cm)

YG500R Scale Disposal



This Health o meter[®] Professional scale must be disposed of properly as steel waste. Follow the national, regional or local regulations which apply to you for disposal of steel waste.

CUSTOMER SERVICE INFORMATION

The YG500R mechanical diaper/small platform scale is shipped assembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact Customer Service.

To prevent scratching any components and avoid damaging the scale parts when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging. Carefully remove the scale from the carton and unwrap the packing materials. Set the carton aside for storage.

For further information or support, please contact Customer Service at:

Telephone: 1 (800) 815-6615

Email: homprocs@homscscales.com

INITIAL SET UP/OPERATING INSTRUCTIONS

INITIAL SET UP

1. Use on a flat, level, and stable surface.
2. On the front of the scale, adjust the tare dial located on the shatter resistant lens to position zero at the top and center of the dial.
3. Adjust the red needle using the calibration dial, located beneath the stainless steel platform to zero.

HOW TO WEIGH

1. Place item gently onto the stainless steel platform.
2. The weight of the item will be indicated by the red needle.

HOW TO USE THE “TARE” FUNCTION

This function allows the display to be zeroed out after weighing an item in a container. It shows the weight of additional items as if you weigh them separately.

1. Place an empty container gently onto the stainless steel platform.
2. Use the tare dial located on the front of the scale to rotate the measuring disk and re-align the red needle to zero weight.
3. Gently add an item to the container, the display will now only show the weight of the item.
4. Repeat steps 2 and 3 when adding more items to container.

MAINTENANCE

The following provide instructions for maintenance and cleaning for the YG500R.

MAINTENANCE

Before first use and after periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check overall appearance of the total scale for any obvious damage, wear and tear.
2. Do not store near chemicals, heaters or radiators. Use and store at room temperature.
3. Do not drop, kick, or jump on the scale.
4. This scale is a sensitive weighing device. Items such as clothing or towels placed or dropped on the scale may cause it to operate incorrectly. In order to prevent this, do not store or leave anything on the scale.
5. If your scale fails to work properly, do not attempt to dismantle it. This will void your warranty. See warranty information instructions on having your scale serviced.
6. All bearings and working parts are permanently lubricated. **DO NOT USE OIL OR LUBRICANTS.**

CLEANING

Proper care and cleaning is essential to ensure a long life of accurate and effective operation.

1. Clean all external surfaces with a clean damp cloth or tissue. Mild antimicrobial or antibacterial soap and water solution may be used. Dry with a clean soft cloth.
2. Do not immerse the scale into cleaning or other liquid solution. The scale is not waterproof.
3. Do not use abrasive cleaners.

WARRANTY

1-Year Limited Warranty

What does the Warranty Cover?

Health o meter® Professional scales are warranted from date of purchase against defects of materials or in workmanship for a period of one (1) year. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Return Department
9500 West 55th Street
McCook, IL 60525

If your scale is not covered by warranty, or has been damaged, an estimate of replacement costs will be provided to you for approval, prior to replacing.



PELSTAR, LLC

9500 West 55th Street • McCook, IL 60525 • USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homsscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license. Health o meter® Professional products are manufactured, designed, and owned by Pelstar, LLC. We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

© Pelstar, LLC 2015

**ISO13485:2003
CERTIFIED**

P/N UMYG500R Rev20150630