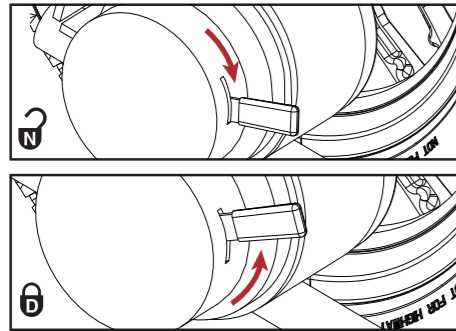
5

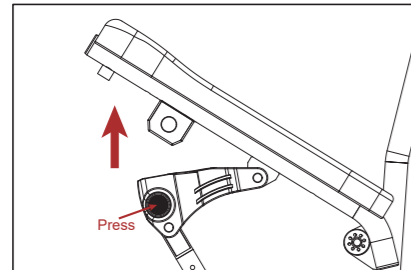
Push the freewheel lever down to disengage the drive motor (freewheel mode) and position the wheelchair close to a standard electrical outlet.

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6

ARMREST FLIP-UP

One hand press the armrest button, another hand flip up the armrest. And press down the armrest to the proper position to fix armrest.

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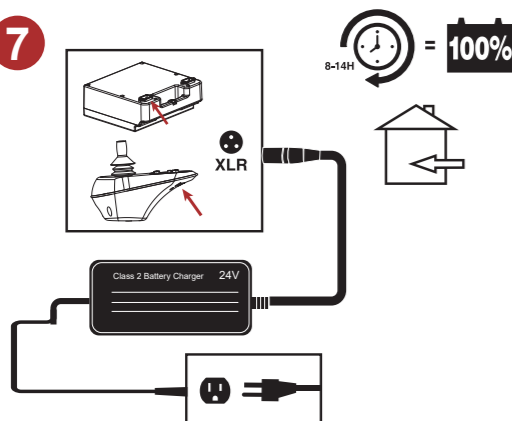
7

Fully charge the batteries prior to use. Battery and controller charger port options. Refer to the owner's manual for complete charging instructions.

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This Quick Start Guide offers basic information on initial wheelchair setup only. Read the owner's manual and all supplemental information thoroughly before operating the wheelchair for information on safety, battery charging, basic operation, comfort adjustments, disassembly, and care and maintenance.

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Metro Mobility USA LLC
Address: 159 Liberty Ave., Mineola, NY 11501
888-616-3876
support@metromobilityusa.com

STOP: READ FIRST

RECYCLING AND DISPOSAL

Cardboard may be recycled. Dispose of foam in accordance with state or local ordinances.

READ BEFORE UNPACKING

Apparent Loss or Damage

Upon receipt of this shipment, if any evidence of loss or damage is revealed through visual inspection, then it must be noted on the freight bill and signed by the carrier's agent. Failure to do this may result in the carrier's refusal to honor loss or damage claim. The carrier will furnish you with the necessary forms to file the claim.

Concealed Loss or Damage

When loss or damage is not apparent until the unit is unpacked, a claim for concealed loss or damage is required. When damage is discovered, make a written statement to the carrier for inspection. This request must be made within 48 hours of delivery. Keep all cartons and fillers for inspection. Carrier will furnish inspection report and necessary forms to file concealed loss or damage claim, since such damage or loss is the carrier's responsibility.

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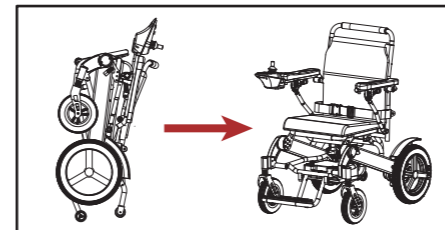
NOTE: Make sure wheelchair delivery is complete by using the component key above.

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1

UNFOLD WHEELCHAIR

Fully unfold the wheelchair, and then fasten the lock which under the backrest.

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
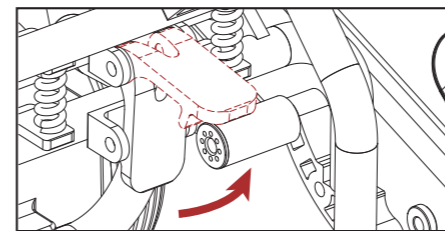
2

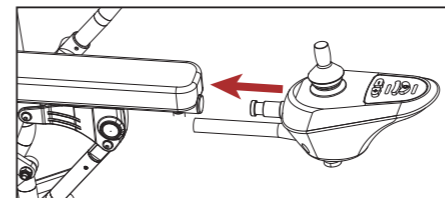INSTALL CONTROLLER

Insert the controller into armrest tube. And then lock the screw.

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
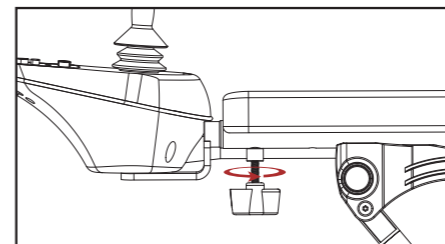
3

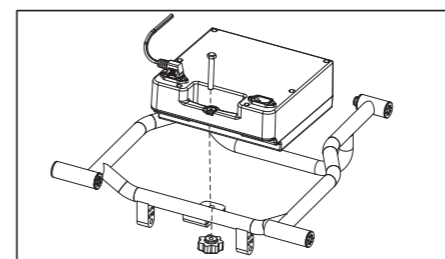INSTALL BATTERY

Put battery groove in the back tube loosen the screw . Then connect the battery plug with controller plug .

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
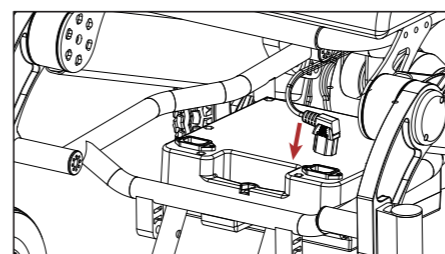
4

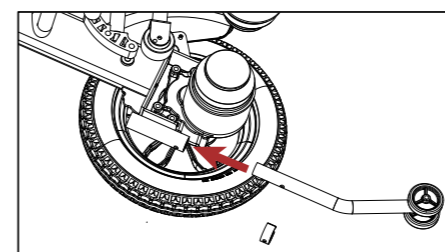INSTALL AND DISMANTLE ANTI-TIPPER

Press the button to install / dismantle the anti-tipper.

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