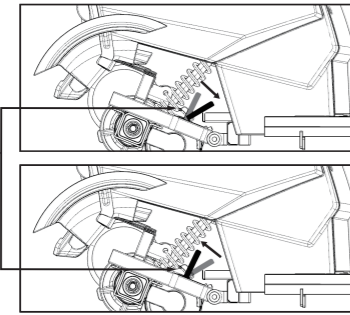
5

Push the freewheel lever forward to disengage the drive motor (freewheel mode) and position the scooter close to a standard electrical outlet.

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STOP: READ FIRST

RECYCLING AND DISPOSAL

Cardboard may be recycled. Dispose of foam in accordance with state or local ordinances.

READ BEFORE UNPACKING

Apparent Loss or Damage

Upon receipt of this shipment, if any evidence of loss or damage is revealed through visual inspection, then it must be noted on the freight bill and signed by the carrier's agent. Failure to do this may result in the carrier's refusal to honor loss or damage claim. The carrier will furnish you with the necessary forms to file the claim.

Concealed Loss or Damage

When loss or damage is not apparent until the unit is unpacked, a claim for concealed loss or damage is required. When damage is discovered, make a written statement to the carrier for inspection. This request must be made within 48 hours of delivery. Keep all cartons and fillers for inspection. Carrier will furnish inspection report and necessary forms to file concealed loss or damage claim, since such damage or loss is the carrier's responsibility.

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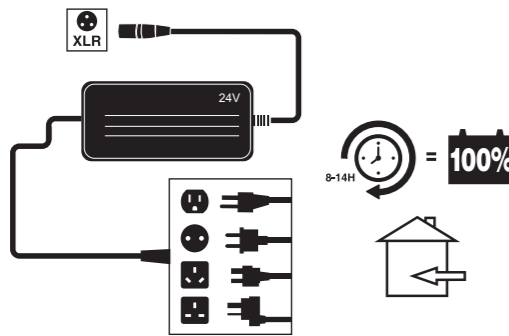
6

Fully charge the batteries prior to use. Refer to the owner's manual for complete charging instructions.

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This Quick Start Guide offers basic information on initial wheelchair setup only. Read the owner's manual and all supplemental information thoroughly before operating the wheelchair for information on safety, battery charging, basic operation, comfort adjustments, disassembly, and care and maintenance.

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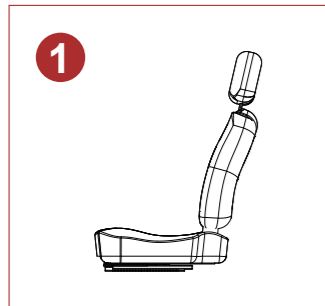
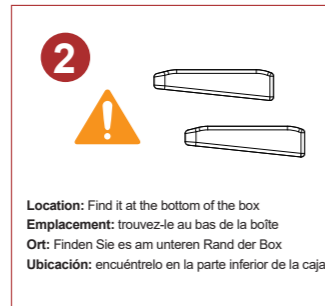
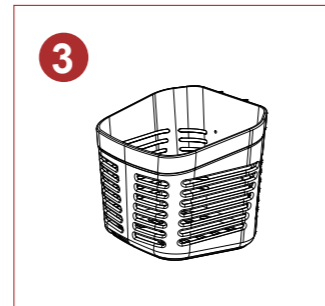
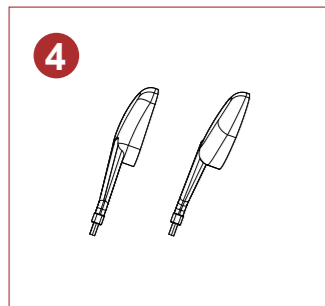
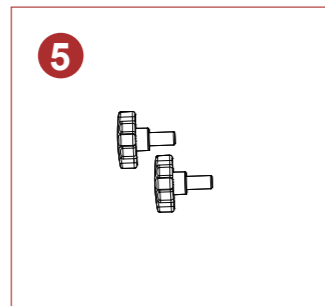
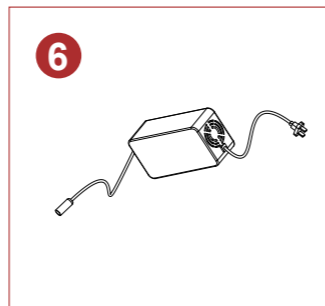
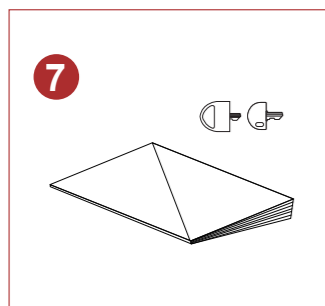
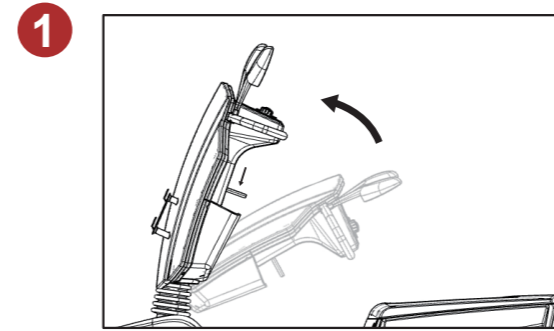
NOTE: Make sure wheelchair delivery is complete by using the component key above.

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
Adjustment of tiller positioning
Press down the lever and move it to your preferred position.

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
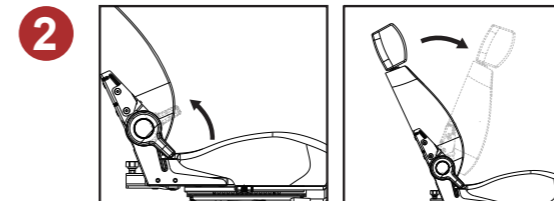
Adjust the seat back lever for seating back angle.
The flip-up armrest height can be adjusted by turning the adjustment dial.

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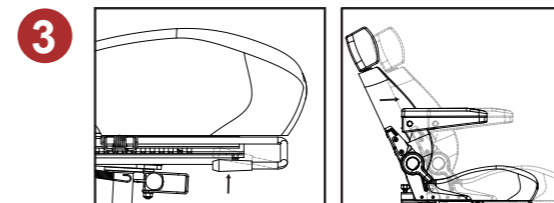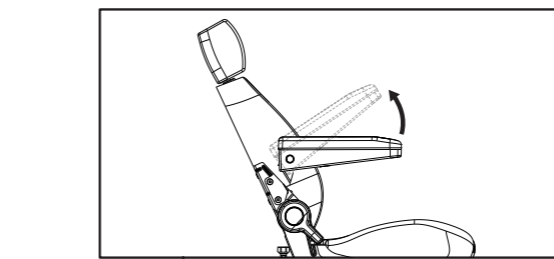Turn the swivel lever forwards to rotate the seat
Push the front lever downwards to move the seat forward or backward.

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