
Mimitakara[®] JAPAN **ITC**_{Type} Digital Hearing Device

Accessories Listing



① Hearing Device ② Portable Case x 1 ③ Cleaning Brush x 1 ④ Ear Cap (S,M,L x 1) ⑤ Trial Battery x 2

Operating Diagram

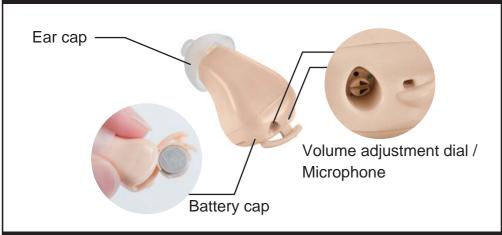

Removing the Hearing Device from the Case



Open the lid of the case. Pull upwards on the hearing device until it is in the vertical position.



Remove the hearing device from the case. Close the battery cap.

To Insert the Hearing Device Battery



Open the battery cap



Install a 312(PR41) battery into the hearing device as shown. Positive terminal facing outwards, close the battery cap.

Adjusting the Volume

Adjusting the volume by using another side of cleaning brush.



Volume adjustment dial Black(Off) Green(Low) Red(High



The volume will be higher when you turn the switch clockwise. And the volume will be lower when you turn the switch counterclockwise.

Place the Hearing Device



Insert hearing device into the case.



Close the case, the battery will stay automatically isolation .






- you will need to replace it.
- removing.



opening is exposed.


3. Place the ear cap onto the brim and roll it back into position as illustrated. Then return the turnover brim to the original state.

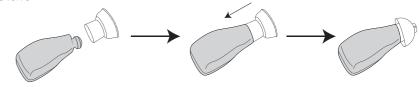

Usage

1. Turn the power on, and hold the body of the hearing device.

2. Use your finger to grasp the microphone and block the hole.

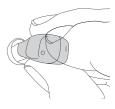


3. Place the hearing device in your ear while you are grasping it.

put the hearing device into your ear

Replacing the Ear Cap

* If the ear cap is damaged or wax cannot be removed from it,

1. Remove the old ear cap by pulling it back over the rim and

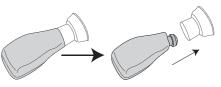


2. To replace the new ear cap turn it inside out so that the large



X Keep the hearing device away from corroding gas or dust and do not expose to direct sunlight, high temperatures or humidity.

Before and After Each Use

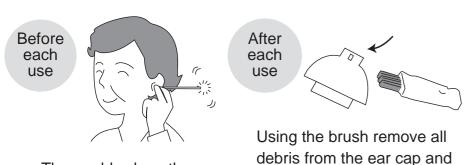


Thoroughly clean the ear.

ensure the hole on the side of the nozzle is not blocked.

Maintenance

- 1. Before using the hearing device, clean your ear using cotton swabs.*If the nozzle of the ear cap is clogged with earwax, sound quality will be impaired.
- 2. On a regular basis clean the hearing device using the cleaning brush, paying particular attention to the nozzle of the ear cap where wax can collect.

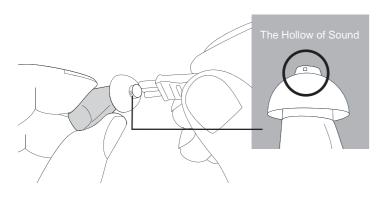

Warning

- Follow the instructions carefully to avoid personal injury or damage to the appliance.
- Keep the hearing device out of reach of children.
- To avoid injury remove the hearing device before sleeping.
- Remove the hearing device before taking part in vigorous exercise to avoid damaging the hearing device.
- Remove the hearing device before using a mobile phone as the radio waves will cause disturbance to the hearing device.
- The hearing device must be kept dry. Do not moisten the hearing device, or expose to hair spray. Remove before baths and showers.

Note: Please dispose of the hearing device and batteries safely.

Before Contacting Customer Services(Q&A)

Before contacting customer services please examine the hearing device and refer to the checklist below. If the hearing device still will not work, please contact customer services.

1. Cannot hear any sound

Check that the power is on by twisting the volume adjustment dial on the side of the hearing device .

- 2. There can be a number of reasons that the hearing device does not perform Correctly. Please refer to " solutions ". Check that the ear cap is not clogged with wax and remove any debris with the cleaning brush.
- 3. Please note it is also important to see a doctor periodically to check the problem is not due to hearing deterioration. Check if the battery is flat, to replace with new ones.

Guarantee

Thank you for purchasing this product, which has been made to demanding high quality standards and is guaranteed for domestic use against manufacturing faults for a period of 12 months from the date of purchase.

This guarantee does not affect your statutory rights. If your product fails due to a defect in material or workmanship during this period, please return it to the place of purchase. Normal wear and tear is not covered under the guarantee.

This guarantee is invalid if the product has been misused, subject to neglect, or has been attempted to be repaired by anyone other than our own service center.

Due to continuous product improvement, we reserve the right to change the product specification without prior notice.

Please retain for future reference.

Colors and contents may vary.

Waste electrical products should not be disposed of with household waste.

Please recycle where facilities exist.

Check with your Local Authority or retailer for recycling advice.

Name / Item Code: Digital Hearing Device / UP-6SY Purpose & Function: Enhances natural ability Standard Frequency: 2500Hz Maximum Output Sound Level Gain: 34 ± 5dB Highest Sound Pressure Level for Input SPL 90dB: 109 ± 5dB below 125dB Equivalent Input Noise Level: Below 29dB Total Harmonic Distortion Value of Reality: 2500Hz: below 3%; 800Hz: below 4%; 500Hz: below 4% Battery Current: Below 0.8mA Battery Type: Button size 312(PR41) battery Continuous Use: 260 hours Operating Temperature / Humidity: +10°C ~ +40°C, 30% ~ 85% RH Storage Temperature / Humidity: -10°C to +60°C, 30% ~ 85% RH Size: <Hearing Device> 13.2(L) x 12.5(W) x 23.5(H)mm <Case> 102.8(L) x 59(W) x 23.5(H)mm Weight: <Hearing Device> 2.2g <Case> 37.6g Materials: <Hearing Device> ABS <Case> ABS <Ear Cap> silicone

Suitable: Not recommend for serious hearing loss

Specification & Function

