Mimitakara®


Rechargeable BTE Hearing Amplifier

ACCESSORIES

- ① Charging Case
- 2 AC Adapter
- 3 AA Alkaline Battery
- 4 Hearing Amplifier
- ⑤ Ear Cap (two types : L,M,S and Mini)
- **6** Cleaning Brush



CONTENTS

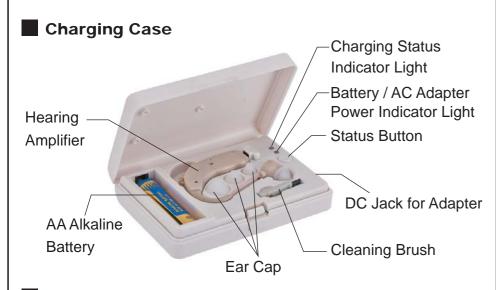

Hearing Amplifier



ON / OFF/NF SETTING METHOD

- OFF: The power off
- ON: The 10-second delay function prevents feedback noise while in use.
- NF (Noise-Free) : Low frequency background noise excluded.

CHARGING

Charging by Battery

- 1. Install a battery in the compartment as shown.
- 2. Please check that two lights are GREEN after you push the Status Button.
- 3. Please charge for at least 7 hours.




■ Charging by AC Adapter

- 1. Insert the AC Adapter.
- 2. Please check that two lights are GREEN after you push the Status Button.
- 3. Please charge it for 5 hours at least.



INDICATOR LIGHTS

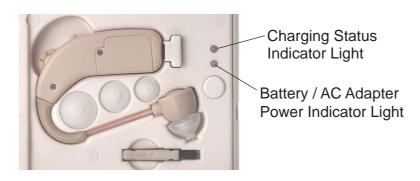

| Charging Status Indicator Light | GREEN | Hearing amplifier is charging normally |
|--|-------|--|
| | OFF | Hearing amplifier is fully charged (or Hearing amplifier is not correctly inserted in the charging case) |
| Battery / AC Adapter Power Indicator Light | GREEN | Power supply is normal |
| | RED | AA Battery power is low (insert new AA battery or use AC adapter) |
| | OFF | No power (insert new AA battery or use AC adapter) |


MAINTENANCE METHOD

Please clean your ear before use.
 If the inner ear is blocked with ear wax, then sound quality will be reduced.





2. Please clean out any earwax in the hearing amplifier Earwax can most often congregate around the sound tip as show below.

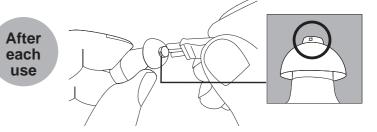

OPERATING INSTRUCTION

Notice: The 10-second delay function prevents feedback noise while in use.



Wear hearing amplifier on ear properly.



Insert ear cap into ear canal tightly to avoid feedback noise.



Turn on the power, or switch to "NF". If there is feedback noise please change to larger ear cap.



Rotate the volume switch to the desired volume level.

TROUBLE SHOOTING

1. Is power on?

Turn on the power switch.

2. Is the Ear Cap blocked with wax or debris?

Clean the Ear Cap using the Cleaning Brush provided.

3. Is the battery dead?

Place the Hearing amplifier in the Charging Case, and ensure it is charging by battery or AC adapter.

4. We suggest periodic medical evaluation by a licensed physician in diseases of the ear after using hearing aids.

SPECIFICATION

General Title of Commodity: Rechargeable BTE Hearing Amplifier

Purpose & Function: For assisting people of hearing obstacle

Standard Frequency: 1600Hz

Max. Output Sound Level Gain: 50 ± 5dB

Highest Sound Pressure Level for Input SPL 90dB: 145dB ± 5dB

Equivalent Input Noise Level: Below 29dB

Built-in Rechargeable Battery Current: < 3mA

Rechargeable Time: Up to 500 times in total

Continuous Use: 12-16 hours when fully charged

Operating Temperature / Humidity: +10°C ~ +40°C, 30% ~ 85% RH

Storage Temperature / Humidity : -10°C ~ +60°C, 30% ~ 85% RH

Size: <Hearing Amplifier > 55(L) x 40(W) x 10(H) mm

<Charging Case> 100(L) x 65(W) x 22(H) mm

Weight: <Hearing Amplifier > 7.6g / <Charging Case> 58.4g

Material: <Ear Cap> silicone

<Hearing Amplifier > / <Charging Case> / <AC Adapter> ABS



Very Important Notice

~ Recharge Any Time~

Under normal conditions, the battery can be recharged and discharged approximately 500 times.

In order to extend the rechargeable battery life, we strongly recommend users to charge the hearing aid whenever it is not in use.

- 1. Please Note: There is no need to recharge your hearing amplifier until it is almost out of power. To charge your hearing amplifier whenever you are not using it will be helpful to maintain the rechargeable battery & extend the usage life for recharging & discharging up to 500 times.
- 2. According to the character of rechargeable battery, when it is almost run out of power, the sound from hearing amplifier will reduce. Please don't treat it as any defective, just put your hearing amplifier back to the charging case or use the AC adapter to recharge your hearing amplifier.

↑ NOTICE

- To extend the rechargeable battery life, fully charge the hearing amplifier and then switch off when not in use for a long period of time.
- Keep both your ears and the hearing amplifier, free from earwax.
- With proper care, the rechargeable battery can last for up to, and beyond, 500 charge cycles.
- When the rechargeable battery is low, the sound becomes quieter.
 This means the battery is low, and needs to be charged.
- Keep this product away from infants, children and pets.
- Prior to first use, read instructions carefully to ensure proper operation, care and handling of your hearing amplifier.
- Do NOT use this product under the following conditions: operating mobile phone, sleeping, showering, washing your hair / face, swimming, or any vigorous exercise, to avoid damage to the hearing amplifier or you.

SITUATION USE

1. Conversations:

Keeping the average volume is fine.




2. Watch TV:

Let's enjoy your favorite program! You won't interfere with family at all.



3. Bird-watch:

Clearly hear "Chirrup" in the distance.

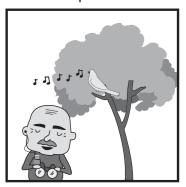

4. Listening to a speech:

Even sitting at the back, you will have no trouble hearing a speech.

