




ChairPro™ / MultiPro™ UNDERSEAT ALARM


U.S. Patent No. 6,963,286

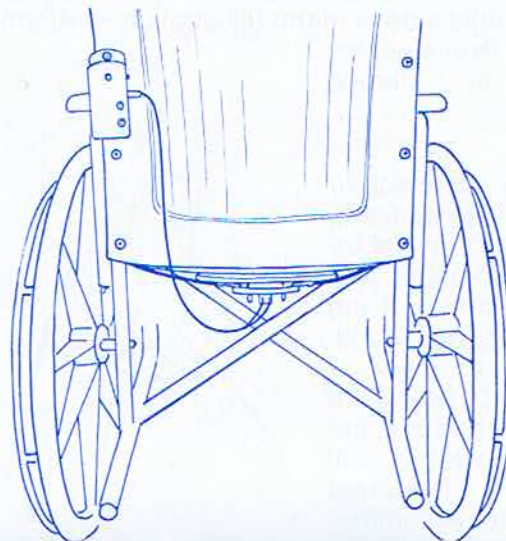


Model #	Fits Wheelchairs	Unit
909391	18" - 22"	Each

INSTALLATION-UNDERSEAT SENSOR UNIT


1. The under-seat sensor unit includes a quick-detach feature that permits the sensor to be removed easily when the wheelchair has to be power washed or when wheelchair is folded. (Illustration A)

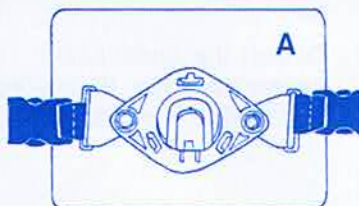

2. Remove the second and third screws on both sides of the wheelchair seat. (Illustration B)

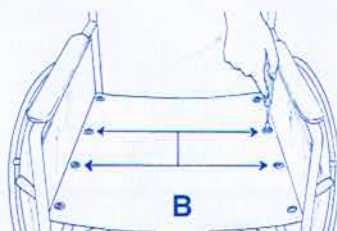


3. Disconnect the hanger straps from the sensor unit. Place the web straps with grommets between the wheelchair seat upholstery and the seat frame. Align the grommets with the screw holes. Reinsert and fully tighten the screws. The hanger straps should now be secured as shown. (Illustration C)

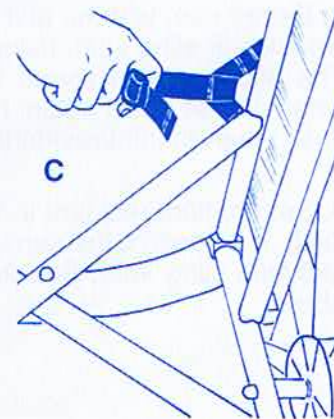


UNDERSEAT SENSOR UNIT


4. Connect both hanger straps to the sensor unit. (Illustration D) Make certain that the prongs on the sensor's plug are facing toward the back of the seat.

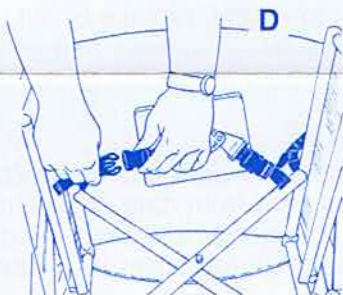


5. Pull the loose ends of both hanger straps to center the sensor and bring it into contact with the underside of the wheelchair seat. (Illustration E)

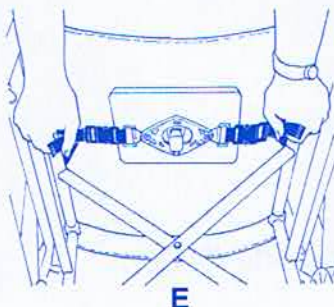

6. Attach the two spade-connectors to the under-seat sensor (Illustration F)

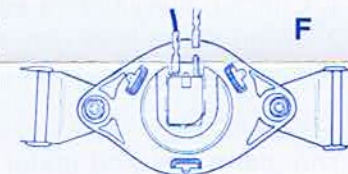

7. When installation is completed, the under-seat sensor, quick-detach feature, and power cord should appear as shown. (Illustration G)

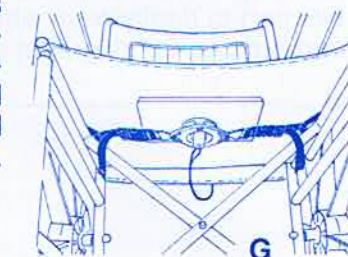


8. If you plan to use a wheelchair cushion, see #4 under "Operation" below.

Continued on reverse side.

OPERATION

1. Insert mini-hone jack into Mini-Phone Plug receptacle on right side of alarm (Illustration H) Alarm will sound. Press Auto-Reset to silence alarm.

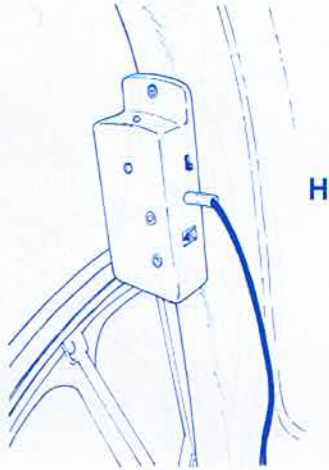

2. If a wheelchair cushion is to be used, place it on the seat for a few moments and then remove it. If the alarm does not sound, proceed to steps 3 through 5 below. If alarm does sound, the hanger straps will have to be loosened slightly to accommodate the weight of the cushion. Loosen straps, retest with cushion, then proceed through # 5 below.

3. Before use, test the unit yourself by sitting on the wheelchair sling seat, rising, and then sitting again. The alarm should sound when you rise and stop when you sit down again. Note: LED should always flash when patient/resident is seated.

4. Seat patient/resident in wheelchair. The LED will flash. If it doesn't, the sensor is probably not in contact with sling seat. See item #2 under installation above.


5. Select High or Low alarm volume using the switch on the right side of alarm unit.

6. If alarm is activated and the patient/resident has not returned to the chair, the alarm may be silenced by pressing the Auto-Reset button. The alarm is now in stand-by mode and ready for use. Alarm will automatically reset itself when patient/resident is again seated in chair. There is no On/Off switch.

Important:

- Alarm unit and under-seat sensor must be removed when chair is power washed.
- Under-seat sensor must be removed when wheelchair is folded. Failure to do so may cause damage to sensor.

Detach the under-seat sensor as shown. (Illustration I). Reattach and test the system as described above.

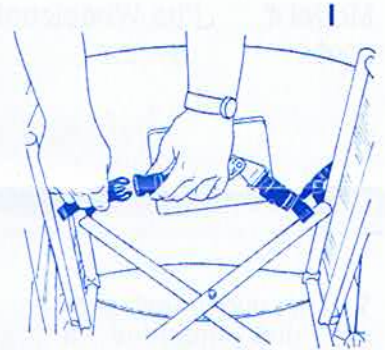

SAFETY REMINDERS

1. Check battery before each use. If alarm is weak or fails to sound, replace 9-volt battery.
2. When alarm is in use, check LED regularly. If it is not flashing, replace battery. If LED still fails to flash, check under-seat sensor to make certain that it is in contact with underside of seat.

GUARANTEE

The alarm unit and under-seat sensor are replacement guaranteed against defects in materials and workmanship under conditions of normal use for a period of one year from date of purchase. Please return defective part to dealer from whom the system was purchased. The dealer will send the defective part to Skil-Care and arrange for shipment of a replacement. This guarantee does not cover the battery supplied with the unit.

Skil-Care[™]
CORPORATION

Advanced design for nursing and therapy.[™]

Alarm unit made in China. Under-seat sensor made in USA.

Visit us online
at Skil-Care.com