Oring Installation

The Orin Bed should be placed at least 12inch away from the wall, opposite to the direction of exit.


Before you begin... A minimum of two people are required to assemble the ORIN Bed.







Place the two cardboard boxes flat in the designated position, check if the boxes are intact and free from damage.

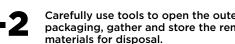






-3

Place all unpacked accessories in a centralized location, check the quantity and completeness of all product accessories, and prepare for installation.



- (1) Orin Bed Frame x1
- (2) Headboard x1
- (3) Footboard x1
- (4) Side Panels x2
- (5) Orin User Manual x1
- 6 Side Guard Ears x2
- (7) Mattress-Body Section x1
- (8) Mattress-Leg Section x1
- (9) Safety Belt x1
- (10) Tube Protection Rope x1
- (1) Remote Control x2
- (12) Shock Absorption Sponge x1
- (13) Feet x4
- (14) Screw x4

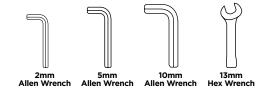

Please prepare the following installation tools: 2mm/5mm/10mm Allen Wrenches, 13mm Hex Wrench

Oring Installation

2-1


Flip the bed over and use a 10mm Allen Wrench to screw the foot 13 into the pre-drilled mounting holes on each of the 4 legs







Place the bed frame flat in the designated position, ensure the bed frame is stable, and cut the three black cable ties.

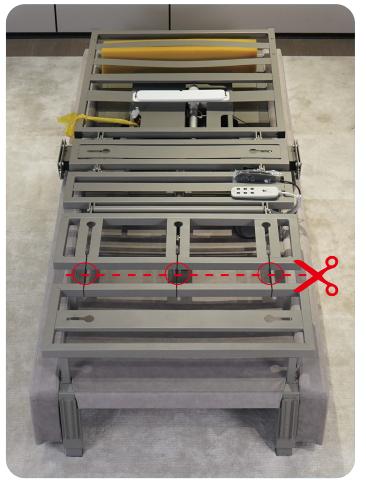






Begin installing the mattress, use the 2mm Allen wrench to install the left and right side retainer.






Remove the outer non-woven fabric protection, with the mattress bottom facing up, use the 13mm Allen wrench to remove all nuts and store them properly.

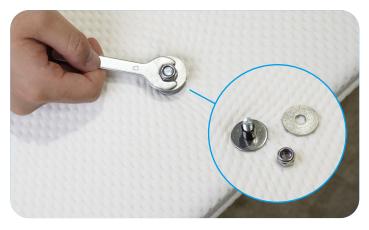

Oringed Installation


3-4

Align the screws with the holes and insert them downward into the designated holes. There are a total of 7 holes for the two mattresses combined.

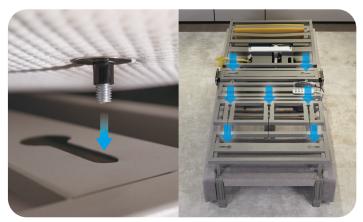


4-1

Place the mattress on the bed frame, connect the air tube and power cord, and check for tightness.

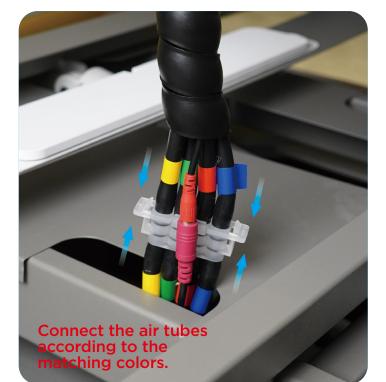



Align the headboard and footboard with the standard slots, insert them fully, and ensure they are installed steadily without any tilting.

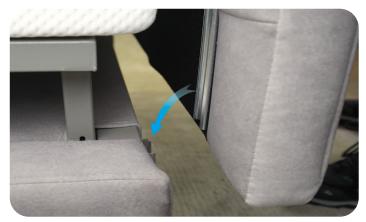




Use your hands or an external hexagonal wrench, or professional power tools, to tighten the washers and nuts upwards (not too tightly).






After completing the wiring in step 3-4, cut the yellow protective rope as packaging material.

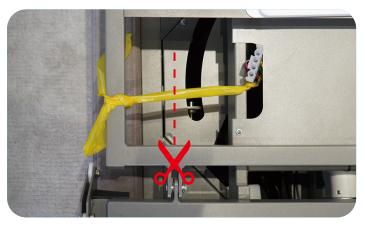




Align the left and right bed sideboards with the standard slots, gently insert them, and ensure they are laid flat for use. To disassemble, press and hold the semicircular metal lock button, then gently remove the sideboard in the direction indicated.

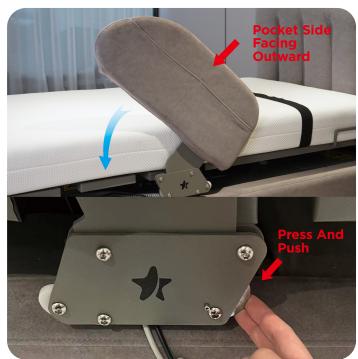

Oring Installation

Please reset the system before your first time use. This technique can also be used in the future in case of remote memory loss.

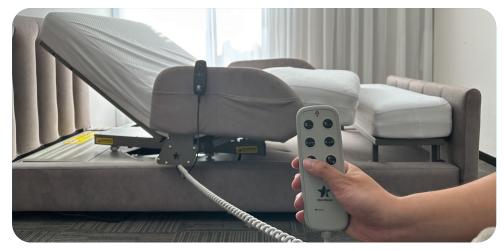


5-1 Please use the one button flat key (Bottom Left) to adjust the bed to the prerotating position.

5-2

Then hold the two button 3-5second, the reset process start (the yellow light on). Hold it till the bed turn back to the FLAT position and light will turn off automatically as the process finish.



2 Bed Models for Selection

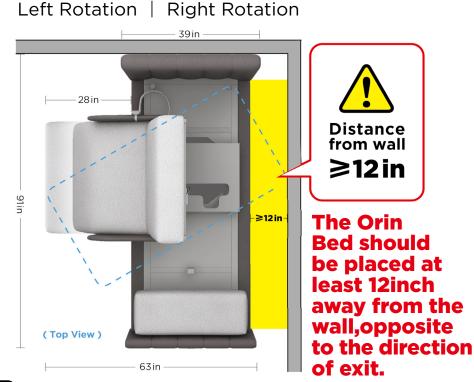

5-3 After raising the backrest using the hand controller, you can remove the shock-absorbing sponge manually.



Please Kindly Register and Activate Your Warranty: Visit our website, please go to www. starsleepus.com

Create an Account

or Log In: If you do not already have an account, you can easily create one. If you have an existing account, simply log in.

Upload Your Invoice:

To complete the registration process, kindly upload a copy or screenshoot of your purchase invoice.