

## limited lifetime warranty

Your ProHeal branded Overbed Table is warranted to be free of defects in materials and workmanship for the lifetime of the product from the time of the original purchaser.

The applicable warranty period will begin from the date of shipment to the original purchaser unless there is an end date on the component in which case the warranty will expire earlier than the warranty period or the end date.

This device was manufactured to the highest standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is a statement of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of reliable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as casters, rubber accessories, and grips, which are subject to normal wear and require periodic replacement.

If you have a question about your ProHeal device or this Limited Lifetime Warranty, please contact [Support@ProHealProducts.com](mailto:Support@ProHealProducts.com).

Scan to sign up for our Limited Lifetime Warranty



Text WARRANTY to (844) 564-2518

# PROHEAL

ProHeal Durable Medical Equipment

## OVERBED TABLE



### Items:

- PH-16210H (Cherry color)
- PH-16211H (Gray color)
- PH-16212H (Mahogany color)
- PH-16213H (Oak color)
- PH-16215H (Carbon Gray color)

**772-776-4325**

**[www.prohealproducts.com](http://www.prohealproducts.com)**

IF YOU HAVE ANY QUESTIONS OR CONCERNS CALL US FIRST!



DO NOT SEND BACK TO RETAILER

WE'RE HERE TO SUPPORT YOU.

**772-776-4325 SUPPORT@PROHEALPRODUCTS.COM**

### Safety Summary

To ensure the safe and proper installation of overbed table, these instructions MUST be followed:

#### ⚠ WARNING

- DO NOT use this product or any available optional equipment without first completely understanding these instructions and any additional instructional material provided with this product. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise injury or damage may occur.
- DO NOT use substitute parts if components are damaged or missing. Contact [Support@ProHealProducts.com](mailto:Support@ProHealProducts.com)
- Adjustable overbed tables have a powerful internal spring. Exercise extreme caution when working near the spring.
- DO regularly check frame screws to ensure they are tight.
- DO regularly check to ensure castors roll freely and locks work properly.
- The overbed table has a weight limitation of 40 lbs.
- After any adjustments, repair or service and prior to use, be sure all attaching component parts are secure.

#### ⚠ ACCESSORIES WARNING

ProHeal products are specifically designed and manufactured for use in conjunction with ProHeal accessories. Overbed Table accessories designed by other manufacturers have not been tested by ProHeal and are not recommended for use with ProHeal products.

1

# PROHEAL

## Overbed Table Non-Tilt


Model No. PH-16210H/16211H/16212H/16213H/16214H

### Overbed Table Assembly

**WARNING: DO NOT** remove the shipping screw until overbed table assembly is complete.


#### Spare parts:

- Pan head screws
- Bolt
- Wrench
- Bolt plate
- Base
- Caster with lock
- Caster without lock
- Column assembly
- Table top

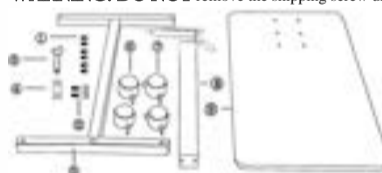

Step 1: Open the box and remove all of the parts. Verify all parts are present.

Step 2: Insert and tighten the six pan head screws ( ).

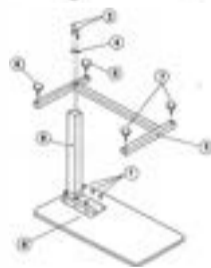

Step 3: Place the bolt plate ( ) on base ( ). Align the holes in the base ( ) with the bottom of the column. Install the 2 bolts ( ) and tighten with wrench ( ). The base must be installed in the same direction of the table top ( ).



Step 4: Please use ( ) wrench to tighten threaded caster bolts into the base ( ). The non-locking casters ( ) should be installed in the base furthest from the column while the locking caster ( ) should be closest to the column.

2