

KALAMIA PH 3F WIRELESS HANDCONTROL PROGRAMMING

KEEP THESE INSTRUCTIONS FOR FUTURE REFERENCE

FOLLOW THESE INSTRUCTIONS IF HANDCONTROL IS NOT WORKING AND BATTERIES ARE GOOD (BATTERIES SHOULD BE GOOD IF BUTTONS LIGHT UP WHEN ONE IS PRESSED)

1. UNPLUG THE POWER CORD
2. HOLD BOTH TOP BUTTONS OF THE HANDCONTROL DOWN
3. PLUG IN POWER CORD WHILE STILL HOLDING DOWN BOTH TOP BUTTONS
4. REMAING HOLDING BOTH TOP BUTTONS UNTIL BED MAKES A NOISE OR MOVE A LITTLE APROX. 30 SECONDS
5. RELEASE BOTH BUTTONS
6. TEST BUTTONS ON REMOTE, IF THEY DO NOT WORK BE SURE BATTERIES ARE GOOD AND REPEAT STEPS 1-5

HANDCONTROL SHOULD BE SYNCHED TO THE BED. IF NOT, REPEAT ABOVE STEPS.

If this does not fix the problem either the hand control or the control box is bad, usually it is the hand control that needs to be replaced.