

maintenance tips

In order to keep your Transport Chair in good working condition, proper care and maintenance is important. Be sure that your chair is in proper working condition at all times. A qualified dealer or repair center (wheelchair rental dealer) should be contacted for periodic inspections and repairs.

1. Once a week, wipe off the chrome frame using a clean soft cloth. If the chair has been exposed to moisture, dry the chair immediately. The metal parts of the chair should be polished once a month with an auto wax.
2. Clean the upholstery with a mild soap solution, rinse and wipe dry. Worn or torn upholstery should be replaced immediately as it may not support your body weight.
3. Inspect the wheels at least once a month. Correct any imbalances by loosening the lock nut on the axle, then tighten the axle just enough to remove imbalance.
4. Solid rubber tires require limited upkeep but need to be cleaned periodically with a damp cloth. The solid rubber tires should to be replaced if they become cracked or severely worn.

Limited 3 Year Warranty

Your ProHeal Products branded product is warrantied to be free of defects in materials and workmanship for three years of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. The 3-Year Limited Warranty is an expression of our confidence in the materials and workmanship of our product and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

ASTRA TRANSPORT CHAIR

The ProHeal logo is oriented vertically. It features a stylized, flowing purple line that curves around the letter 'P' and extends upwards, resembling a swoosh or a stylized 'H'. The word 'PROHEAL' is written in a bold, serif font, with the letters 'P', 'R', 'O', 'H', 'E', 'A', and 'L' stacked vertically. The 'P' is the largest and most prominent letter, with the 'R' and 'O' following below it. The 'H' is the next largest, followed by 'E', 'A', and 'L'. The entire logo is rendered in a dark purple color.

transport chair



WEIGHT CAPACITY
17"/19" 250 LBS

It is important for the caregiver to be familiar with proper assisting procedures in any situation that requires the use of the Transport Chair. Safety is the most important consideration. Consult your physician, nurse or physical therapist to determine the most suitable methods for your individual needs and abilities.

Note: The transport chair does not conform to Federal Standards for automobile seating and should not be used as a seat in a motor vehicle.

operating tips

how to operate your transport chair:

Your Transport Chair Dealer (Wheelchair rental Dealer) can assist you in making adjustments and provide you with operational information.

1. To Open the chair, tilt to one side and push down on the seat rails.
2. Lift the back-support to an upright position. The lock to secure the back will engage automatically. To fold down and store, pull up on both locks at the same time, and carefully fold down the back support.
3. To attach the footrests, align the holes on the footrest arm with the pegs on the frame of the Transport Chair. (See Diagram)
4. To Adjust the length of the footrests, push the pin on the telescoping tube shaft. The lowest part of the footrest should not be less than 2.5 inches from the ground in order to permit proper clearance.

5. To remove the Swing-away footrest, release the lock. Swing outward and rearward. Simply lift up and off to remove the footrest.
 6. The safety belt is already attached for use and should not be removed from the Transport Chair.
- Note:** The Transport Chair should not be operated without the assistance of an attendant.

safety tips

please read carefully before using
your transport chair:

1. Before entering or leaving the chair, engage both wheel locks against the tires on the rear wheels. Make sure the front casters are in the forward position before transferring in or out of the chair.
2. Do not move forward on the seat while leaning forward out of the chair. If it is necessary to do so, be certain that the front casters are in the forward position. If you are picking an object up from the floor, go past the object, and then go backwards. The casters will then swing to the forward position.
3. Leaning out of the transport chair without proper assistance could cause tipping.
4. Do not step on the foot plates when transferring, as this could cause tipping.

Note: You and your attendant should consult a qualified professional for correct techniques on ramp negotiations.

