

# Steel Rollator With 6" Wheels



Model #	Product Name
R300-6-KD-BLK	Steel Rollator with 6" Wheels - BLACK
R300-6-KD-BLU	Steel Rollator with 6" Wheels - BLUE
R300-6-KD-RED	Steel Rollator with 6" Wheels - RED



Innovative Healthcare Solutions

MedaCure Inc.  
221 South 31st Street  
Kenilworth, NJ 07033 USA  
[www.medacureinc.com](http://www.medacureinc.com)





Innovative Healthcare Solutions

MedaCure Inc.  
221 South 31st Street  
Kenilworth, NJ 07033 USA  
[www.medacureinc.com](http://www.medacureinc.com)

## Assembly Instructions

**Do not attempt to adjust or operate Rollator without carefully reading all instructions in this manual.**

1. Remove all contents from packaging: backrest, back wheels (with brake handles), rollator frame with seat and carry bag, front wheels. Included are six tightening knobs and four c-clips for easy and secure assembly.
2. Unfold rollator frame and turn onto its side. Attach wheels without brakes to the front of the rollator by inserting the wheel into the corresponding tube (Figure A). Align hole on wheel tube with frame and secure in place using tightening knob A, threading the knob on the inside (Figure B). Insert c-clip above knob to secure wheel in place (Figure C). Repeat with second wheel.

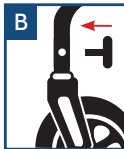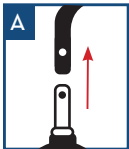

3. Rotate the rollator frame. Attach the back wheels (with brake handles) by inserting tube into the rollator frame, making sure the wheels are facing the outside of the rollator. Align hole on wheel tube with frame and secure in place using the tightening knob, threading the knob on the inside. Insert c-clip above knob to secure wheel in place. Affix brake cable to the "S" shaped clamp provided on the rollator frame. Repeat with second wheel.
4. Position rollator upright. Insert the brake handle tube into the frame and set to desired height. Secure in place using tightening knob B, threading the knob in from the outside. Repeat for second handle, making sure both are set to the same height.
5. Attach padded backrest by depressing the black thumb buttons and sliding into the black brackets. Note that the back rest can be folded up when not in use.

**Weight Capacity: 300 lbs. (125 kg)**



## Safety Instructions

1. Please make sure the Rollator is fully opened, seat is down and the folding bar under seat is locked in the down position.
2. Be sure that both knobs are fully tightened and are at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. Test the loop locks before operating the Rollator. When wheels are locked (see illustrations for locking and releasing locks below), lock bar on rear wheels should prevent wheels from rolling. If the wheels roll while brakes are locked, please take unit back to dealer for adjustment. When locks are released, the wheels should roll freely.
4. Always lock loop locks before sitting and do not use seat when unit is on an incline or uneven ground.
5. Replace tires when significantly worn or damaged.

### Important Safety Notice

Do not have anyone push you while you are seated on the Rollator. This is a walking aid only and is not to be used as a transportation device.

## Safety Instructions

Always engage and disengage wheel locks on both sides simultaneously.


1. To Slow Down or Come to a Temporary Stop—Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
2. To Temporarily Lock Wheels in Position—Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks— Pull up on both loop locks with palms of hand to original position.



## Instructions To Adjust The Hand Brake System

### Important Safety Notice

The hand brake system fitted on this rallator complies with the manufacturer's quality standards. The hand brake system has been preset (adjusted) to the best performance settings. To better suit you personally, please follow the instructions set out below. You may need to adjust the hand brake systems from time to time as they will wear out with normal use.

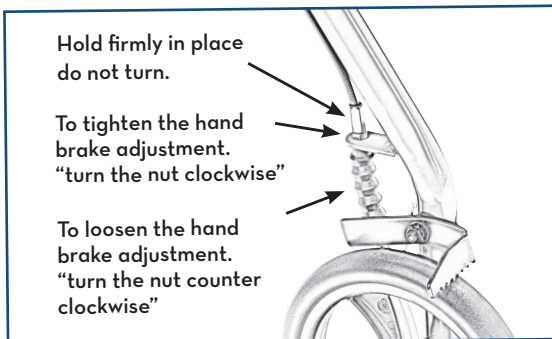

## Safety Instructions

Medacure recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit [www.medacureinc.com](http://www.medacureinc.com) for more info.

## Lifetime Limited Warranty

Your Medacure brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Medacure device or this warranty, please contact an authorized Medacure dealer.

**⚠ WARNING:** This product can expose you to chemicals, including diisononyl phthalate (DINP), which is known to the State of California to cause cancer. For more information go to [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov).



**Innovative Healthcare Solutions**

MedaCure Inc.  
221 South 31st Street  
Kenilworth, NJ 07033 USA  
[www.medacureinc.com](http://www.medacureinc.com)



**Innovative Healthcare Solutions**

MedaCure Inc.  
221 South 31st Street  
Kenilworth, NJ 07033 USA  
[www.medacureinc.com](http://www.medacureinc.com)