



Stroller Tray

1. To assemble the tray, slide the two metal plates into the slots on the bottom (*Fig 7A*).
2. Undo the hook and loop fasteners on both side panels (for access) (*Fig 7B*).
3. Using the included wrench and medium sized hex key, partially loosen the bolt (*Fig 7C*).
4. Slide the slot of the mounting bracket around the bolt between the washer and pipe (*Fig 7D*).
5. Snap the other end of the tray mounting bracket onto the stroller frame (*Fig 7E*).
6. Repeat steps 3-5 on the other side.
7. Retighten the bolts.
8. Slide the metal rods into the mounting brackets.
9. Tighten the tray screw (*Fig 7F*).
10. To lock the tray in place, tighten the hand knobs on the mounting brackets.
11. Refasten the hook and loop fasteners on both side panels (*Fig 7*).



Figure 7