

SPR Storage Rack

Item #7297

Installation Instructions



Hardware Included:

- 2 Bases
- 6 Hooks
- 12 Screws

Assembly:

- 1) Choose the location on the wall and locate the wall studs. The rack is designed for mounting to 16" centered wood or metal studs. If you can not locate the wall studs, please use other mounting hardware to secure the base.
- 2) Use the base as a guide and mark the wall through the holes. Drill a small starter hole for each of the screws, then attach the mount to the wall.
- 3) Insert the hooks downward into the top holes and make sure that they go through the bottom holes.



WARRANTY

This product is guaranteed to be free from defects in material and/or workmanship under normal use and service for a period of one year from date of purchase. The above guarantee is null and void if unauthorized repair and/or modification is performed on the unit.



Sammons Preston
A Patterson Medical Company
1000 Remington Blvd., Suite
210
Bolingbrook, IL 60440
Phone: 800-323-5547
Direct Phone: 630-378-6000
Fax: 800-547-4333
Direct Fax: 630-378-6310



Authorized European
Representative
Homecraft Rolyan
A Patterson Medical Company
Nunn Brook Road
Huthwaite
Sutton in Ashfield
Nottinghamshire
NG17 2HU, UK

UK Customer Services
Tel: 08444 124 330
Fax: 08448 730 100

International Customer
Services
Tel: +44 1623 448 706
Fax: +44 1623 448 784