


Swing-Away Foot Support

Reorder #	Description	Wheelchair Size	Unit
☐703470	Swing-Away Foot Support, Left	Universal	Each
☐703471	Swing-Away Foot Support, Right	Universal	Each
☐703472	Swing-Away Foot Support, Left & Right	Universal	Pair








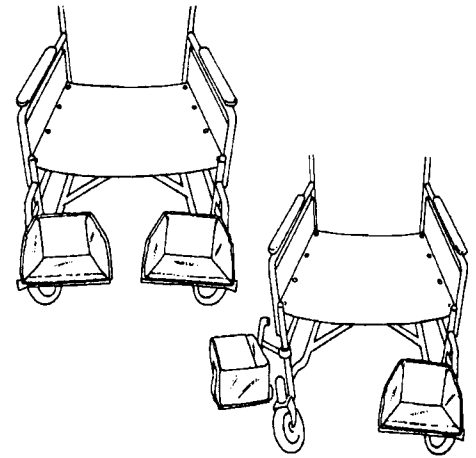


Purpose

Prevents one or both feet from slipping off the sides or back of wheelchair footrest. Swings to the side for safe and easy transfer. Single units may be used by patients who require positioning and support for one foot while using the other to self propel. Fits all standard wheelchair and hemi-chair footrests.

Instructions

1. Loosen wing nut and remove knobbed screw from clamp. (Illustration A)
2. Before placing Swing-Away Foot Support on footrest, detach Velcro[®] strip and position it to the side. Place clamp around footrest support bar. (Illustration B)
3. Insert knobbed screw through both holes in clamp and secure wing nut. (Illustration C)
4. Bring Velcro[®] strip under wheelchair footrest plate. (Illustration D)
5. Insert Velcro[®] strip through D-ring. (Illustration E)
6. Bring Velcro[®] strip back under footrest and secure Velcro[®] hoop. (Illustration F) Note: It is important that you take up all slack so that foot support will be securely positioned on footrest plate.
7. There are narrow Velcro[®] strips on both the left and right sides of the Swing-Away Foot Support. These provide additional stability for the foot support. Secure these around the footrest support bar as shown. (Illustration G)

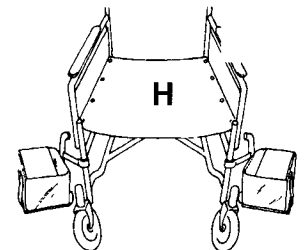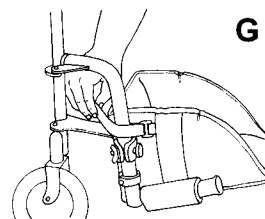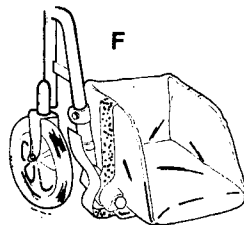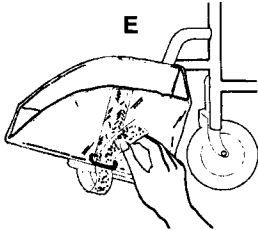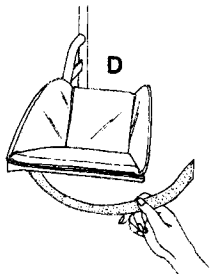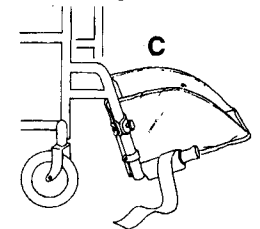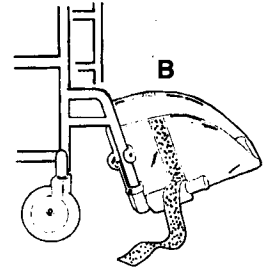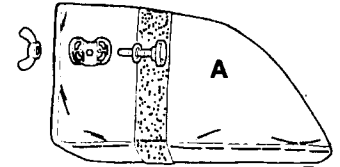

Important Safety Notice

Be certain to swing each footrest to the side before transferring patient to or from the wheelchair. (Illustration H)

Product Care

Swing-Away Foot Supports may be wiped clean using a damp cloth. A spray-type cleanser may be used. Removed lint and other accumulated particles from the Velcro[®] material by means of a small, stiff brush.