
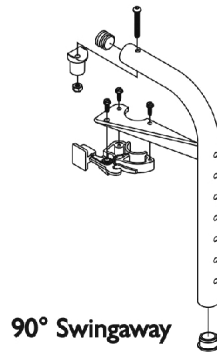
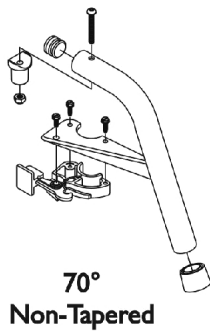
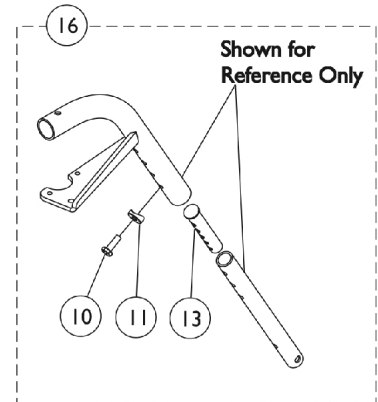
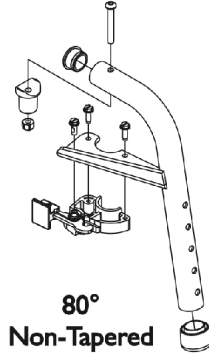
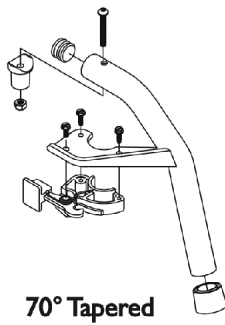
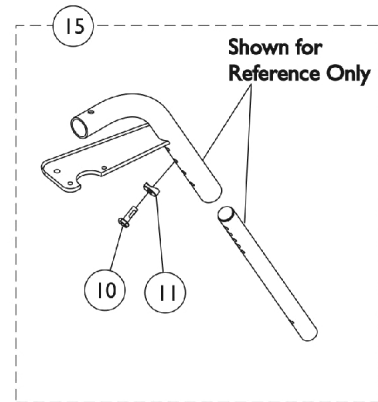
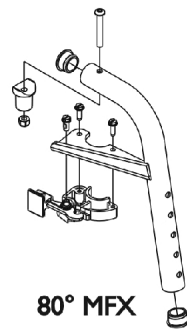
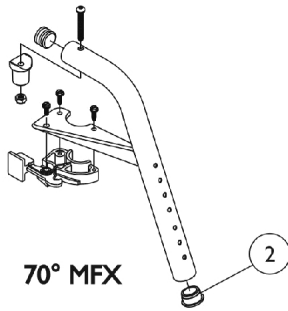
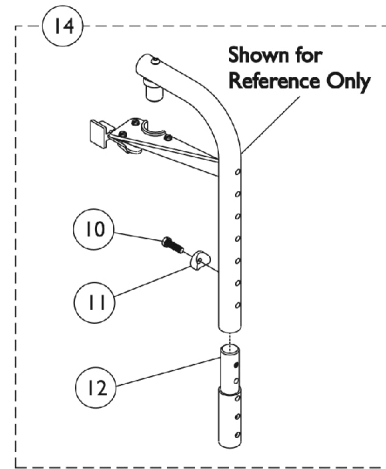
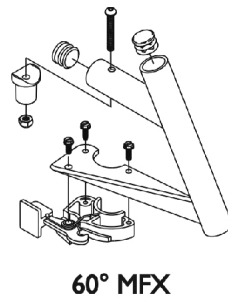


Swingaway Footrest Support Assemblies

60, 70, 80 and 90 Degrees

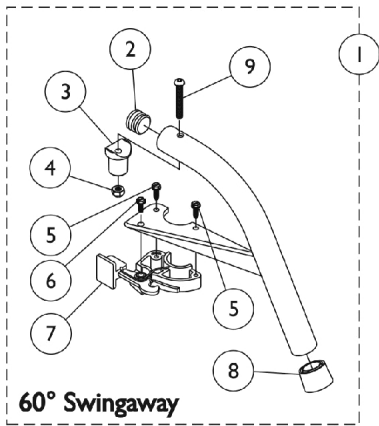

Swingaway Footrest Support Assemblies

60, 70, 80 and 90 Degrees

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,B	See Chart	#1 Swingaway Footrest Support Assemblies (60, 70, 80, and 90 Degrees)	1	1
2	G	1056328	Plug Button, Black (1")	1	1
3		1017148	Pivot, Footrest	1	1
4		1025736	Locknut (1/4-20)	1	1
5		1025451	Screw, Slotted Hex Washer Head (#10-32 x 5/8")	1	2
6		1017423	Screw, Slotted Hex Washer Head w/ Patch (#10-32 x 1/2")	1	1
7		1025454	Housing, Latch - Right	1	1
7		1025455	Housing, Latch - Left	1	1
8		1026124	Collar, Anti-Rattle	1	1
9		1001762	Screw, Socket Button Head (1/4-20 x 1-3/4")	1	1
10		1045267	Screw, Hex Head w/ Patch (1/4-20 x 3/4")	1	1
11		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	1
12	A	1038232-NLA	NLA - Extension, Footrest, Color	1	1
13	H	1020534	Insert, Threaded	1	1
14	A,C	8881038231	Kit, 3" Extension w/ Hardware (Pair)	1	1
15	D	1028004	Kit, Pivot Slide Tube Mounting Hardware (Pair)	1	1
16	E,F	1026703	Kit, Pivot/Slide Tube Mounting Hardware (Pair)	1	1

NOTE: A - Specify frame finish when ordering.
 B - Includes Footrest Support and items 2-9
 C - Includes items 10-12. Hardware quantity accommodates the right and left side
 D - Includes items 10 and 11. For use on 60 degree and 70 degree front riggings, see exception below.
 E - Includes items 10, 11 and 13. Hardware quantity accommodates the right and left side
 F - For use on 60 degree front riggings without longer pivot/slide tube and 1651 footplate option only.
 G - Use a quantity of two when replacing parts for an MFX style footrest.
 H - As of 8/15/03, the Threaded Insert was revised from 3 to 5 Holes. Threaded Insert is Backward Compatible and does not require ordering in Pairs.

#1 Swingaway Footrest Support Assemblies (60, 70, 80, and 90 Degrees)

Includes Footrest Support and items 2-9

Footrest Support Style	Model	Right	Left
60 Degree Swingaway	SWING1319	8881026992	8881026993
60 Degree MFX	SWING914	8881026998	8881026999
70 Degree MFX	70MFX	8881047481	8881047480
70 Degree Non-Tapered	70NTAPER	8881026994	8881026995
70 Degree Tapered	70STAPER	8881026996	8881026997
80 Degree Non-Tapered	80NTAPER	8881170566	8881170567
80 Degree MFX	80MFX	8881170568	8881170569
90 Degree Swingaway	90SWING	8881062596	8881062597

Specify frame finish when ordering