



Re-Order #	Description	Size	Weight	Unit
703350	Synthetic Sheepskin Slippers - Wheelchair	Small/Medium	0.25 lbs.	Pair
703351	Synthetic Sheepskin Slippers - Wheelchair	Medium/Large	0.25 lbs.	Pair
703352	Synthetic Sheepskin Slippers - Walker	Small/Medium	0.25 lbs.	Pair
703353	Synthetic Sheepskin Slippers - Walker	Medium/Large	0.25 lbs.	Pair

Purpose

The Synthetic Sheepskin Relief Slippers protect the feet for residents who cannot tolerate the abrasion or pressure from standard shoes or slippers. The synthetic sheepskin is soft and minimizes friction between the foot and slipper. The loose pile synthetic sheepskin also minimizes perspiration.

Models 703350 & 703351 have an elastic band that slips over the wheelchair footplates and helps prevent the feet from sliding off the footrests. Model 703352 & 703353 incorporate a non-skid bottom and is designed for walking but also can be used in the wheelchair.

Important Safety Precaution

Since the wheelchair variety (model 703350 & 703351) does not have a non-skid bottom, they should not be use while ambulating. Prior to walking with models 703352 & 703353 check to see if the non-skid surface has maintained the non-skid property. If the bottom is slick, discard and purchase a replacement.

Easy Care

When using the wheelchair variety, make sure that the straps fit snugly. If not then the slippers aren't appropriate for that model of wheelchair. Both variety of slippers are machine washable an temperatures not to exceed 180°F (82°C).