

Re-Order #	Description	Size	U/M
703280	Footrest Extender	16"-18" x 1"	Set
703281	Footrest Extender	16"-18" x 2"	Set
703282	Footrest Extender	16"-18" x 3"	Set
703283	Footrest Extender	16"-18" x 4"	Set
703291	1" Replacement	16"-18" x 1"	Ea.
703294	Footrest Extender - Bariatric	20"-24" x 1"	Set
703296	Footrest Extender - Bariatric	20"-24" x 2"	Set
703297	Footrest Extender - Bariatric	20"-24" x 3"	Set
703271	1" Replacement	18"-20" x 1"	Set
703272	2" Replacement	18"-20" x 2"	Set
703273	3" Replacement	18"-20" x 3"	Set
703274	4" Replacement	18"-20" x 4"	Set
703256	1" Replacement	20"-22" x 1"	Set
703257	2" Replacement	20"-22" x 2"	Set
703258	3" Replacement	20"-22" x 3"	Set
703259	4" Replacement	20"-22" x 4"	Set



Re-Order #	Description	Size	U/M
703210	Footrest Extender	18"-20" x 1"	Set
703220	Footrest Extender	18"-20" x 2"	Set
703230	Footrest Extender	18"-20" x 3"	Set
703240	Footrest Extender	18"-20" x 4"	Set
703250	Footrest Extender - Bariatric	20"-22" x 1"	Ea.
703251	Footrest Extender - Bariatric	20"-22" x 2"	Ea.
703252	Footrest Extender - Bariatric	20"-22" x 3"	Set
703253	Footrest Extender - Bariatric	20"-22" x 4"	Set

Note: All models above fit 16"-18", 18"-20" and 20"-22" Wheelchairs and Geri-Chairs

Purpose

Skil-Care's Two-Piece Footrest Extender is designed to solve two important problems: It provides support to control foot drop and it prevents resident's feet from slipping off the footrests. The footrest extender is designed to fit 16-inch, 18-inch and 20-inch wide wheelchairs and geri-chairs depending on the model chosen.

Application

1. Before seating resident, lock wheel brakes and rotate footrests to the out-of-the-way position. Assist patient into chair and position him/her as far back on the seat as possible. Reposition footrests.
2. Place footrest extender on footrests. Place cushion section on footrest with the padded backboard against footrest support bars (*Fig. 1*).
3. Place straps behind footrest support bars, secure buckles and tighten straps (*Fig. 2*).
4. The footrest extender may be positioned to accommodate various degrees of ankle plantar flexion. This may be accomplished by securing the top straps to the upper section of the footrest support bars as shown in the illustration (*Fig. 3*).
5. Important Safety Note. Remove the footrest extender and rotate the footrests to the out-of-the-way position when assisting your resident from the chair. Do not permit your resident to stand when the footrest extender is in place. Replacement footrest cushions are available in one-, two-, three- and four-inch heights.

Easy Care

Skil-Care's Two-Piece Footrest Extender is made from durable vinyl for longlasting use. To clean, just spray and wipe with a damp cloth. Accumulated lint and other particles may be removed from the hook-and-loop material with a small stiff brush.

Fig. 1

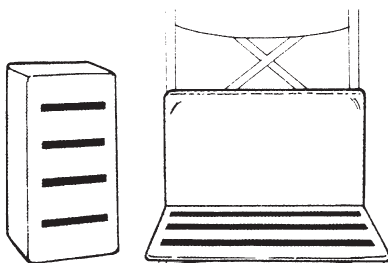


Fig. 2

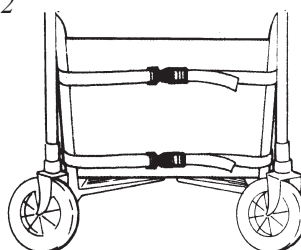


Fig. 3

