

assembly instructions

1. While facing front of walker, grasp sides of tray and clamp brackets on walker front frame.
2. Check all clamps to assure secure attachment.
3. With thumb and tips of fingers, pull the tray out and up until locked for use position.
4. To stow tray, lift up and push forward and tray will drop down.

instrucciones de armado

1. Con la andadera puesta enfrente de usted, agarre los costados de la bandeja y las ménsulas de abrazadera en el armazón delantero de la andadera.
2. Inspeccione todas las abrazaderas para garantizar una sujeción segura.
3. Con el pulgar y la punta de los dedos, jale la bandeja hacia fuera y hacia arriba hasta que quede en posición trabada para el uso.
4. Para plegar la bandeja, levántela y empujela hacia delante, y la bandeja descenderá.

PROHEAL
WWW.PROHEALPRODUCTS.COM
250 95TH STREET | SURFSIDE, FL | 33154
772-776-4325

IF YOU HAVE ANY
QUESTIONS OR
CONCERNS PLEASE
CALL US FIRST!



DO NOT SEND BACK TO RETAILER

WE'RE HERE TO SUPPORT YOU.

772-776-4325 SUPPORT@PROHEALPRODUCTS.COM



说明书：166*166MM/157克铜板/双面彩印