



ReOrder #	Description	Unit
909201	Fits all hospital-style beds, wheelchairs, geri-chairs, lounge and other chairs.	Set

Purpose

Skil-Care's Universal Alarm Mounting Bracket Belt enables caregivers to move multipurpose alarms from the bed to the wheelchair. It also permits bed alarms to be mounted to the back of the headboard to discourage patient tampering.

Easy to Use

Application (chair)

1. Place belt around geri-chair, wheelchair, or other chair backrest. Secure buckle and tighten belt.
2. Remove backing from the enclosed Velcro[®] strip and attach strip (hook material) to back of alarm unit.
3. Secure alarm unit to loop material on bracket (*Fig. 1*).

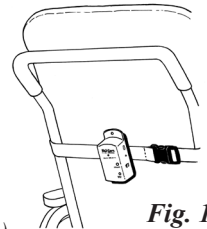


Fig. 1

Application (Bed)

1. Place belt vertically around headboard and adjust bracket so it is aligned vertically with belt. Secure buckle and tighten belt (*Fig. 2*).
2. Remove back from the enclosed Velcro[®] strip and attach strip (hook material) to back of alarm unit.
3. Secure alarm unit to loop material on bracket.

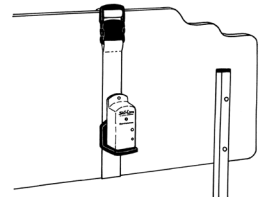


Fig. 2

This product is guaranteed for 6 months.

Flip-dessus pour la traduction française



N° Réf	Description	Unité
909201	Adaptable à tous les types de lits, de fauteuils roulants, fauteuils gériatriques, de relaxation ou autres	À l'unité


Objectif

La ceinture de fixation pour alarme universelle Skil-Care permet au personnel soignant de déplacer les alarmes multi-usages entre le lit et le fauteuil. Elle permet également de placer les alarmes derrière la tête du lit afin d'empêcher que le patient ne les touche.

Facile à Utiliser


Pose (sur fauteuil)

1. Placer la ceinture autour du dossier du fauteuil gériatrique, du fauteuil roulant ou autre type de fauteuil. Attacher la boucle et resserrer la ceinture.
2. Retirer la pellicule protectrice de la bande Velcro® fournie et fixer la bande (celle qui « accroche ») au dos de l'alarme.
3. Fixer soigneusement l'alarme au dispositif de fixation attaché à la ceinture (Fig. 1).

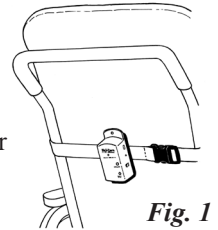

Pose (sur lit)

1. Placer la ceinture en position verticale autour de la tête de lit et régler le dispositif de fixation afin qu'il soit aligné (également en position verticale) avec la ceinture. Attacher la boucle et resserrer la ceinture (Fig. 2).
2. Retirer la pellicule protectrice de la bande Velcro® fournie et fixer la bande (celle qui « accroche ») au dos de l'alarme.
3. Fixer soigneusement l'alarme au dispositif de fixation attaché à la ceinture.

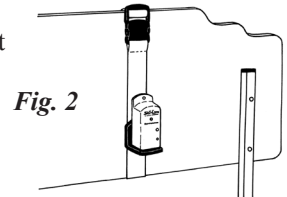

Ce produit est garanti pendant 6 mois.

Flip-dessus pour la traduction française