Health o meter®

P r o f e s s i o n a l

2210KL-AM / 2210KL4-AM 2210KG-AM / 2210KG4-AM

High Resolution Digital Neonatal / Pediatric Tray Scale

Equipped with
Everlock®

User Instructions

PELSTAR®, LLC 9500 West 55th St. McCook, IL 60525-7110 USA www.homscales.com

Health o meter® Professional 2210KL-AM, Health o meter® Professional 2210KL4-AM,
Health o meter® Professional 2210KG-AM, Health o meter® Professional 2210KG4-AM

© Pelstar®, LLC 2024

weigh**easier**®

Thank you for purchasing this Health o meter® Professional product. Please read this manual carefully, and keep it for easy reference or training.

TABLE OF CONTENTS

Cautions and Warnings.....	3
Specifications for This Scale.....	4
Certifications / Disposal	5
Assembly Instructions	6
Set Up Instructions.....	8
Everlock® Instructions	11
Operating Instructions.....	12
Calibration.....	17
Troubleshooting	18
Maintenance and Cleaning	19
Warranty	20

Note: This scale has been factory calibrated, and does not require calibration prior to use.

Please register your scale for warranty coverage at:

www.homscales.com

For User Instructions updates and revisions please go to:

www.homscales.com

CAUTIONS AND WARNINGS

INTENDED USE

This Health o meter® Professional neonatal / pediatric scale is intended to be used in a professional medical environment by trained medical staff. This product was designed to weigh pediatric patients who are safely positioned on the scale tray. Do not modify the product or use it for anything other than its intended purpose.

To prevent patient / caregiver injury or damage to your scale, please follow the instructions in this user manual very carefully.

- Do not exceed the weight capacity specified for this scale.
- For accurate weighing, this scale must be placed on a flat, stable surface.
- For accurate weighing, verify before each use the proper operation according to the procedure described in this manual.
- Do not use in the presence of flammable or explosive materials.
- If the scale becomes damaged, it should not be operated until properly serviced.
- Do not transport the scale with a patient or object on the scale.
- Ensure that the scale does not come in contact with liquids, excessive temperature, or excessive humidity.
- Do not gas sterilize or autoclave the scale.
- Scales contain delicate sensors. Do not bang into doors, walls, etc., as damage may result.

Patient Safety

- This scale is designed for static weighing of patients only. No scale should be used for patient transfer.
- To prevent patient injury, the patient must be attended throughout the entire weighing event.
- During weighing process, place baby on the scale, holding one hand just above baby.

When operating the scale with batteries:

- If the display indicates low battery, replace the batteries as soon as possible.
- When replacing / inserting batteries, be sure to use all new batteries.

For Optimum Accuracy:

- If moving the scale throughout a facility, avoid excessive temperature changes.
- Avoid scale placement near heating/cooling vents, heating appliances, areas of high airflow or direct sunlight.

In no event whatsoever shall Pelstar, LLC be liable for damages or injuries arising from or connected with the assembly, use, or misuse of its products.

SPECIFICATIONS FOR THIS SCALE

General

Health o meter® Professional scales use the most sophisticated microprocessor technology available. Each precision instrument is designed to provide accurate, reliable and repeatable weight measurements, as well as other features that make the weighing process simple, fast and convenient.

In order to achieve satisfactory results, the operator must read this manual thoroughly before using this scale.

If any questions arise regarding this scale or its use, please contact Health o meter® Professional Scales Customer Service at 1-800-815-6615.

Scale Specifications

Capacity and Resolution	20,000 grams / 20 kg / 45 lb Resolution: 1 g (to 10,000 g); 5 g (10,001 to 20,000 g); 0.1 oz (0 – 45 lb)
Power Requirements	Adapter Model No. UE15WCP1-090050SPA, Part No. UE160714HKKK1-P or Adapter Model No. UES06WNCP-090050SPA, Part No. UE190716HKSH2RM Health o meter® Professional Model (ADPT30) (included*) Input: 100-240VAC, 50/60Hz, 500mA Output: 9.0V 0.5A <small>*AC adapter not available on 2210KL-AM-C models with firmware version 2.4 or 2.42 or scale versions not evaluated to model code SPE 3000</small> or 6 D Cell Alkaline batteries
Environmental	Operating temperatures: 68°F to 85°F (20°C to 30°C) Storage temperatures: 32°F to 122°F (0°C to 50°C) Maximum Humidity: 95% RH

DEFINITION OF SYMBOLS

Caution or attention required.

Proper positioning of battery polarity.

Degree of protection against electrical shock. Type BF.

Direct electrical current.

Attention, Consult Accompanying Documents

CERTIFICATIONS / DISPOSAL

Certification Descriptions

 	<p>Pelstar, LLC has been officially certified as the manufacturer of Health o meter® Professional medical devices. The Pelstar, LLC quality assurance system covers the development, production, sales and service of medical scales and measuring systems.</p>
--	---

Scale Disposal

This Health o meter® Professional scale must be disposed of properly as electronic waste. Follow the national, regional or local regulations which apply to you for disposal of electronic waste or batteries. Do not dispose of this device in the domestic waste stream.

ASSEMBLY INSTRUCTIONS

Before Assembly

Each neonatal/pediatric tray scale is shipped disassembled in one carton. Carefully inspect the carton for shipping damage before unpacking. If damage is found, contact your shipper or a Health o meter[®] Professional representative immediately at 1-800-815-6615. Claims must be filed with the shipper as soon as possible after receipt of the package. The following information details what you will find inside the main carton as you unpack the parts for assembly.

To prevent scratching any components, carefully remove each assembly from the carton and unwrap the packing materials. Set the carton aside for storage. To avoid damaging the scale parts when unpacking, do not use a box cutter, knife, scissors, or any sharp object to open the protective inner packaging.

Parts List

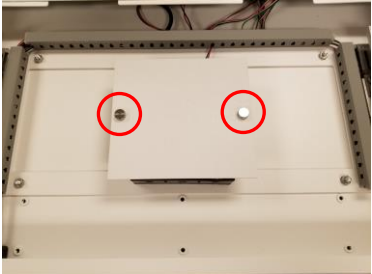
Carton

- (1) Scale platform
- (1) Acrylic tray
- (6) D-cell Alkaline batteries
- (1) User instructions
- (1) Power adapter*

*AC adapter not available on 2210KL-AM-C models with firmware version 2.4 or 2.42 or scale versions not evaluated to model code SPE 3000

ASSEMBLY INSTRUCTIONS (CONT)

The scale is shipped without the required six (6) D-cell batteries installed. The battery compartment is located underneath the scale cover. Use the following procedure to install the batteries:

<p>1. Remove the two thumb screws on the back of the scale and set aside. Carefully lift the cover off the scale.</p>	 
<p>2. Remove the two thumb screws on the top of the battery compartment located in the center of the inside of the scale.</p>	
<p>3. Insert the batteries following the +/- orientation for proper positioning. Note: slide the batteries towards the contact and away from the spring to ensure secure electrical connection. Replace the battery tray cover and thumb screws. Before replacing scale cover, press the power button to ensure the scale powers on. Reinstall the scale cover by replacing the two thumb screws on the back of the scale.</p>	
<p>4. Position the scale with the display buttons facing you. Align the tray so that the measuring tape is positioned over the back end of the scale platform. Carefully lower the tray onto the four mounting pins on the scale platform. When the tray is properly aligned with the pins, gently push down on the tray over each pin until its locks into position.</p>	

Moving the Scale

The scale should be lifted by the two black handles only.

NOTE: The scale tray should never be used for transporting a baby or objects. The tray surface must be clear of any objects during transport.

SET UP INSTRUCTIONS

To enter the Options mode, while the scale is turned off push and hold **ENTER/SELECT** while pressing and releasing the  button.

Auto Off Setting – Powering the Scale via Batteries

When powering via batteries the display can be set to power off after a period of inactivity. The default setting is 2 minutes (120 seconds). Follow this procedure to set the Auto Off time or disable the Auto Off. **Note:** Disabling the Auto Off and using the “Continuous” setting will affect battery usage and result in the need for more frequent battery replacement.

1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOB” appears on the display, then release. Press **ENTER/SELECT** to change the setting.
AOB=C Disables the Auto Off and scale is continuously on
AOB=1 Scale will power off after 60 seconds (1 minute) of inactivity
AOB=2 Scale will power off after 120 seconds (2 minutes) of inactivity
AOB=3 Scale will power off after 180 seconds (3 minutes) of inactivity
2. Press **CLEAR/REWEIGH** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Auto Off Setting – Powering the Scale via AC Adapter

*AC adapter not available on 2210KL-AM-C models with firmware version 2.4 or 2.42 or scale versions not evaluated to model code SPE 3000

When powering via the AC adapter the display can be set to power off after a period of inactivity or remain on. The default setting is “disabled”. Follow this procedure to choose the desired Auto Off setting when powering via the AC adapter.

1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOB” appears on the display, then release. Use the  or  buttons to advance the display screen until “AOA” appears. Press **ENTER/SELECT** to change the setting.
AOA=d Scale will not power off and will remain on powered via the AC adapter
AOA=E Scale will power off after a period of inactivity. The Auto Off time period of inactivity is based on the setting chosen in the AOb setting as noted above.
2. Press **CLEAR/REWEIGH** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Auto Hold

The Auto Hold function allows the scale to continue displaying the weight measurement after the patient is removed from the weighing tray. The Auto Hold setting can be adjusted to display the weight for 0, 30, or 60 seconds. The default setting is 30 seconds. Follow this procedure to adjust the Auto Hold setting.

1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOB” appears on the display, then release. Use the  or  buttons to advance the display screen until “AHd” appears. Press **ENTER/SELECT** to change the setting.
AHd = 0 Weight measurement will not be held on the display after the patient is removed
AHd = 1 Weight measurement will hold on the display for 30 seconds
AHd = 2 Weight measurement will hold on the display for 60 seconds
2. Choose the setting you prefer and press **CLEAR/REWEIGH** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

SET UP INSTRUCTIONS (CONT.)

Sound Options

The display can be set to stay silent or beep while operating the scale. The default setting for Sound is “enabled”. Follow this procedure to enable or disable sound.

1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOB” appears on the display, then release. Use the ▲ or ▼ buttons to advance the display screen until “Snd” appears. Press **ENTER/SELECT** to change the setting.
Snd = E Enables sound so the scale beeps when keys are pressed.
Snd = d Disables sound so the scale stays silent when keys are pressed.
2. Choose the setting you prefer and press **CLEAR/REWEIGH** to save your selection and exit the options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Locking Unit of Measure

(This feature is not available on KG models)

The weight measuring unit (pounds/LB, kilograms/KG, grams/GM) can be locked to display weight only in the selected unit of measure. The unit of measure lock default setting is “unlocked”. Follow this procedure to lock the **UNIT** button.

Note: This lock is not permanent and can be unlocked by the user. For a permanent lock see the Everlock section on page 11.

Note: If Everlock® is engaged, this feature will not appear in the Options mode.

1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOB” appears on the display, then release. Use the ▲ or ▼ buttons to advance the display screen until “Unt” appears. Press **ENTER/SELECT** to change the setting.
Unt = U Allows the user to toggle weight measurement between LB and KG.
Unt = L Locks the unit button so weight is only displayed in pounds (LB)
Unt = | Locks the unit button so weight is only displayed in kilograms (KG)
Unt = g Locks the unit button so weight is only displayed in grams (g)
2. Choose the setting you prefer and press **CLEAR/REWEIGH** to save your selection and exit the options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Enabling Unit Display with Unit of Measure Locked

Users can still view the alternate weight measuring unit by enabling the “Unit Display” option. Enabling the “Unit Display” option allows the user to press and hold the **UNIT** button to briefly display the weight in the alternate weight measuring unit. If locked in kilograms or grams, Unit Display will allow you to view the weight in pounds. If locked in pounds, Unit Display will allow you to view the weight in kilograms. If locked in grams, Unit Display will allow you to view the weight in pounds. After releasing the **UNIT** button the display will return to the locked weight measuring unit. The “Unit Display” default setting is “Disable”. Follow this procedure to enable “Unit Display” when the weight measuring unit is locked. **Note:** If Everlock® is enabled, this feature will not appear in the Options mode.

1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOB” appears on the display, then release. Use the ▲ or ▼ buttons to advance the display screen until “UdP” appears. Press **ENTER/SELECT** to change the setting.

SET UP INSTRUCTIONS (CONT.)

- UdP = d Disables the **UNIT** button so it will not display the other weight measuring unit.
UdP = E Allows the user to view the other weight measuring unit by holding down the **UNIT** button even if the weight measuring unit is locked. Enabling this function will only allow the user to view the other unit while pressing the **UNIT** button; it will not unlock the weight measuring unit.

2. Choose the setting you prefer and press **CLEAR/REWEIGH** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Enabling Wireless Option

To allow the scale to wirelessly connect to another device, the wireless option must be enabled. The wireless default setting is “disabled” Follow this procedure to enable or disable the wireless option.

Note: This function is only available if a wireless module has been factory installed.

1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOb” appears on the display, then release. Use the ▲ or ▼ buttons to advance the display screen until “rAd” appears. Press **ENTER/SELECT** to change the setting.

rAd = d Disables the wireless option.

rAd = E Enables the wireless option and allows connection to another wireless device.

Note: If there is not a wireless module installed the display will show “Error” if trying to enable the wireless option.

2. Press **CLEAR/REWEIGH** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Setting Backlight Brightness

This option allows the user to set the brightness of the backlight on the display. Users can choose from four levels of brightness or choose to disable the backlight. The brightness default setting is level 4 – the brightest level. Follow this procedure to set the brightness of the display or disable the backlight.

Note: If the scale is being powered by batteries, the backlight will dim to level 1 when the scale is in idle mode.

Note: High levels of brightness will utilize more battery power.

1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOb” appears on the display, then release. Use the ▲ or ▼ buttons to advance the display screen until “bLt” appears. Press **ENTER/SELECT** to change the setting.

bLt = 0 Disables the backlight

bLt = 1 Lowest level of brightness

bLt = 2 2nd lowest level of backlight brightness

bLt = 3 3rd highest level of brightness

bLt = 4 Highest level of backlight brightness

2. Choose the setting you prefer and press **CLEAR/REWEIGH** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

SET UP INSTRUCTIONS (CONT.)

Recall Option

(Setting not available on firmware versions 2.4 and 2.42)

The recall option allows the scale to view the last weight measured on the display. The default setting for Recall is “enabled”. Follow this procedure to enable or disable Recall.

1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOB” appears on the display, then release. Use the **▲** or **▼** buttons to advance the display screen until “rCL” appears. Press **ENTER/SELECT** to change the setting.
rCL = d Disables the Recall option.
rCL = E Enables the Recall option so the last weight measurement can be recalled.
2. Press **CLEAR/REWEIGH** to save your selection and exit the Options mode.
3. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

Everlock®

The Everlock® feature allows the measuring unit to be permanently locked on the scale, disabling the **UNIT** button. The Everlock® default setting is “disabled”. Follow this procedure to engage Everlock®.

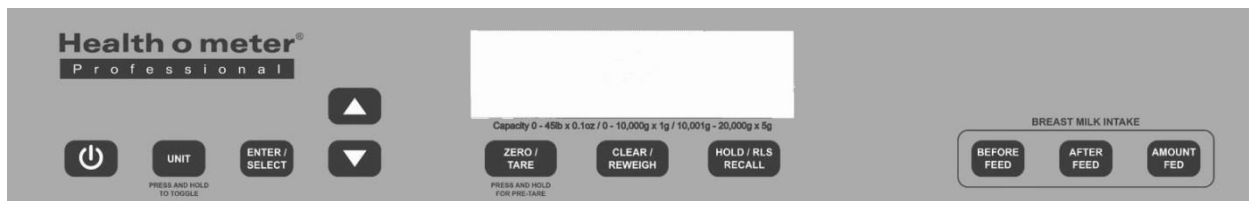
Warning: Everlock is permanent and cannot be reversed.

Notes:

- Press **CLEAR/REWEIGH** at any time to exit the Everlock® procedure.
 - Once Everlock® is engaged the Unt and UdP options will no longer be displayed when scrolling through the Options menu.
 - There is no method to undo Everlock® without sending the scale back to the manufacturer.
1. While the scale is turned off, press and hold **ENTER/SELECT** then press and release the  button. Hold **ENTER/SELECT** until “AOB” appears on the display. Use the **▲** or **▼** buttons to advance the display screen until “ELC” appears. Press **ENTER/SELECT** to change the setting.
ELC =d Default setting with Everlock® disabled.
ELC = E Engages Everlock®. Once this option is selected, the display will prompt you to begin the locking procedure. If this is desired, follow the instructions below.
 2. After Everlock® is selected, “PASS1” will appear on the display. This prompts the user to enter a password. Press the following keys in this sequence:
HOLD, BEFORE FEED, AFTER FEED, AMOUNT FED
 3. If entered correctly, “ELC=L” (if scale was in LB mode), “ELC=┌” (if scale was in KG mode) or “ELC=g (if scale was in GM mode) will appear on the display.
 4. Use the **▼** and **▲** keys to toggle between (L) Pounds, (┌) Kilograms or (g) Grams. When the desired unit is displayed, press **ENTER/SELECT** and “PASS2” will appear on the display.
 5. This will prompt the user to enter the second password. Press the following keys in this sequence: **AMOUNT FED, AFTER FEED, BEFORE FEED, HOLD**
 6. If entered correctly, the scale will display “ELC =E”.
 7. Press **CLEAR/REWEIGH** to save your selection and exit the Options mode.
 8. The scale will restart. The display will show “UEr”, followed by a version number, and then dashes. When the display shows “0.0” the scale is ready to use.

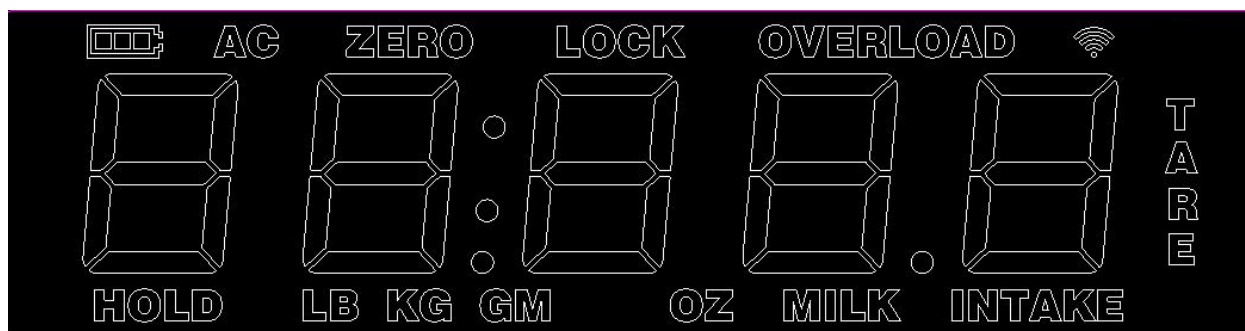
Warning: Do not power off the scale until CLEAR/REWEIGH is pressed to save the Everlock option, the scale has restarted and the display has returned to “0.0”

OPERATING INSTRUCTIONS

Function	Description
	Turns scale on and off. Also used when entering Calibration or Options mode.
HOLD/RLS/RECALL	Holds the displayed weight until the button is pressed again to release the value. Will recall the previously locked weight if pressed when nothing is on the scale.
UNIT	2210KL-AM/2210KL4-AM Only: Toggles between pounds (LB), kilograms (KG) and grams (GM). If Unit Display setting is enabled, press and hold UNIT button to briefly display the alternate weight measuring unit. Note: The Unit button is disabled on KG only models or on KL models with Everlock® engaged. The UNIT button is also disabled if the “Unit Display” setting is disabled.
	Used to navigate through Options mode or to set pre-tare value.
ZERO/TARE	Zeroes out or tares the scale prior to a weighing.
CLEAR/REWEIGH	Exits out of a mode or process. Also allows for repeated weighing of the patient without removing the patient from the scale.
ENTER/SELECT	Selects an option or value.
	Used to navigate through Options mode or to set pre-tare value.
BEFORE FEED	Used in store the patient’s weight before being fed as part of the process to measure breast milk intake.
AFTER FEED	Used in store the patient’s weight after being fed as part of the process to measure breast milk intake.
AMOUNT FED	When pressed, displays the amount fed measurement as part of the process to calculate breast milk intake.

OPERATING INSTRUCTIONS (CONTINUED)

Screen Icons

Icon	Description
	Indicates scale is being powered by batteries. Also indicates level of battery power remaining.
AC	Indicates scale is being powered by AC adapter.
ZERO	Displays along with dashes when scale is zeroing.
LOCK	Flashes during weighing process. Stays displayed while weight measurement is locked.
OVERLOAD	Displays when weight on the scale exceeds capacity.
LB	Indicates weight mode is in pounds.
KG	Indicates weight mode is in kilograms.
GM	Indicates weight mode is in grams.
HOLD	Displayed when a weight is held or recalled after pressing the HOLD/RLS/RECALL button.
TARE	Displayed when a weight is tared.
MILK INTAKE	Displayed along with the calculated breast milk intake.
	BT models only: Icon is flashing: Indicates that the scale's wireless module is searching for an auxiliary wireless device to connect to. Icon is solid: Indicates that the scale's wireless module is connected to an auxiliary wireless device.

OPERATING INSTRUCTIONS (CONTINUED)

Weighing a Patient

Note: When this scale is turned on, it will always default to the settings and units last used (LB, KG or GM).

1. With all weight off the scale, press the  button to turn on the scale. Wait until zeroes appear on the display.
2. Gently lay the patient onto the center of the weighing tray. "LOCK" will appear and flash on the display along with a weight measurement. When a stable weight is determined "LOCK" will stay displayed along with the patient's weight. Excessive patient movement may affect the time it takes for the scale to lock on a weight.

Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.

- To reweigh the patient without removing them from the tray, press **CLEAR/REWEIGH**.
 - After the weight locks, and before removing the patient from the weighing tray, the weight measurement can be held on the display by pressing **HOLD/RLS/RECALL**. To release the held weight, press **HOLD/RLS/RECALL** again.
3. After removing the patient from the tray, the Auto Hold feature allows the scale to continue displaying the weight measurement after the patient is removed from the tray. The default Auto Hold time is 30 seconds. To disable the Auto Hold or change to 60 seconds, see page 8.

Weight Recall

1. To recall the last weight measured, press **HOLD/RLS/RECALL**.
2. To clear the display of the recalled weight, press **HOLD/RLS/RECALL** again.

Zero Function

The **ZERO/TARE** button zeroes the scale prior to a weight measurement. **ZERO/TARE** is also used in the Tare function as outlined below.

Tare Function

When using this scale, the weight of an object, such as a blanket, can be subtracted from the total weight in order to determine the weight of the patient alone.

Weighing a Patient and Taring an Unknown Weight

The minimum weight that can be tared is 0.02 lb / 0.01 kg / 10 g. **Note:** The scale must lock onto a weight before it can be tared.

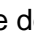
1. With all weight off the scale, press the  button to turn on the scale. Wait until zeroes appear on the display.
2. Place the object to be tared on the center of the weighing tray. The display will show a value for the object's weight.
3. Press **ZERO/TARE**. The word "TARE" will appear on the display then the scale will return to zero along with the TARE icon.
4. Keep the object on the platform, and then gently lay the patient on the center of the weighing tray. The scale will automatically deduct the weight of the object and only display the patient's weight.
 - To reweigh the patient without removing them from the tray, press **CLEAR/REWEIGH**.
 - After the weight locks, and before removing the patient from the weighing tray, the weight measurement can be held on the display by pressing **HOLD/RLS/RECALL**. To release the held weight, press **HOLD/RLS/RECALL** again.

OPERATING INSTRUCTIONS (CONTINUED)

5. After removing the patient and the tared item from the tray, the Auto Hold feature allows the scale to continue displaying the weight measurement after the patient is removed. The default Auto Hold time is 30 seconds. To disable the Auto Hold or change to 60 seconds, see page 8.
6. After the held weight is cleared off the display, “UndEr” will reappear along with the TARE icon. Press **ZERO/TARE** to clear the tared value.
Note: If the scale powers off, the tare value will be erased.

Weighing a Patient and Taring a Known Weight (Pre-Tare)

Pre-Tare can be performed for objects between 0.2 oz – 5 lb / 0.005 kg – 2.2 kg / 5 g – 2200 g

1. With all weight off the scale, press the  button to turn on the scale.
2. Wait until zeroes appear on the display. Press and hold **ZERO/TARE** until the word “TARE” appears on the display.
3. The display will show zero and the tare icon will flash. Enter the weight of the object to be tared. Use the  and  keys to increase or decrease the weight. Holding down the arrow keys will cause the rate of the change to progress faster.
4. Once the desired tare value is set, lock the value by pressing **ENTER**. The TARE icon will stop flashing and remain on, and “UndEr” will appear on the display along with the TARE icon.
5. Gently lay the patient and the object to be tared on the center of the weighing tray. The scale will automatically deduct the weight of the object and only display the patient’s weight.
 - To reweigh the patient without removing them from the tray, press **CLEAR/REWEIGH**.
 - After the weight locks, and before removing the patient from the weighing tray, the weight measurement can be held on the display by pressing **HOLD/RLS/RECALL**. To release the held weight, press **HOLD/RLS/RECALL** again.
6. After removing the patient and the tared item from the tray, the Auto Hold feature allows the scale to continue displaying the weight measurement after the patient is removed. The default Auto Hold time is 30 seconds. To disable the Auto Hold or change to 60 seconds, see page 8.
7. After the held weight is cleared off the display, “UndEr” will reappear along with the TARE icon. Press **ZERO/TARE** to clear the tared value.
Note: If the scale powers off, the tare value will be erased.

Breast Milk Intake Function

This scale offers the ability to calculate the patient’s breast milk intake after a feeding. Follow this procedure to determine the intake amount.

Note: The patient must first be weighed before the feeding.

1. With all weight off the scale, press the  button to turn on the scale. Wait until zeroes appear on the display.
2. Gently lay the patient onto the center of the weighing tray. “LOCK” will appear and flash on the display along with a weight measurement. When a stable weight is determined “LOCK” will stay displayed along with the patient’s weight. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight.

Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.

3. Once the weight is locked, press **BEFORE FEED** to store the patient’s weight before the feeding, “StORd” will appear on the display.

Note: When **BEFORE FEED** is pressed it automatically clears any stored weight in the **AFTER FEED** and **AMOUNT FED** memory.

OPERATING INSTRUCTIONS (CONTINUED)

4. Carefully remove the patient from the weighing tray and feed the patient.
Note: The scale can weigh other patients while the initial patient is being fed. However, only one patient's weight can be stored in the milk intake functions.
Note: The stored Before Feed weight remains in memory even if the scale powers off.*
**Scales with firmware version 2.4 will not store the weight if the scale is powered off. Disable the Auto Off setting to ensure stored weights are saved.*
5. After the patient is fed, perform the post-feed weighing. With all weight off the scale, press the  button to turn on the scale. Wait until zeroes appear on the display.
6. Gently lay the patient onto the center of the weighing tray. "LOCK" will appear and flash on the display along with a weight measurement. When a stable weight is determined "LOCK" will stay displayed along with the patient's weight. Depending on the movement of the patient on the scale, it may take several seconds for the scale to lock onto the weight.
Caution: To prevent patient injury, the patient must be attended throughout the entire weighing event.
7. Once the weight is locked, press **AFTER FEED** to store the patient's weight. "StORd" will appear on the display.
Note: "ERROR" will appear on the display if the After Feed weight is less than the weight stored in the Before Feed memory.
Note: The stored After Feed weight remains in memory even if the scale powers off.
8. Press **AMOUNT FED** to calculate the breast milk intake. The breast milk intake will be displayed along with the MILK INTAKE icon. The amount fed will be held on the display until it is manually cleared. To clear the held milk intake amount off the display, press **CLEAR/REWEIGH**. When the milk intake amount is cleared the display will show the After Feed weight.
Note: The patient can be removed from the weighing tray after **AMOUNT FED** is pressed. The patient does not need to be on the weighing tray for the scale to calculate the amount fed.
Note: The Amount Fed weight remains in memory even if the scale powers off.
9. To clear the After Feed weight from the display and return the scale to normal weighing mode, remove the patient from the weighing tray. After removing the patient from the tray, the Auto Hold feature allows the scale to continue displaying the weight measurement after the patient is removed. The default Auto Hold time is 30 seconds. To disable the Auto Hold or change to 60 seconds, see page 8.

Additional Milk Intake Functionality Notes

- A Before Feed or After Feed weight will not store if the scale did not lock on the weight.
- When in idle mode (scale is on but nothing is being weighed), users can press and hold **BEFORE FEED** or **AFTER FEED** to see the weight that was stored. To view the stored Breast Milk Intake amount, press **AMOUNT FED**. To exit out of the Amount Fed display, press **CLEAR/REWEIGH**.
- The stored weights remain in memory until a new Before Feed or new After Feed weight is stored.
- To manually clear the stored weights, press and hold **CLEAR/REWEIGH** for approximately 3 seconds. "CLEAR" will appear on the display. The Before, After and Amount Fed values will be set to zero in memory.
- A weight cannot be stored in Before Feed or After Feed during Auto Hold. A patient must be on the tray or the weight measurement is held on the display after manually pressing **HOLD/RLS/RECALL**.
- When storing a Before Feed or After Feed weight, the baby must be on the tray or the Hold button must be pressed while baby is on the tray.

CALIBRATION

Calibration

Your scale has been factory calibrated and does not require calibration prior to use. There are no requirements for calibration in the field; users should adhere to their institution's calibration policies. If required, the scale can be calibrated. A 5kg weight is required for calibration. If the scale is Everlocked into pounds, the calibration procedure will still require use of a 5kg weight.

Note: Avoid scale placement near heating/cooling vents, heating appliances, areas of high airflow or direct sunlight. Calibration must be done in a controlled environment.

Note: Only weights certified and traceable to national standards should be used for calibration procedures.

1. Press and hold **CLEAR/REWEIGH** and **HOLD/RLS/RECALL** at the same time, then press and release the  button to power on the scale. Remain pressing **CLEAR/REWEIGH** and **HOLD/RLS/RECALL** until "CAL" appears on the display. Release all buttons.
2. Once the buttons are released the following message will display, "CAL99", "CAL98" etc., until the display counts down to zero. This allows the load cells time to settle before calibration starts. The down arrow key can be pushed at any time to cancel the countdown, however this is not recommended.
3. Once the time has elapsed, "2Ero SCALE" will scroll horizontally across the screen. With no weight on the scale, press **ZERO/TARE**.
4. The scale will zero and "PLACE 5 ON SCALE PrESS EnTER" will scroll across the screen.
5. Place a certified 5kg weight on the center of the weighing tray, then press **ENTER/SELECT**. "CAL" will flash on the screen while the scale is calibrating.
 - A. If the weight is within calibration range, a five digit number will be displayed for approximately 5 seconds then display the calibration weight. Remove the weight from the scale and the scale will restart.
 - B. If the weight placed on the scale does not match the correct calibration standard (5 kg), "Out OF RANGE" will appear on the display. Calibration is not performed. Press **CLEAR/REWEIGH** to exit calibration mode. Obtain the correct amount of calibration weight and begin the calibration process again.

TROUBLESHOOTING

Troubleshooting

Before contacting service personnel, refer to the following instructions to check and to correct any failures. If you have further questions, call Health o meter® Professional Scales Customer Service at 1-800-815-6615.

Symptom	Possible Cause	Corrective Action
Scale does not turn on	<ol style="list-style-type: none"> 1. Batteries are not inserted correctly. 2. Batteries are not fully charged. 	<ol style="list-style-type: none"> 1. Check that batteries are “seated” properly in battery tray. See assembly section on page 7 to access the batteries and check their positioning. 2. Check that all new and fresh batteries are properly inserted into the battery tray.
Incorrect weight readings	<ol style="list-style-type: none"> 1. Tray is touching a wall or other object. 2. Tray is not level. 	<ol style="list-style-type: none"> 1. To test, remove all objects from tray. Turn scale on. Display should read “0”. Gently push down on the tray with your finger and release. The display should show some weight value and then quickly return to “0”. 2. Ensure the tray is locked completely onto the four locking pins.
Weight reading consistently too high or too low	Scale is out of calibration.	<ol style="list-style-type: none"> 1. See Calibration section on page 17 to recalibrate the scale. 2. Contact Health o meter Professional Scales Customer Service at 1-800-815-6615.

MAINTENANCE AND CLEANING

Maintenance

Caution: Before first use, or after long periods of non-use, check the scale for proper operation and function. If the scale does not operate correctly, refer to qualified service personnel.

1. Check the overall appearance of the scale for obvious damage, wear, and tear.
2. If applicable, inspect the AC adapter cord for cracking, fraying, and for broken/bent prongs.

Cleaning and Disinfecting

Proper care and cleaning is essential to ensure a long life of accurate and effective weighing.

Caution: Disconnect the scale from the AC adapter power source prior to cleaning the unit.

1. Health o meter® Professional recommends using one of the following solutions on a soft cloth or disposable wipe:
 - mild soap and water solution
 - 70% isopropyl alcohol
 - solution with 1-5% hydrogen peroxide concentrationAfter cleaning/disinfecting, wipe with a cloth dampened with water and then with a clean dry cloth. To avoid residue build-up or damp surfaces, ensure the screen and scale parts are completely dry after cleaning.
2. Never use rough or abrasive materials to clean the scale, as these will damage the scale's finish.
3. Do not submerge the scale in water or any other liquid.
4. Do not pour or spray fluids directly on the scale.

Replacement Parts

Parts may be available if a piece of the scale needs to be replaced. Contact Health o meter Professional Scales Customer Service at 1-800-815-6615 to inquire about the availability of these replacement parts.

Part #	Description
ADPT30	Power adapter <small>*AC adapter not available on 2210KL-AM-C models with firmware version 2.4 or 2.42 or scale versions not evaluated to model code SPE 3000</small>
30-00172	Keypad
40-00274	2 sided tray
40-00275	4 sided tray
60-00302	Feet

WARRANTY

Limited Warranty

What does the Warranty Cover?

This Health o meter® Professional scale is warranted from date of purchase against defects of materials or in workmanship for a period of two (2) years. If product fails to function properly, return the product, freight prepaid and properly packed to Pelstar, LLC (see "To Get Warranty Service", below, for instructions). If the manufacturer determines that a defect of material or in workmanship exists, the customer's sole remedy will be replacement of the scale at no charge. Replacement will be made with a new or remanufactured product or component. If the product is no longer available, replacement may be made with a similar product of equal or greater value. All replaced parts are covered only for the original warranty period.

Who is Covered?

The original purchaser of the product must have proof of purchase to receive warranty service. Please save your invoice or receipt. Pelstar dealers or retail stores selling Pelstar products do not have the right to alter, or modify or in any way change the terms and conditions of this warranty.

What is Excluded?

Your warranty does not cover normal wear of parts or damage resulting from any of the following: negligent use or misuse of the product, use on improper voltage or current, use contrary to the operating instructions, abuse including tampering, damage in transit, or unauthorized repair or alternations. Further, the warranty does not cover natural disasters, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights, and you may also have other rights that vary from country to country, state to state, province to province or jurisdiction to jurisdiction.

To get Warranty Service make sure you keep your sales receipt or document showing proof of purchase. Call (+1) 800-638-3722 or (+1) 708-377-0600 to receive a return authorization (RA) number, which must be included on the return label. Attach your proof of purchase to your defective product along with your name, address, daytime telephone number and description of the problem. Carefully package the product and send with shipping and insurance prepaid to:

Pelstar, LLC
Attention R/A# _____
Return Department
9500 W. 55th St.
McCook, IL 60525

Extended Warranty Available

This scale is eligible for Health o meter® Professional ScaleSurance Extended Warranty Program. ScaleSurance extends the warranty period for an additional two years. This extension to the standard Limited Warranty can be purchased with new scales or for a facility's existing scale before its current warranty has expired. To learn more, visit www.homscales.com/scalesurance/ or contact your medical supply distributor.

*Not available in all countries.

PELSTAR®, LLC

9500 West 55th St. McCook, IL 60525-7110 USA

1-800-638-3722 or 1-708-377-0600

PLEASE REGISTER YOUR SCALE FOR WARRANTY COVERAGE AT:

www.homscales.com

Health o meter® is a registered trademark of Sunbeam Products, Inc. used under license.

Health o meter® Professional products are manufactured, designed, and owned by Pelstar®, LLC.

We reserve the right to improve, enhance, or modify Health o meter® Professional product features or specifications without notice.

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Equipada con
Everlock[®]

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PELSTAR[®], LLC 9500 West 55th St. McCook, IL 60525-7110 USA www.hom scales.com

Health o meter[®] Professional 2210KL-AM, Health o meter[®] Professional 2210KL4-AM,
Health o meter[®] Professional 2210KG-AM, Health o meter[®] Professional 2210KG4-AM

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Health o meter® est une marque déposée de Sunbeam Products, Inc. utilisée sous licence.
Les produits Health o meter® Professional sont fabriqués, conçus et détenus par Pelstar®, LLC.