FilterFree Plus

Cool Mist Humidifier

Use and Care Manual

- ♥ Touch control display
- ♥ Whisper quiet
- ♥ Auto dimming lights
- ♥ Easy to fill and clean
- ♥ No filters required
- ♥ Auto shut off

This humidifier adds water vapor to the surrounding air. If you have any questions about the operation of your Vicks FilterFree Plus Cool Mist Humidifier, call our toll-free Consumer Relations line at 1-800-VAPOR-1-2 or at www.VicksHumidifiers.com/contact-us.

IMPORTANT!
READ AND SAVE THESE INSTRUCTIONS

VUL565
Series

Important Safety Instructions

READ AND SAVE THESE IMPORTANT SAFETY INSTRUCTIONS BEFORE USING THIS HUMIDIFIER

WHEN USING ELECTRICAL APPLIANCES, BASIC PRECAUTIONS SHOULD ALWAYS BE FOLLOWED TO REDUCE THE RISK OF FIRE, ELECTRICAL SHOCK AND INJURY TO PEOPLE, INCLUDING THE FOLLOWING:

1. The humidifier should **ALWAYS** be placed on a firm, flat, waterproof surface at least four feet (1.2 m) away from bedside, 12 inches (30 cm) from the wall and out of reach of children and pets. Be sure the humidifier is in a stable position and the power cord is away from heated surfaces and out of the way to prevent the humidifier from being overturned.
2. The humidifier should not be left unattended in a closed room since air could become saturated and could cause personal injury or property damage. Condensation can form on the front of unit, on walls and furniture, leave the room door partly open. **DO NOT** aim Mist Outlet directly at children, walls or furniture.
3. Before using the humidifier, extend the cord and inspect for any signs of damage. **DO NOT** use the unit if the cord has been damaged. **DO NOT** operate with power cord coiled or with twist tie in place.
4. The humidifier has a polarized plug (one blade is wider than the other) as a safety feature. This plug will fit into a polarized outlet only one way. If the plug does not fit fully into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. **DO NOT** attempt to defeat this safety feature.
5.  **CAUTION:** To avoid fire or electric shock hazard, plug the humidifier directly into a 120V AC electrical outlet. To avoid risk of fire, electric shock or personal injury, **DO NOT** use an extension cord or power strip.
6. The humidifier should **ALWAYS** be unplugged and emptied when not in operation or while being cleaned. Shut off and unplug humidifier before moving. **DO NOT** move or tilt humidifier while it is in operation. Plug and unplug unit with dry hands. **NEVER** pull by cord.
7. **DO NOT** operate the humidifier without water. Turn off and unplug unit when tank is empty.
8. Humidifier requires regular cleaning. Refer to and follow cleaning instructions.
9. **DO NOT** operate outdoors; this humidifier is intended for indoor residential use only.
10. **DO NOT** cover or insert objects into any openings on the unit.
11. **DO NOT** block intake or output vents.
12. **DO NOT** attempt to repair or adjust any electrical or mechanical functions on this humidifier. Doing so will void your warranty.
13. **DO NOT** place in direct sunlight, high temperature areas or near computers or sensitive electronic equipment.
14. **DO NOT** touch the nebulizer while the unit is plugged in as burns could result.
15. **DO NOT** touch skin with exposed VapoPad or place VapoPad on furniture, fabric, bedding or plastic.
16. **DO NOT** add any essential oils, or medications (e.g. Kaz Inhalant or Vicks VapoSteam) into Mist Outlet, Base, Water Reservoir or Water Tank.
17. For residential use only.

SAVE THESE INSTRUCTIONS

FCC Compliance

This device complies with Part 18 of the FCC rules.


WARNING: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply within the limits for Consumer ISM equipment, pursuant to Part 18 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference with radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio TV technician for help.

How it Works

The Vicks FilterFree Plus Cool Mist Humidifier is designed to quietly provide visible cool mist. This filter free humidifier has a built-in nebulizer that quietly vibrates the water at ultrasonic frequency to break the water droplets into a fine, cool mist that you can see and feel. The mist helps reduce dry air discomforts by increasing humidity levels. Enhance your humidification experience by using Vicks VapoPads for added comfort.

Components

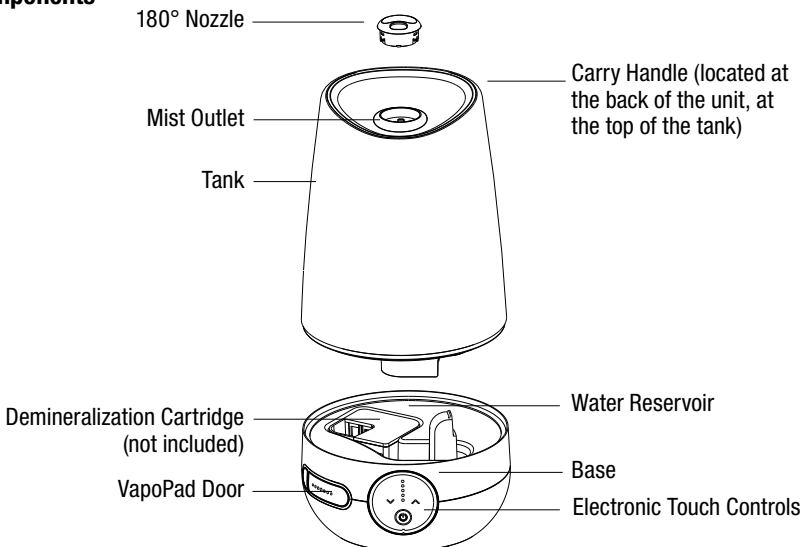Accessories (Not included)

The following accessories work great with your humidifier:

Vicks VapoPads (One menthol VapoPad sample included)

Use Vicks VapoPads with your humidifier for soothing vapors. Enjoy up to 8 hours of soothing comfort per pad. Choose from three scents: Soothing Menthol Vapors, Comforting Sleepy Time Comfort (eucalyptus, rosemary and lavender) and Calming Vapors (Menthol and Lavender).

VSP-19FP VBR-5FP VSP30FP

Protec® Demineralization Cartridge


This humidifier works with a Demineralization Cartridge that helps remove minerals from the water. This helps prevent the build up of minerals ("white dust") that can occur in hard water areas. If you live in a hard water area, we recommend the use of this demineralization cartridge if not using distilled water.

PDC51V2 Series

Setting Up Your Humidifier


1. Remove packaging materials. **Keep instructions.**
2. Remove twist tie and extend power cord.
3. Place on a firm, level, water resistant surface a minimum of 12 inches (30 cm) from any walls.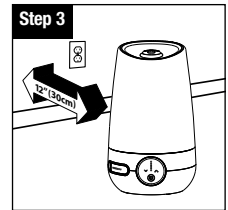

Filling/Refilling

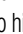
NOTE: Tank should be carried using two hands, in the carry handle located on Tank Top and the other hand supporting the base of the Tank. **Manufacturer and Distributor will not accept responsibility for property damage caused by water spillage.**


1. Remove Tank and turn upside down.
2. Turn Tank Cap in the direction of the open lock symbol to remove cap.
3. Fill Tank with cool water. **DO NOT** fill with warm or hot water. **DO NOT** ADD ESSENTIAL OILS, MEDICATIONS, OR INHALANT LIQUIDS such as Vicks VapoSteam to the Water Tank. Doing so could cause damage to the humidifier and void the warranty.
4. **Securely** replace Tank Cap, aligning arrow to lock symbol.
5. Place Tank back on Base.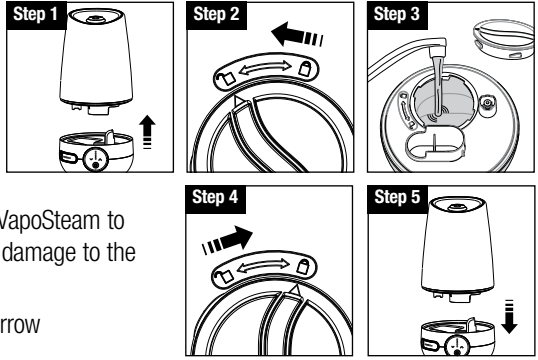

⚠ CAUTION: USE CARE WHEN HANDLING WATER TANK TO AVOID UNNECESSARY IMPACT. BANGING OR DROPPING THE WATER TANK COULD RESULT IN DAMAGE TO THE TANK THAT WILL CAUSE LEAKING.

Operating Humidifier


1. Plug the humidifier in. Power on by pressing the power icon. The unit will default to high output, continuous humidity setting. Press again to turn off. The humidifier has a memory function and will revert to the last setting each time you power on. **⚠ CAUTION: TO AVOID ELECTRIC SHOCK, DO NOT PLUG HUMIDIFIER INTO AN OUTLET WITH WET HANDS.** Unplugging the humidifier resets to default settings.
2. **Output:** Press the up and down arrows   to toggle between the low and high mist level settings. A higher setting will deliver mist into the room at a higher rate. The water level in the Water Tank will decrease faster on a higher setting, since more moisture is going into the air. A lower setting will provide less moisture output and run longer before the Water Tank needs to be refilled.
3. Position humidifier to direct mist output away from walls, furniture and bedding.
4. **Humidity Level:** When a comfortable humidity level is reached, lower the output setting. A comfortable humidity level is between 40-60%.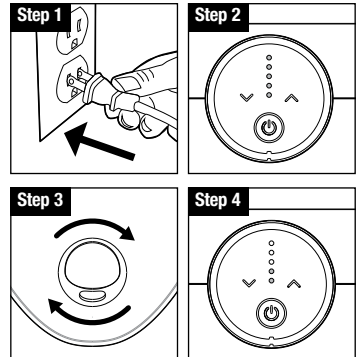

To monitor your humidity level, use a humidity monitor such as the Vicks V70 or V70-CAN model. (Not included)
If you notice condensation on the walls, windows or around unit, your humidity level is too high.
Turn humidifier off.

NOTE: IF THE POWER ICON  CHANGES TO AN AMBER COLOR, THE TANK IS EMPTY. TURN THE POWER OFF, REFILL AND REPLACE THE TANK AND TURN THE POWER BACK ON TO CONTINUE TO USE THE HUMIDIFIER.

Using Vicks VapoPads

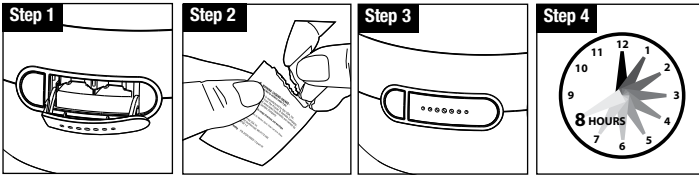Your humidifier can be used with Vicks VapoPads to provide up to 8 hours of soothing vapors. One free sample of the menthol VapoPad is included with your humidifier. To use your unit with Vicks VapoPads:

1. Open VapoPad Door.
2. Open VapoPad by tearing notch on pad bag. **DO NOT** touch pad with hands. If pad is touched **DO NOT** rub face or eyes as it may cause irritation.
3. Insert Vicks VapoPad with angled end facing towards the product. Up to two VapoPads can be inserted. Close door.
4. After 8 hours, discard scent pad. To remove the scent pad(s) open the VapoPad Door by pressing the button on the left side of the Scent Pad area and the VapoPad will eject slightly to be removed.

To continue using scent pads, repeat steps 1-4.

VapoPad Recommended Usage:

- Not recommended for infants under 10 lbs (4.5 kg).
- For children between 10-22 lbs (4.5-10 kg) do not use more than 2 pads in a 24 hr period.

Sample VapoPad ingredients: Eucalyptus Oil, Menthol, Glycol, Cedar Leaf Oil & Others.

NOTE: ALWAYS reference VapoPad packaging for recommended usage instructions and warnings.

Water Quality and Your Humidifier

High mineral content in your water, also known as “hard water” may cause a white mineral residue to accumulate on surfaces in the room near the humidifier. This mineral residue is commonly referred to as “White Dust.” The higher the mineral content or hardness of your water, the greater the potential that you may have white dust. “White Dust” is not a result of a defect or flaw in the humidifier. It is a result of the amount of minerals suspended in the water you are using. **If you live in a hard water area, the use of distilled water will minimize the production of white dust.**

The use of a Protec® Demineralization cartridge (PDC51V2/ PDC51CV2 Series) will help reduce white dust. For best results replace as necessary (every 2-4 weeks, or 15-30 fillings under normal use) or when the humidifier has not been run for an extended period of time.

Weekly Cleaning







It is recommended that you empty the Water Tank and Reservoir when the unit is not in use and let them air dry. To keep your humidifier running efficiently, clean it regularly. Weekly cleaning is recommended. To properly clean your humidifier we recommend the separate processes of Scale Removal and Disinfecting shown below. **These two processes must be performed separately.**

⚠ CAUTION: BEFORE MOVING HUMIDIFIER BASE, TURN OFF, UNPLUG AND EMPTY THE TANK. All maintenance should be done in the kitchen or bathroom on a water resistant surface near a faucet. **DO NOT** wash any components of this humidifier in the dishwasher.

Instructions for Scale Removal

1. Turn off and unplug the humidifier. Remove any accessories in tank.
2. Remove Water Tank from Base. Remove Tank Cap, taking care that the rubber Cap Seal does not come off. Empty water from Water Tank and Base.
3. Add 2 cups of undiluted, distilled white vinegar to Water Tank. Replace Tank Cap and swish vinegar solution around in Tank. Place Tank on Base. Vinegar solution will drain into Water Reservoir and loosen mineral buildup (scale) on Nebulizer and Float as they soak in the solution. It will also loosen scale on bottom of Water Tank. **DO NOT** run humidifier with vinegar solution.
4. Allow the vinegar to remain in Tank and on Base for 15-20 minutes.
5. After soaking, unlock Tank Cap and pour solution out in sink. Pour solution from Water Reservoir out in sink. Wipe Nebulizer and Float with soft cloth to remove loosened mineral deposits.
6. Rinse Water Reservoir and Water Tank until smell of vinegar is gone. Avoid getting water into any openings on the humidifier base.

⚠ CAUTION: DO NOT submerge Base in water. Doing so will damage the humidifier and void the warranty.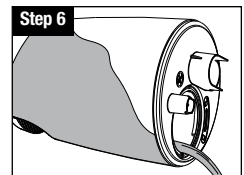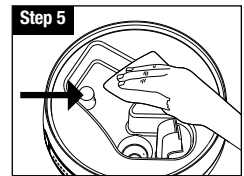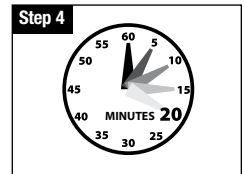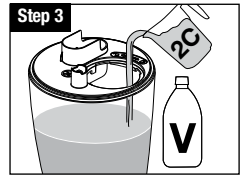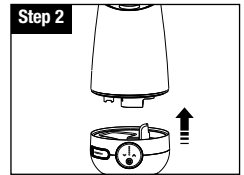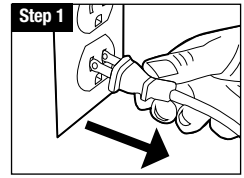

Weekly Cleaning (Continued)

Instructions for Disinfecting

Follow Steps 1-3 in Scale Removal Instructions in previous section.

NOTE: Bleach may stain clothes. It is recommended that you wear rubber gloves during this process to protect your hands.

1. Add 1 tsp. of bleach to 1 gallon (3.8 L) of water and add to Water Tank. Replace Tank Cap. Swish solution around in Tank. Place Tank on Base. Bleach solution will drain into Water Reservoir and will disinfect Water Reservoir.



NOTE: Using more than 1 tsp. of bleach per gallon of water may result in damage to your humidifier.

2. Allow the bleach solution to remain in Tank and on Base for 15-20 minutes.
3. Unlock Tank Cap and pour solution out in sink. Pour solution from Water Reservoir out in sink. Wipe Nebulizer, Nozzle and Float with a soft cloth.
4. Rinse Water Reservoir and Water Tank until smell of bleach is gone. Avoid getting water into any openings on the humidifier base.

NOTE: DO NOT mix vinegar and bleach solutions together. **Scale Removal and Disinfecting must be done separately.**

To clean the scent pad area, gently wipe out scent pad heater area with a soft damp cloth.

DO NOT USE VINEGAR OR BLEACH SOLUTION TO CLEAN THIS AREA.

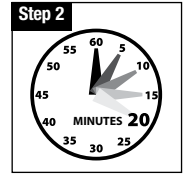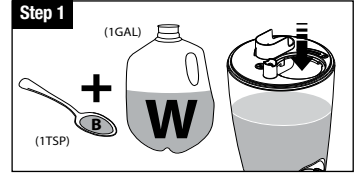End-of-Season Care and Storage

Follow the **CLEANING** instructions when the humidifier will not be used for at least one week, or it is the end of the season. At the end of the season, remove any accessories used with the humidifier and allow the humidifier and accessories to dry completely before storing.

DO NOT store with water inside the Base, Water Reservoir or Tank.

Pack unit safely and store in a cool, dry location.

Troubleshooting

If your humidifier is not working properly, refer to the following:

Problem	Possible Cause	Solution
Power button has been pressed but the control panel lights are off.	<ul style="list-style-type: none">• Unit not plugged in• No power at outlet	<ul style="list-style-type: none">• Plug unit in• Check home circuits, fuses; test outlet
Power is on and little or no mist is produced	<ul style="list-style-type: none">• No water in tank• Unit is not level• Mineral deposits on Nebulizer• Power setting may be too low• Water Tank or Base were washed with detergent	<ul style="list-style-type: none">• Fill Water Tank with cool water• Place on a level surface• Clean Nebulizer (refer to Weekly Cleaning instructions)• Press the 'up' arrow to increase output• Rinse thoroughly with clean water until there is no foam or smell of cleaning solution
Water overflows from Reservoir	<ul style="list-style-type: none">• Tank may be damaged• Tank Cap is not tight• Tank Cap Seal may be missing	<ul style="list-style-type: none">• Check the Tank for any cracks or damage and replace if necessary• Close Tank Cap to lock position• Check to see if Cap Seal is attached to Tank Cap. If not, contact Consumer Relations
Condensation forms around humidifier or on windows and walls	<ul style="list-style-type: none">• Mist Intensity is set too high for room size or pre-existing humidity level• Humidifier may be too close to wall or window	<ul style="list-style-type: none">• Decrease mist output to lower setting or open door to the room• Move humidifier away from wall or point Mist Outlet away from wall or window

NOTE: Failure to keep this unit clean from mineral deposits normally contained in any water supply will affect the efficiency of operation of this unit. Customer failure to follow these instructions may void the warranty.

Consumer Relations

For questions, technical support or to order replacement parts, contact Consumer Relations.

Call us toll-free at: 1-800-VAPOR-1-2 (1-800-827-6712)

Hours: Mon-Fri 8:00AM-6:00PM EST

Or visit our website at: www.VicksHumidifiers.com/contact-us

Please be sure to specify model number.

Electrical Ratings

Capacity: 1.2 gallons (4.5 L) Electrical Rating: 120V, 60 Hz. 25W.

2 Year Limited Warranty

YOU SHOULD FIRST READ ALL INSTRUCTIONS BEFORE ATTEMPTING TO USE THIS PRODUCT.

- A. This 2 year limited warranty applies to repair or replacement of product found to be defective in material or workmanship. This warranty does not apply to damage resulting from commercial, abusive, unreasonable use or supplemental damage. Defects that are the result of normal wear and tear will not be considered manufacturing defects under this warranty. **KAZ IS NOT LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES OF ANY NATURE, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS LIMITED IN DURATION TO THE DURATION OF THIS WARRANTY.** Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you also may have other rights which vary from jurisdiction to jurisdiction. This warranty applies only to the original purchaser of this product from the original date of purchase.
- B. At its option, Kaz will repair or replace this product if it is found to be defective in material or workmanship.
- C. This warranty does not cover damage resulting from any unauthorized attempts to repair or from any use not in accordance with the instruction manual.
- D. This warranty does not cover the filters, pre-filters, UV bulbs or other accessories (if included) except for material or workmanship defects.

Contact us at 1-800-827-6712 or at www.VicksHumidifiers.com/contact-us

Please be sure to specify a model number.

NOTE: IF YOU EXPERIENCE A PROBLEM, PLEASE CONTACT CONSUMER RELATIONS FIRST OR SEE YOUR WARRANTY. DO NOT RETURN THE HUMIDIFIER TO THE ORIGINAL PLACE OF PURCHASE. DO NOT ATTEMPT TO OPEN THE BASE OR TAMPER WITH THE POWER KNOB YOURSELF, DOING SO MAY VOID YOUR WARRANTY AND CAUSE PERSONAL INJURY OR DAMAGE TO THE PRODUCT.

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© 2022 All Rights Reserved.

Manufactured by: Kaz USA, Inc., a Helen of Troy Company
400 Donald Lynch Blvd., Suite 300, Marlborough, MA 01752

Certain trademarks used under license from The Procter & Gamble Company or its affiliates.

Protec® is a registered trademark owned by Helen of Troy Limited.

To contact us call 1-800-VAPOR-1-2 (1-800-827-6712) or visit our website at www.VicksHumidifiers.com/contact-us

Made in China.

For Responsible recycling, please visit:

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Fabricado por Kaz USA, Inc., a Helen of Troy Company
Protec® es una marca registrada de Helen of Troy Limited.