# StarryNight

**Cool Moisture Humidifier**

**USE AND CARE MANUAL**

- ♥ Cool, invisible moisture
- ♥ Filters minerals from the water
- ♥ Multiple output settings

If you have any questions about the operation of your Vicks Starry Night Cool Moisture Humidifier, call our toll-free Consumer Relations line at 1-800-VAPOR-1-2 (1-800-827-6712) or contact us at [www.VicksHumidifiers.com/contact-us](http://www.VicksHumidifiers.com/contact-us).

**IMPORTANT!**  
**READ AND SAVE THESE INSTRUCTIONS**

**V3700**  
**Series**

# Important Safety Instructions






## READ AND SAVE THESE IMPORTANT SAFETY INSTRUCTIONS BEFORE USING THIS APPLIANCE

**WHEN USING ELECTRICAL APPLIANCES, BASIC PRECAUTIONS SHOULD ALWAYS BE FOLLOWED TO REDUCE THE RISK OF FIRE, ELECTRICAL SHOCK AND INJURY TO PERSONS, INCLUDING THE FOLLOWING:**


1. The appliance should **ALWAYS** be placed on a firm, flat, waterproof surface at least four feet away from bedside, six inches from the wall and out of reach of children and pets. Be sure the appliance is in a stable position and the power cord is away from heated surfaces and out of the way to prevent the Humidifier from being tipped over.
2. The appliance should not be left unattended in a closed room since air could become saturated and cause personal injury or property damage. Leave room door partly open. **DO NOT** aim mist directly at children, walls or furniture.
3. Before using the appliance, extend the cord and inspect for any signs of damage. **DO NOT** use the unit if the cord has been damaged. **DO NOT** operate with power cord coiled or with twist tie in place.
4. The appliance has a polarized plug (one blade is wider than the other) as a safety feature. This plug will fit into a polarized outlet only one way. If the plug does not fit fully into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. **DO NOT** attempt to defeat this safety feature.
5.  **CAUTION:** To avoid fire or shock hazard, plug the appliance directly into a 120V AC electrical outlet. To avoid risk of fire, shock or personal injury, **DO NOT** use an extension cord or power strip.
6. The appliance should always be unplugged and emptied when not in operation or while being cleaned. Shut off and unplug humidifier before moving. **DO NOT** move or tilt humidifier while it is in operation. Plug and unplug unit with dry hands. **NEVER** pull by cord.
7. **DO NOT** operate the appliance without water. Turn off and unplug unit when tank is empty.
8.  **CAUTION:** When moving or lifting the full Water Tank, use two hands; one hand on the Tank Handle and the other on the bottom of the Tank.
9. Appliance requires regular cleaning. Refer to and follow cleaning instructions.
10. **DO NOT** operate outdoors; this appliance is intended for indoor use only.
11. **DO NOT** cover or insert objects into any openings on the unit.
12. **DO NOT** block intake or output vents.
13. **DO NOT** attempt to repair or adjust any electrical or mechanical functions on this appliance. Doing so will void your warranty.
14. **DO NOT** touch skin with exposed Vicks VapoPads or place on furniture, fabrics, bedding or plastics.
15. **NEVER** add any essential oils, medications or inhalant liquids to the Water Tank, such as Vicks VapoSteam or Kaz Inhalant.
16. For residential use only.

**SAVE THESE  
INSTRUCTIONS**

# Vicks Starry Night Cool Moisture Humidifier (V3700 Series)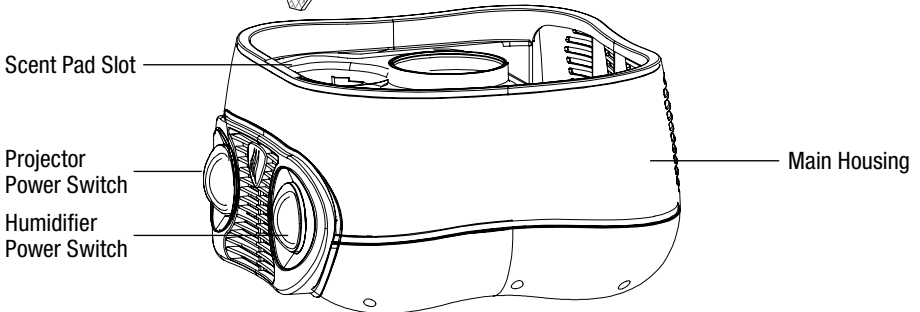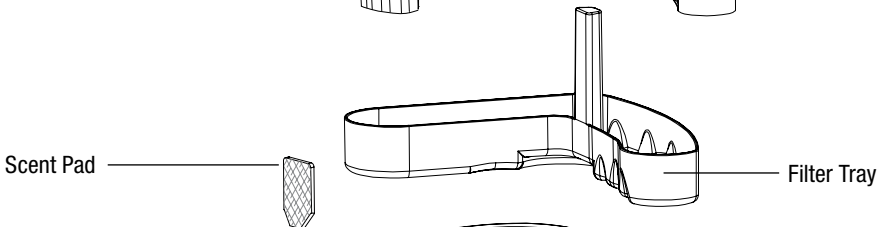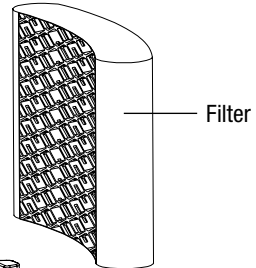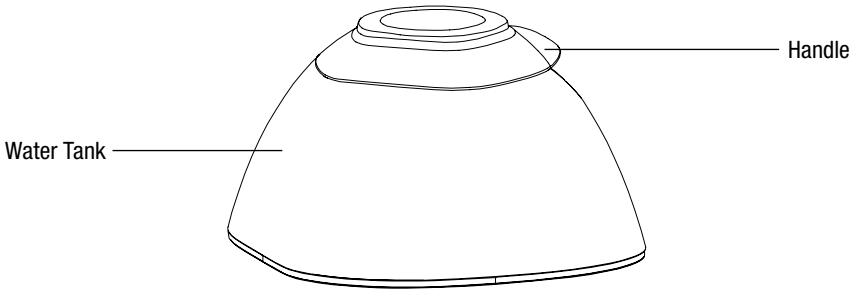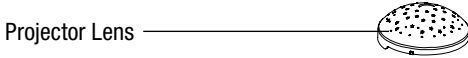


# Setting Up Your Humidifier

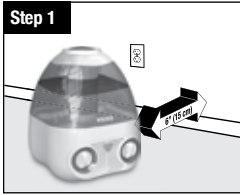1. **Placement:** Select a firm, level location at least 6" (15cm) from any walls for proper airflow. Place the humidifier on a water-resistant surface, as water can damage furniture and some flooring. Kaz will not accept responsibility for property damage caused by water spillage.

## Soak Filter

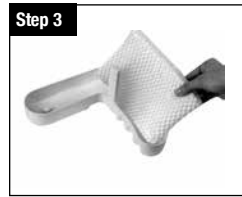1. Remove tank and set aside. Remove filter from humidifier and soak the filter in cool water. **DO NOT** wring filter.
2. Remove filter tray from unit before installing the wet filter.
3. Fold filter flat. Insert wet filter into filter tray. Place filter tray back in humidifier.

## Scent Pad

1. Remove tank and locate Scent Pad slot.
2. If there is a used scent pad in the slot carefully remove it. Replace with a new scent pad. Open VapoPad sachet by tearing notch on pad bag. **DO NOT** touch pad with hands. If pad is touched, do not rub face or eyes as it may cause irritation.
3. Insert scent pad with angled side facing downward into the scent pad slot. You can use the bag as a holder during insertion.

Vicks Scent Pads work with heat to gradually release soothing aromatic vapors. One complimentary scent pad is included with your unit.

## VapoPads Recommended Usage:

- Not recommended for infants under 10 pounds.
- For children between 10-22 pounds, **DO NOT** use more than 2 pads in a 24 hour period.

# Setting Up Your Humidifier

(Continued)

**⚠ WARNING: DO NOT rub face or eyes after touching scent pad. DO NOT put fingers in mouth after touching scent pad. Oils in the pad could act as an irritant to eyes.**

**⚠ WARNING: As long as your humidifier is on, the scent pad heater will also be activated. This area can be warm to the touch.**

**NOTE:** Replace with Vicks Scent Pads, models VSP-19/VSP-19-CAN or Vicks Sleepy Time Vapopad VBR-5, available at most retailers or [www.VicksHumidifiers.com](http://www.VicksHumidifiers.com).

## Ingredients:

Menthol Vapors (VSP-19 Series): Eucalyptus Oil, Menthol, Glycol, Cedar Leaf Oil & Others

Sleepy Time Vapors (VBR-5 Series): Eucalyptus Oil, Camphor, Rosemary Oil, Lavender 40/42 & Others

## Fill Tank

1. Turn tank upside-down. Remove tank cap by turning counter-clockwise ↶.
2. Fill tank with cool tap water. Replace tank cap. Tighten by turning clockwise ↷. Place tank back on humidifier.
3. Once the filled water tank has been placed on the unit do not move humidifier. Doing so could cause leakage and spills.

**DO NOT Use Hot Water when Filling Humidifier. DO NOT ADD ESSENTIAL OILS, MEDICATIONS OR INHALANT LIQUIDS** to the Water Tank such as Vicks VapoSteam or Kaz Inhalant. Doing so could cause damage to the humidifier and void the warranty.

# Operating Your Humidifier

1. **Power:** With the power switch in the OFF position, plug the filled humidifier into a polarized 120V outlet. **DO NOT FORCE THE POLARIZED PLUG INTO THE OUTLET;** it will only fit one way.

**⚠ DO NOT PLUG IN HUMIDIFIER WITH WET HANDS TO AVOID ELECTRIC SHOCK.**

1. **Output:** Set fan speed to desired output: High (☂) Max Moisture, Low (☂) Night time sound sleep mode

1. **Starry Night Projector:** Turn the power switch to the on position to activate the starry night projector


**NOTE:** Projector can be operated independently of the humidifier.

## Changing Your Filter

**NOTE:** It is recommended that all maintenance be done in the kitchen or bathroom on a water resistant surface. Remove water tank from unit before moving.

The filter should be changed every 30 to 60 days depending on your water quality. Keep in mind, depending on your water quality, mineral deposits can still clog or discolor the filter, affecting its life and that of the humidifier. This product does not protect users or others from bacteria, virus, or other disease causing organisms.

### To change the filter: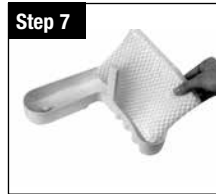

1. Uses replacement filter WF2.
2. Turn off and unplug your humidifier.
3. Remove the water tank and filter tray.
4. Remove filter tray from unit. Discard old filter and set filter tray aside.
5. Soak new filter in cool water.
6. **DO NOT** wring filter.
7. Insert new filter into filter tray. Place filter tray back in humidifier.

**NOTE:** For best fit and performance remove filter tray from the humidifier and insert presoaked new filter.

8. Replace the water tank.

# Wicking Filter Maintenance

It is recommended you change your Wicking Filter every 30-60 days depending on water quality and usage. If you have hard water you may need to replace your filter more frequently. Change your filter if you notice:

- The filter is hard and crusty
- The filter starts to give off an odor
- Moisture output is decreased

Be sure to use only Protec® Replacement filters. This humidifier takes Protec PWF2 Replacement Filter.

To prolong the life of your Wicking Filter, turn it over each time you fill the Water Tank. This will keep the top of the filter from drying out and will help the filter to age more evenly.

If you notice the filter getting hard, you may soak the Wicking filter in cool water to loosen mineral buildup. This will temporarily improve the performance until a replacement filter is purchased.


**DO NOT** attempt to clean the Wicking Filter with vinegar or any chemical solutions. Doing so may damage the Filter.

## Cleaning

Cleaning involves two steps, SCALE REMOVAL and DISINFECTING. Be sure to perform these in the proper order to avoid harmful chemical interactions.



**NOTE:** REMOVE FILTER BEFORE CLEANING. Failure to remove filter during cleaning will destroy the filter

### Scale Removal (Recommend this process be performed on a weekly basis)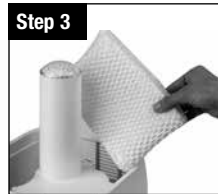

1. Turn off and unplug your humidifier.
2. Remove and empty water tank.
3. Remove filter and set aside.
4. Remove Filter Tray and fill filter tray and water tank with undiluted vinegar and soak for 10-15 minutes.
5. Using undiluted vinegar wipe base thoroughly.
6. Rinse water tank, filter tray and base thoroughly with water to remove scale.

## Disinfecting (Recommend this process be performed on a weekly basis)

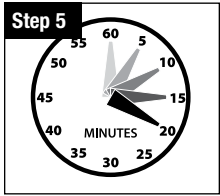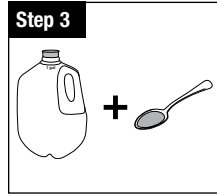1. Turn off and unplug your humidifier.
2. Remove and empty water tank.
3. Remove filter and set aside. Dilute 1 tablespoon (TBS) of bleach in 1 gallon of cool water.
4. Fill water tank with water and bleach solution. Remove water tray from unit and fill with the remaining water and bleach solution. Wipe base with the same solution.
5. Let stand for 20 minutes.
6. Rinse water tray and water tank thoroughly with water until the smell of bleach is gone.

**NOTE:** The use of other water treatment products and chemicals may cause deterioration of the filter or damage the humidifier, thereby hindering the humidifier's effectiveness.

## End-of-Season Care And Storage

Follow the CLEANING instructions when the humidifier will not be used for at least one week or more, or at the end of the season.

At the end of the season, remove and throw away the filter; **DO NOT** store with a used filter.

Dry the humidifier completely before storing. **DO NOT** store with water inside the Base or Water Tank.

Pack the unit in its original carton and store in a cool, dry location.

Before next use, remember to clean your humidifier and install a new filter.

## Consumer Relations

**To order replacement parts** contact Consumer Relations

Call us toll-free at: 1-800-VAPOR-1-2 (1-800-827-6712)

Or visit our website at: [www.VicksHumidifiers.com/contact-us](http://www.VicksHumidifiers.com/contact-us)

**Please be sure to specify model number V3700.**

## To Order Replacement Parts

V3700-**Star Projector Cover**

V3700-**Star Projector Lens**

V3700G-**Star Projector Lens (green)**

V3700P-**Star Projector Lens (purple)**

V3700M- **Star Projector Lens (pink)**

V3700-**Filter Tray**

Replacement Menthol Scent Pads-**VSP-19/VSP-19-CAN**

Replacement Soothing Sleepy Time Scent Pads **VBR-5**

Replacement filter part number **WF2**

V3700-**Tank**

V3700G-**Tank (green)**

V3700P-**Tank (purple)**

V3700M-**Tank (pink)**

V3700-**Tank Cap**

**STOP:** IF YOU EXPERIENCE A PROBLEM, PLEASE CONTACT KAZ CONSUMER RELATIONS OR SEE YOUR WARRANTY.

## Electrical Ratings

The Vicks models are rated at 120V, 60 Hz.

**NOTE: DO NOT ATTEMPT TO OPEN THE MOTOR HOUSING YOURSELF. DOING SO MAY VOID YOUR WARRANTY AND CAUSE DAMAGE TO THE HUMIDIFIER OR PERSONAL INJURY.**

## Troubleshooting

**Q: I hear the fan turning, but I don't see any mist. Is my humidifier operating properly?**

**A:** Dry air is pulled through the system and a fan disperses invisible, moisture balanced air. There is no "mist" emitted. When operating normally you will feel only cool air coming from the unit. Monitor the water level in your Tank/ Base. If you notice a decrease in the Water Tank over 24 hours then the unit is operating normally.

**Q: My unit has been running for several hours. I hear the fan spinning, but I do not notice the water level in the tank decreasing.**

**A:** First, check the filter to ensure it is wet. If the pad appears dry, remove it from the Base and gently rinse under cool water. Reposition wet filter in the Base. You may also want to note the current moisture level in your home. If the humidity level is over 60% the unit will not put out much moisture as your air is already saturated.

**Q: My filter has turned light brown and rust colored. Do I need a new filter?**

**A:** Since your filter has been made with Protec<sup>®</sup>, the change in color indicates only a mineral deposit. Soak filter in a sink full of cool water to help break up deposits. The discoloration may remain but will not affect the humidifier's performance. If performance is affected then it is time to replace the filter.

**Q: My humidifier doesn't appear to be as clean as when I first purchased and/or I'm smelling an unpleasant odor. What should I do?**

**A:** If you have been using your humidifier regularly for over 30 days, follow the cleaning instructions. Weekly cleaning is recommended.

**Q: How do I know if it's time to replace my filter?**

**A:** If your moisture output is decreased or the filter becomes hard, crusty or starts giving off an odor, it is time to replace your filter.

**Q: I recently bought my humidifier and it is operating properly. The fan is running and the water level in the tank is decreasing but the room still feels very dry.**

**A:** If you have just started using your humidifier, it will take a few days to return the normal moisture balance to your room. Wood furniture, flooring, carpeting and other objects in the room will absorb moisture before it can re-saturate the air. If you run this humidifier with the door open you may be humidifying a larger area than this humidifier was intended for. Try running with the doors closed temporarily until the room has reached the desired level of humidity. If your house does not have adequate insulation this could effect the amount of moisture your room is maintaining.

**Q: I'd like to know what my relative humidity level is in my house, how can I obtain this information?**

**A:** You can purchase a device such as the Vicks Humidity Monitor (model V70) that will display the relative humidity level in your room.

## 3 Year Limited Warranty

**YOU SHOULD FIRST READ ALL INSTRUCTIONS BEFORE ATTEMPTING TO USE THIS PRODUCT.**

- A. This 3 year limited warranty applies to repair or replacement of product found to be defective in material or workmanship. This warranty does not apply to damage resulting from commercial, abusive, unreasonable use or supplemental damage. It also does not apply to the filter or Protec™ Cleaning Cartridge. Defects that are the result of normal wear and tear will not be considered manufacturing defects under this warranty. **KAZ IS NOT LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES OF ANY NATURE. ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS LIMITED IN DURATION TO THE DURATION OF THIS WARRANTY.** Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you also may have other rights which vary from jurisdiction to jurisdiction. This warranty applies only to the original purchaser of this product from the original date of purchase.
- B. At its option, Kaz will repair or replace this product if it is found to be defective in material or workmanship. Defective product should be returned to the place of purchase in accordance with store policy. Thereafter, while within the warranty period defective product may be returned to Kaz.
- C. This warranty does not cover damage resulting from any unauthorized attempts to repair or from any use not in accordance with the instruction manual.
- D. This warranty does not cover the filters, pre-filters, UV bulbs or other accessories (if included) except for material or workmanship defects.

Call us toll-free at 1-800-VAPOR-1-2 (1-800-827-6712) or contact us at [www.VicksHumidifiers.com/contact-us](http://www.VicksHumidifiers.com/contact-us)  
Please be sure to specify Model Number V3700.

**NOTE: IF YOU EXPERIENCE A PROBLEM, PLEASE CONTACT CONSUMER RELATIONS FIRST OR SEE YOUR WARRANTY. DO NOT RETURN THE PRODUCT TO THE ORIGINAL PLACE OF PURCHASE. DO NOT ATTEMPT TO OPEN THE MOTOR HOUSING YOURSELF, DOING SO MAY VOID YOUR WARRANTY AND CAUSE DAMAGE TO THE PRODUCT OR PERSONAL INJURY.**

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© 2023 All Rights Reserved.
Certain trademarks used under license from The Procter & Gamble Company or its affiliates.
Protec® is a registered trademark owned by Helen of Troy Limited.
Kaz USA, Inc., a Helen of Troy Company
Kaz USA, Inc., una Empresa de Helen of Troy
Protec® es una marca registrada propiedad de Helen of Troy Limited.