**kaz** Manufacturing Quality Healthcare Products for Over 75 Years

Kaz USA, Inc.  
250 Turnpike Road  
Southborough, MA • 01772 • USA  
www.kaz.com • consumerrelations@kaz.com

©2011 All Rights Reserved.  
Kaz USA, Inc  
Made in Mexico

The Vicks® Warm Steam Vaporizer is manufactured by Kaz USA, Inc. under license from The Procter & Gamble Company, Cincinnati, OH 45202. VICKS, the shield design, the steam design, and other related trademarks are owned by The Procter & Gamble Company. VapoPads is a registered trademark owned by Kaz Inc., Southborough, MA 01772

Protected by  
US Patent No. D529,159  
Other patents pending

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## Use and Care Manual / Guide d'utilisation Manual de Uso y Cuidado

**IMPORTANT! READ AND SAVE THESE INSTRUCTIONS**

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If you have any questions about the operation of your Vicks Vaporizer, call our toll-free Consumer Service line at 800 477-0457

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If vaporizer does not produce steam after 10 minutes add 1 to 2 pinches (1/8 tsp) of salt. Wait an additional 10 minutes. If vaporizer is still not generating steam add another pinch or two of salt as needed (up to 1/2 tsp).

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**Note:** Please do not add salt to your vaporizer each time you operate the product. Salt will build up over time and you may not need to repeat the process.

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## Important Safety Instructions

### READ ALL INSTRUCTIONS BEFORE USING VAPORIZER, ESPECIALLY THESE BASIC SAFETY PRECAUTIONS:

#### IMPORTANT!

For the most effective and safe use of your vaporizer, please READ AND SAVE THESE INSTRUCTIONS... especially those regarding the addition of salt to the water. In "soft water" areas, salt may be needed in the water to generate steam. In "hard water" areas, the vaporizer will operate without the addition of salt.

#### **⚠ WARNING: KEEP OUT OF REACH OF CHILDREN**


We recognize that many of our customers use vaporizers in homes with young children. Carefully supervise your children when using a vaporizer, especially crawling infants and toddlers. Please be sure to take the time to instruct them that a vaporizer is not a toy... it is a serious device that produces hot steam and could cause severe burns and injuries if they were to come in contact with the unit. Place vaporizer in an area where both the vaporizer and power cord are out of reach of children.

### READ AND SAVE THESE INSTRUCTIONS

1. **Vaporizer should always be placed on a firm, flat, waterproof surface at least four feet away from bedside and out of reach of patient and children.** Be sure the vaporizer is in a stable position and the power cord is out of the way to prevent the vaporizer from being tipped over.
2. Before using the vaporizer, extend the cord and inspect for any signs of damage. **Do not** use the vaporizer if the cord has been damaged.

3. **Do not** place the vaporizer directly on wood furniture or other surfaces that could be damaged by water. Protect by placing the vaporizer on a non-metallic waterproof surface.
4. **Place the vaporizer in an area that is out of the reach of children.**
5. **Do not** operate outdoors; this appliance is intended for indoor use.
6. **Do not** remove the vaporizer head during operation or within 20 minutes after it is unplugged. Serious injury may result.
7. **Do not** touch the steam vapor. Steam can cause burns. **Do not** add liquid inhalants or Vapopads when the vaporizer is plugged in or it is operating.
8. **Do not** operate without water. Unplug unit when tank is empty.
9. **Do not** cover or insert objects into any openings on the vaporizer. **Do not** pour liquid inhalants into the Vapopad slots or into the steam outlet.
10. **Carefully Read Instructions** before adding any medications (e.g., do not add Kaz Inhalant or Vicks VapoSteam® directly into the vaporizer or medicine cup area without first reading the directions in the owners manual and on the Inhalant packaging/bottle.)
11. **Do not** immerse the vaporizer head in water or other liquids. **Do not** pour water into the steam outlet or into the Vapopad slots
12. The vaporizer should always be unplugged and emptied when not in operation or while being cleaned. **Unplug** the vaporizer before moving. **Do not** move or tilt the vaporizer while it is operating. **Plug and unplug the vaporizer with dry hands.**
13. The vaporizer should not be left unattended in a closed room since air could become saturated and leave condensation on walls or furniture. Leave room door partly open. **Do not** aim outlet directly at children, walls or furniture.
14. This vaporizer has a polarized plug (one blade is wider than the other) as a safety feature. This plug will fit into a polarized outlet only one way. If the plug does not fit fully into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. **Do not** attempt to defeat this safety feature.
15. **⚠ Caution:** To avoid the risk of fire, shock or personal injury, **do not** use an extension cord.
16. The vaporizer requires regular cleaning. Refer to and follow cleaning instructions.
17. **Do not** attempt to carry by the vaporizer head. The vaporizer should only be moved when it is unplugged and has been allowed to cool for 20-30 minutes.  
The vaporizer should be carried by the bowl only.
18. For Residential Use Only

**Vicks® Warm Steam Vaporizer** (Model V150SGN Series)  
**Vaporisateur à vapeur tiède Vicks®** (modèle de la série V150SGN)  
**Vaporizador de Vapor Tibio Vicks®** (Modelo de las Series V150SGN)

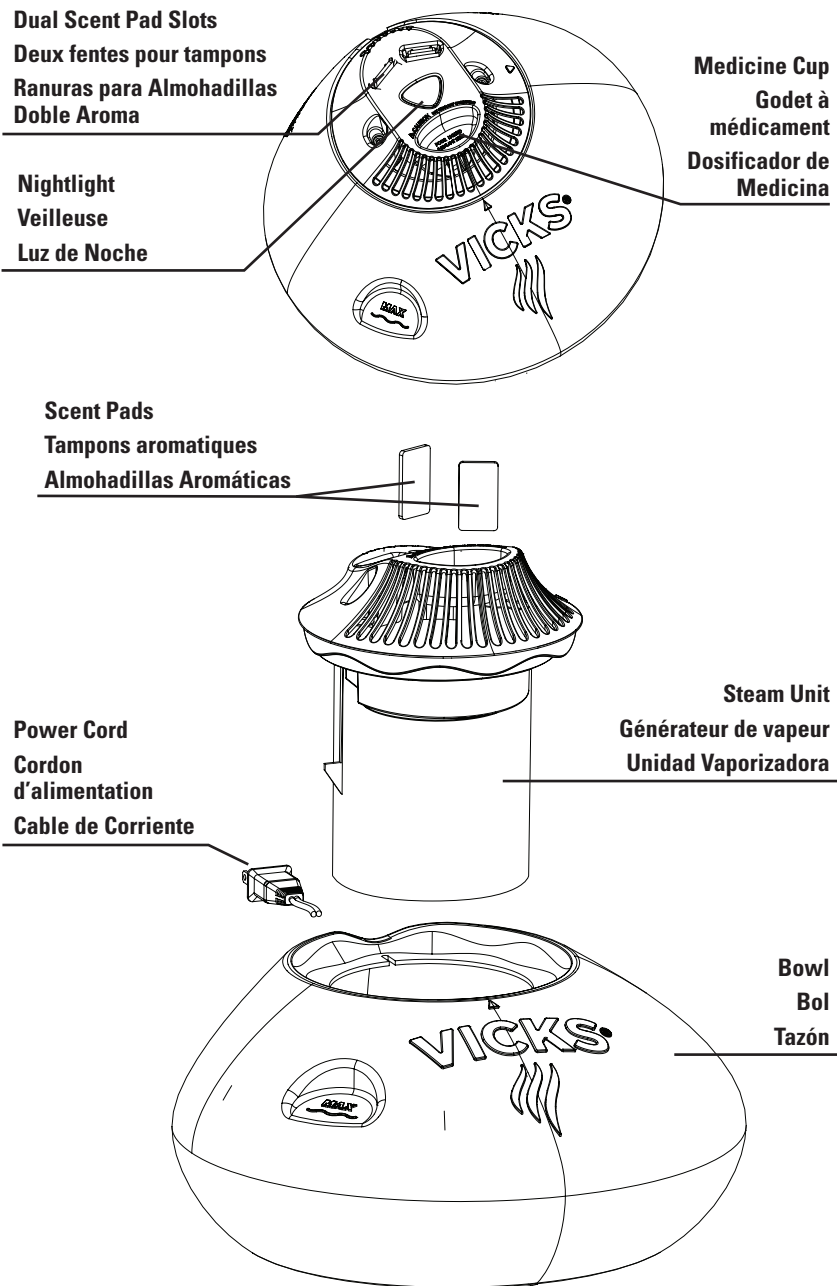## Using Your Vicks Warm Steam Vaporizer

### Instructions

- 1. EXTEND POWER CORD FULLY BEFORE USE.** Failure to do so could cause overheating and possible fire hazard. Always unplug the cord from electrical outlet before removing steam unit or when not in use. Allow the vaporizer to cool for at least 20-30 minutes before handling.
- 2. FILL WATER CONTAINER.** Use finger grips located on the sides of the of the steam unit when handling this part. Remove the steam unit by turning it to align the arrows on the bowl and head. Lift the steam unit from the vaporizer bowl and set the steam unit aside. Fill the container with tap water to **WATER LEVEL MARKING** on bowl. (DO NOT OVERFILL.)
- 3. REPLACE STEAM UNIT.** Hold down the steam unit for ten seconds to allow water to enter the housing and seat properly in water container. If the steam unit floats, container is overfilled.
- 4. PLACE VAPORIZER IN A SAFE POSITION.** Vaporizer should always be placed on a firm, flat surface at least four feet away from bedside and out of the reach of patient and children. Be sure the vaporizer is in a stable position and the power cord is out of the way to prevent the vaporizer from being tipped over. Protect floor, rug or furniture by placing the vaporizer on a non-metallic, waterproof, heat-resistant surface.

**CAUTION:** Do not direct steam flow directly at patient, walls or furniture. You will still obtain desired level of humidity.

**⚠ WARNING:** KEEP OUT OF THE REACH OF CHILDREN. STEAM CAN CAUSE BURNS.

- 5. VAPORIZER INHALANTS AND SCENT PADS** If you choose not to add scent pads or liquid inhalants to the product please continue to step 6.

#### INHALANTS:

**IN U.S.:** For best results fill the medication cup on the steam unit (located directly below nightlight and in front of the steam outlet). Vicks VapoSteam® can also be added directly in the water container (see Vaposteam package directions before using).

Add to Medicine Cup Only

Add to Water or Medicine Cup

**IN CANADA:** For best results fill the medication cup on the steam unit (located directly below nightlight and in front of the steam outlet) with one tablespoon of Vicks VapoSteam®.

**NOTE: In Canada, Vicks VapoSteam® cannot be added directly to the water.**

If you use Vicks VapoSteam® please pay careful attention to the added cleaning directions in the Daily Maintenance and Care and Cleaning sections.

Add to Medicine Cup Only

## VAPOPADS:

- 1) Open scent pad pouch at notch to expose pad. Do not touch exposed pad directly. If you do touch a pad, be sure to wash your hands thoroughly.
- 2) Insert the scent pads into the slots located on the top of the steam unit behind the steam outlet.
- 3) Vapor intensity can be controlled by using 1 or 2 pads. Insert 1 pad into one slot for moderate scent release and insert another pad into the second slot for maximum scent release.
- 4) Before changing or removing the pads unplug the steam unit from the outlet and allow to cool for 20-30 minutes. Pads should be replaced after 8 hours of use.

Replacement VapoPads® can be purchased at most mass and drug retailers. Use only the Vicks® VapoPads® models: In the US, VSP-19, VVP-6 or VBR-5. In Canada, VSP-19-CAN. Other brands of replacement scent pads may not be compatible with this vaporizer.

**CAUTION: DO NOT** place more than one pad in each slot.

**CAUTION: DO NOT** use any other pads or oils in the scent pad slots.


6. **PLUG IN.** Plug the power cord into 110-120 Volts AC outlet in good condition fused for at least 15 Amps. **Do not** use with extension cords. This product has a polarized plug (one blade is wider than the other) as a safety feature. This plug will fit into a polarized outlet only one way. If the plug does not fit fully into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. **DO NOT ATTEMPT TO DEFEAT THIS SAFETY FEATURE.**


**Vaporizer** will produce a gentle flow of steam within a few minutes. **If steaming does not start within ten minutes, you may need to add salt.** Please see step 7 and the *Troubleshooting* checklist.

**Vaporizer will automatically stop producing steam when water has reached minimum level** (about 3/4 inch from bottom of container). **The night-light will remain on**, indicating that the vaporizer is plugged in. **ALWAYS** unplug steam unit and allow to cool for 20-30 minutes before removing from the water container.

**Under conditions of rapid boiling and near the end of each operation, steam may emerge from SAFETY VENTS** (above and on both sides of the steam vent). This condition is normal, preventing over boiling by relieving steam pressure. Do not lift or move the vaporizer while it is in use.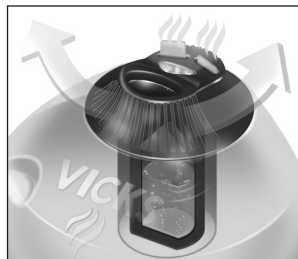

7. **IF YOUR VAPORIZER FAILS TO STEAM AFTER 10-15 MINUTES YOU MAY HAVE LOW MINERAL CONTENT IN YOUR WATER. For areas where tap water has low to moderate mineral content ADD SALT.** For tap water of low to moderate mineral content, ADD 1 TO 2 PINCHES (1/8 teaspoon) of ordinary table salt to the water and mix thoroughly. After 10 minutes if your vaporizer still does not produce steam add another pinch or 2 of salt. Salt has no effect on the purity of the steam or medication.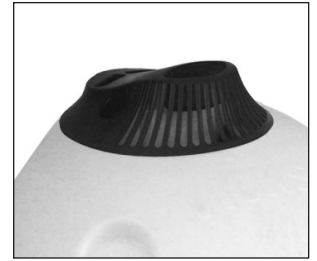

**CAUTION: TOO MUCH SALT CAN CAUSE EXCESSIVE BOILING OR BLOWN FUSE.**

**NOTE:** Please do not add salt to your vaporizer each time you operate the product. Salt will build up over time and you may not need to repeat the process. Before adding salt operate your vaporizer following steps 1-4. If vaporizer does not produce steam after 15 minutes gradually add salt following the above directions.

8. **USE FRESH WATER AND MEDICATION** each time vaporizer is used.

9. **AFTER EACH USE.** Once the vaporizer has been allowed to cool, empty any unused water from the water container. Shake the container gently back and forth to remove any remaining water. Rinse the container thoroughly, drain and wipe with a dry, clean cloth.

**NOTE:** All medicinal ingredients of Kaz Inhalant (or Vicks® VapoSteam® bought in Canada) are released in use, leaving only the mineral oil base in medication cup. This should be emptied and cleaned with a piece of cotton.

**NOTE:** If you are using Vicks® VapoSteam® (bought in the U.S.) be sure any residue in the water container is thoroughly cleaned. Refer to the Care and Cleaning section.

**IMPORTANT:** To maintain proper operation of your vaporizer, be sure to follow the Care and Cleaning instructions.

## Special Notes

### Discoloration/Black Particles

After use, the water residue in the bottom of the container may contain black particles, and in some areas the water in the bottom of the container may discolor. These conditions are normal and do not affect the purity of the steam or medication. The container is easily cleaned with soap and water. Thoroughly rinse and dry the container after cleaning.

### Ventilation

When the vaporizer is used in closed or small rooms, check periodically for proper ventilation, as excessive humidity can cause condensation stains on walls.

## Care and Cleaning

**ALWAYS DISCONNECT THE PLUG FROM THE ELECTRICAL OUTLET AND ALLOW UNIT TO COOL FOR 20-30 MINUTES BEFORE CLEANING. REMOVE STEAM UNIT, LETTING ALL WATER DRAIN BACK INTO THE CONTAINER.**

**DO NOT ATTEMPT TO DISASSEMBLE THE ELECTRODE HEATING UNIT. DOING SO WILL VOID THE WARRANTY AND COULD DAMAGE THE VAPORIZER OR CAUSE PERSONAL INJURY.**

### Daily Maintenance:

Empty any unused water from the water container. Rinse container thoroughly with water, drain and wipe with a dry, clean cloth.

### Weekly Maintenance

1. Empty any unused water from the water container. Rinse container thoroughly with water, drain and wipe with a dry, clean cloth.

2. With the steam unit safely unplugged, soak steam unit in 4 inches of white vinegar for 10 minutes.



Then, over a sink, with fingers covering the bottom hole of the steam unit, add tap water into steam outlet. Cover Steam outlet with other hand, shake vigorously, then empty the water through steam outlet. Repeat several times until black particles are no longer emitted and vinegar odor is gone.

If using Vicks VapoSteam®, make sure any residue is cleaned off steam unit using a mild detergent.

3. Rinse area under the steam guard to remove any dust or dirt particles. With a long toothpick, you can gently poke through the steam outlet and drain hole if they've become obstructed.

#### 4. Disinfecting the Reservoir:

Fill reservoir with water and add 1 teaspoon of bleach per gallon of water. Swish to wet all inner surfaces. Let stand for 20 minutes. Then empty reservoir and rinse with tap water until the bleach smell is gone. Dry with a clean cloth.

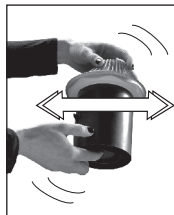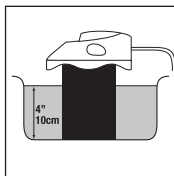### End-Of-Season Maintenance/Storage:

Follow Weekly Maintenance procedure at end-of-season or when the vaporizer will not be used for several days. Be sure to dry all parts and store in a cool, dry location. **DO NOT STORE UNIT WITH WATER INSIDE.**

## Troubleshooting: What To Do If...

### A. Vaporizer does not operate or steams too slowly:

1. Make sure the vaporizer is plugged in and the night-light is lit. If the night-light is not on, you may have a blown fuse, a tripped circuit breaker or a defective electrical outlet. Try a different outlet. You may want to contact an electrician to check the potentially defective outlet.

2. The fault may be the low mineral content of the water in your area. Unplug the vaporizer from the electrical outlet and allow it to cool. Remove the steam unit allowing the water to drain out, add one or two PINCHES (1/8 tsp) of salt to the water and mix thoroughly. Replace the steam unit and plug in the vaporizer. It should steam within 5 minutes. Add salt only by the PINCH, since too much salt can cause excessive boiling or blow a fuse.
3. If the vaporizer has been operating normally and is now failing to steam, it needs to be cleaned. See the "Weekly Maintenance" section of *Care and Cleaning Instructions*.

### B. Vaporizer operates too fast, fuse blows:

**Overboiling, flickering lights, blown fuse or heavy steam flow from Safety Vents generally indicates one of the following causes...**

1. Too much salt was added to the water. To correct the condition: Disconnect the vaporizer from the electrical outlet and allow it to cool. Pour out water and rinse container. Rinse steam unit as described in "Weekly Maintenance" section of *Care and Cleaning Instructions*. Refill with fresh water but **DO NOT ADD SALT**. If the vaporizer now does not operate or steams too slowly, see directions in Section A above.
2. You live in a "hard water" area and minerals in the tap water are causing the vaporizer to steam too rapidly.

To correct the condition: Disconnect vaporizer from electrical outlet and allow to cool. Remove steam unit allowing hot water to first dribble out, pour out water and rinse container. Rinse steam unit as described in "Weekly Maintenance" section of *Care and Cleaning Instructions*. Refill with fresh water but **DO NOT ADD SALT**.

If unit still steams too rapidly, use ½ distilled water and ½ tap water. Food and drug stores usually stock distilled water. In "hard water" areas it is advisable to clean the unit every few days in accordance with the "Weekly Maintenance" section of *Care and Cleaning Instructions*.

### C. Vaporizer is making crackling/buzzing/humming sounds or water container is unusually hot:

1. You may have added too much salt. See section B above.
2. You may live in a "hard water" area. See section B above.
3. This condition can happen when the water level in the container is low. Disconnect vaporizer from electrical outlet, allow to cool, remove steam unit allowing water to first dribble out. Fill the container with tap water to water level marking.
4. This may be an indication your vaporizer needs cleaning. Refer to the "Weekly Maintenance" section of *Care and Cleaning Instructions*.

If you are still experiencing difficulty in operating your vaporizer, please call our Consumer Service Department at 800-477-0457.

## Mail questions or comments to:

Kaz USA, Inc.  
Consumer Relations Dept.  
250 Turnpike Road  
Southborough, MA 01772

Call us toll-free at: **1-800-477-0457**  
E-mail: [consumerrelations@kaz.com](mailto:consumerrelations@kaz.com)  
Or visit our website at: [www.kaz.com](http://www.kaz.com)

**Please be sure to specify model number.**

### ELECTRICAL RATINGS

The Vicks V150SGN Series models are rated at 120V, 60 Hz.

**NOTE:** IF YOU EXPERIENCE A PROBLEM, PLEASE CONTACT CONSUMER RELATIONS FIRST OR SEE YOUR WARRANTY. DO NOT RETURN THE VAPORIZER TO THE ORIGINAL PLACE OF PURCHASE. DO NOT ATTEMPT TO OPEN THE MOTOR HOUSING OR TAMPER WITH INTERLOCK SWITCH YOURSELF, DOING SO MAY VOID YOUR WARRANTY AND CAUSE DAMAGE TO THE PRODUCT OR PERSONAL INJURY.

## 3 YEAR LIMITED WARRANTY

You should first read all instructions before attempting to use this product.

- A.** This 3 year limited warranty applies to repair or replacement of product found to be defective in material or workmanship. This warranty does not apply to damage resulting from commercial, abusive, unreasonable use or supplemental damage. Defects that are the result of normal wear and tear will not be considered manufacturing defects under this warranty.

KAZ USA, INC. IS NOT LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES OF ANY NATURE. ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS LIMITED IN DURATION TO THE DURATION OF THIS WARRANTY.

Some jurisdictions do not allow the exclusion or limitation of incidental or consequential damages or limitations on how long an implied warranty lasts, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you also may have other rights which vary from jurisdiction to jurisdiction.

This warranty applies only to the original purchaser of this product from the original date of purchase.

- B.** At its option, Kaz USA, Inc. will repair or replace this product if it is found to be defective in material or workmanship.
- C.** This warranty does not cover damage resulting from any unauthorized attempts to repair or from any use not in accordance with the instruction manual.
- D.** Return defective product to Kaz USA, Inc. with a brief description of the problem. Include proof of purchase and a \$10 US/\$15.50 CAN check or money order for handling, return packing and shipping charges. Please include your name, address and a daytime phone number. You must prepay shipping charges. We suggest having tracking or delivery confirmation. Send to:

#### In USA.:

Kaz USA, Inc.  
Attn: Returns Department  
4755 Southpoint Drive  
Memphis, TN 38118 USA

#### In Canada:

Kaz Canada, Inc.  
Attn: Returns Department  
510 Bronte Street South  
Milton, ON L9T 2X6  
Canada

Please go to [www.kaz.com](http://www.kaz.com) and register your product under the Customer Care Center and receive product information updates and new promotional offers.

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