
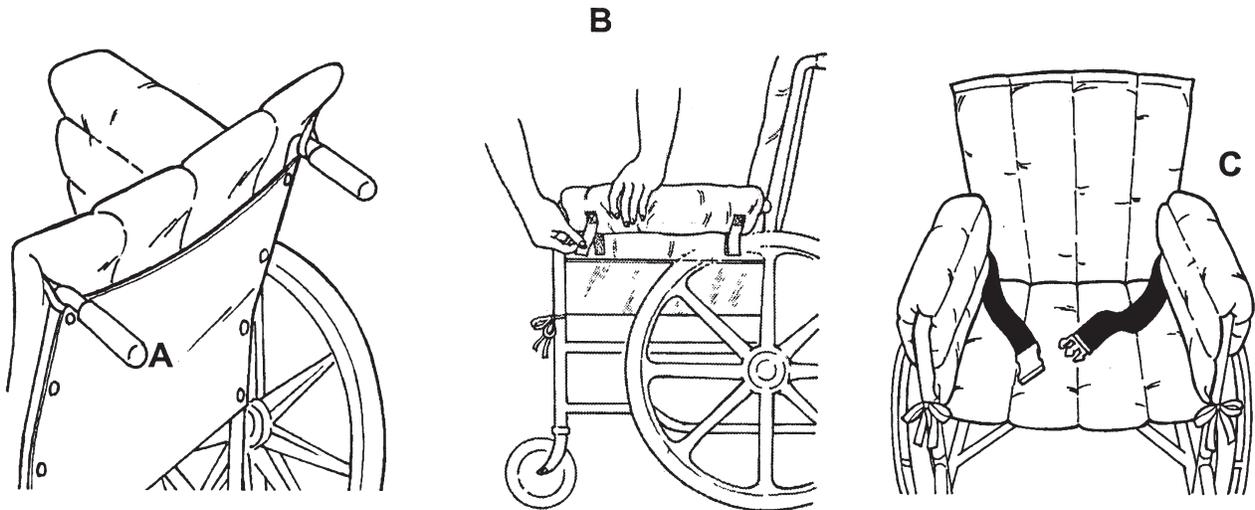


WHEELCHAIR COZY SEAT

Reorder #	Description	Unit
703005	Wheelchair Cozy Seat/ 16"	Each
703006	Wheelchair Cozy Seat/ 18"	Each

Purpose

Polyfoam padded and covered in brushed fabric. Skil-Care's Wheelchair Cozy Seat provided comfort and warmth. It protects against contact with metal wheelchair parts and cushions the armrests.

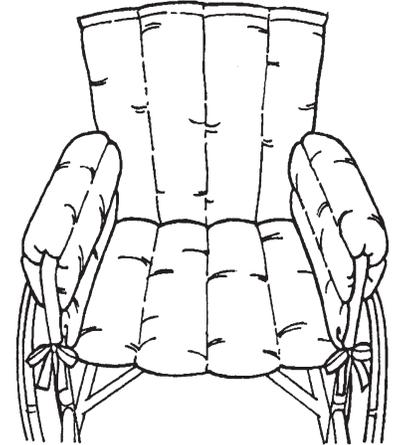

Easy to Use

1. Place the Cozy-Seat on the wheelchair and loop the elastic backrest bands over the handles. See illustration A.
2. Secure the Velcro^R closures on the left and right sides of the armrests. Secure the ties on the left and right sides. See illustration B.
3. If the wheelchair user requires a safety or positioning belt, the belt can be brought through the openings at the lower left and right sides of the backrest. See illustration C.

Easy Care

The Cozy Seat is machine launderable. Do not wash or dry at temperatures that exceed 180°F (82°C). Do not use bleach.