

Re-Order #	Description	Size	U/M
703262	Footrest Extender with 1.5" separator	19"W x 11"D x 9.5"H	Ea.
703264	Footrest Extender with 3" separator	19"W x 11"D x 9.5"H	Ea.
703267	Optional Separator - 1.5" thick	8" x 7" x 1.5"	Ea.
703269	Optional Separator - 3" thick	8" x 7" x 3"	Ea.

Note: The above models fit most 16", 18" and 20" wide wheelchairs.



Purpose

The Wheelchair Footrest Extender with Leg Separator is a versatile product whereas it provides several important functions. It provides support to control foot drop, prevents the residents feet from slipping behind the footrest, protects the feet from harsh metal components on the wheelchair and also ensures that the feet are separated.

Application

1. Before seating resident, lock wheel brakes and rotate footrests to the out-of-the-way position. Assist patient into chair and position him/her as far back on the seat as possible. Reposition footrests.
2. Place the Footrest Extender Leg Separator on the footrests. Cushion section on footrest, padded backboard with straps against footrest support bars (*Fig. 1*).
3. Place straps behind footrest support bars, secure buckles and tighten straps (*Fig. 2*). Note that the Skil-Care label is in the upward position
4. The Footrest Extender Leg Separator may be positioned to accommodate various degrees of ankle plantar flexion. This may be accomplished by securing the top straps to the upper section of the footrest support bars as shown in the illustration (*Fig. 3*).
5. The product can be used with or without the leg separator. The leg separator cushion can be easily moved by rotating the block of foam in the forward direction. To reinstall, make sure that both sides with Velcro are placed in contact with the base.

Easy Care

Skil-Care's Footrest Extender Leg Separator is made from durable vinyl for long lasting use. To clean, just spray and wipe with a damp cloth.

Fig. 1

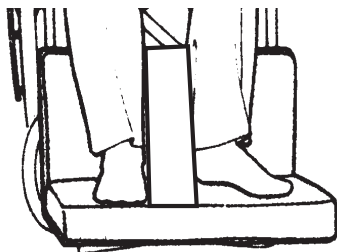


Fig. 2

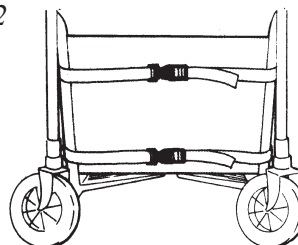


Fig. 3

