


PELVIC HOLDER

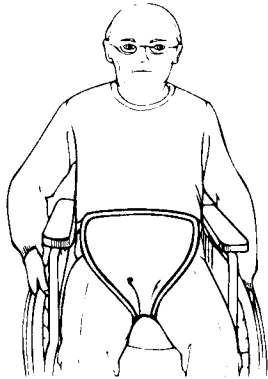

Reorder #	Fabric	Size	Unit
<input type="checkbox"/> 303311	Polyester	Small-Medium	1
<input type="checkbox"/> 303321	CooLine Mesh	Small-Medium	1
<input type="checkbox"/> 303312	Polyester	Medium-Large	1
<input type="checkbox"/> 303322	CooLine Mesh	Medium-Large	1

Caution: Federal law restricts this device to sale by or on the order of a physician or licensed health care professional.

Indications - The Skil-Care Pelvic Holder is recommended for use with a patient who:

- Slides down in the wheelchair
- Requires restraint while in the wheelchair

Contraindications for Use - Do not use this device with a patient who:

- Has a history of easily removing restraints
- Requires posture support while seated in the wheelchair
- Is in a bed, geri-chair, or lounge chair. This device is intended for wheelchair use only.

Application Select proper size for your patient.

Sizing Table for Skil-Care Products (see note)

Binding Color	Size	Weight (lb./kg.)
Brown	Small-Medium	up to 170/up to 77
Blue	Medium-Large	over 170/over 77

Note: Device size to patient weight relationships are approximate. Another criterion for device size selection may include patient torso size. It is the responsibility of the licensed health care professional to determine the proper size for each patient.

1. Before seating patient in wheelchair, make certain that both brakes are locked and both footrests are positioned to the sides of the chair. Seat patient as far back in the wheelchair as possible. Position footrests to the front of the chair and place patient's feet on them.

2. Position Pelvic Holder between patient's legs placing the loops at the patient's waist.

3. Thread the left rear strap through the left front loop. Thread the right rear strap through the right front loop. See illustration A.

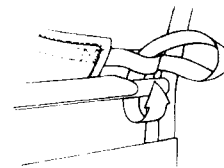


Illustration A

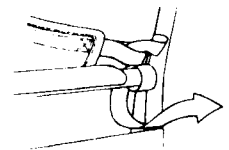


Illustration B

4. Pull both straps completely through the loops so that the device fits snugly. Note: Straps should not be so tight as to interfere with patient's breathing. You should be able to slide your open hand between the device and the patient.

5. Wrap the right strap once around the right metal bar behind the armrest pad. Repeat on left side. See illustration B.

6. Bring both straps behind the wheelchair and tie them together in a bow. See illustration C.

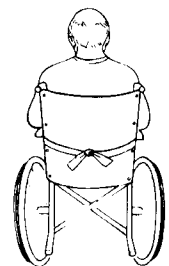


Illustration C

See **important warnings and cautions** on reverse side.

Laundering

This device should not be laundered or dried at temperatures that exceed 180°F (82°C). Do not use bleach.

See **important warnings and cautions** below.

Always...


Always inspect condition of device.



Always read package insert to determine if device is intended for wheelchair and/or bed use.



Always monitor patients in restrictive devices. Make certain to follow your facility's procedures for periodic release, toileting and exercise.

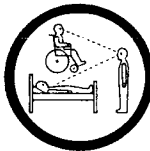

Never...

Never use a device that has worn or frayed fabric or webbin. Discard damaged units immediately. Do not alter products.



Never use a Skil-Care device as a restraint in a motor vehicle.



Never place a patient wearing a restrictive device in a place where he/she is not easily monitored.



(Date of issue: 9/3/96)

Skil-Care[™]
CORPORATION

Advanced design for nursing and therapy.™

29 Wells Ave., Yonkers, NY 10701

914-963-2040

800-431-2972

Fax 914-963-2567

Visit us online
at Skil-Care.com